



Quality Products since 1937

Posey® Roll Guards

BED SAFETY

The Posey Roll Guard is an alternative to side rails and is less restrictive. The Roll Guards help prevent high-risk patients from rolling out of bed, while allowing sufficient room for patient comfort. The Roll Guard easily attaches to the bed with quick-release buckles. The wipe clean anti-microbial vinyl covered guards are easily removed from the outer cover. Available in brushed polyester or vinyl.

- Helps prevent patients from rolling out of bed, while allowing patient to see out.
- Soft, flexible Roll Guard acts as a barrier to protect the patient from banging into side rails when in use.
- Will fit standard twin and hospital style beds.

REF 5700 Roll Guard, brushed polyester cover

REF 5700SC Roll Guard, vinyl cover

REF 5701 Replacement cover, brushed polyester

REF 5701SC Replacement cover, vinyl

Dimensions

Overall Dimensions: 25"L x 43"W

Inner gaps between rolls: 23"W

Roll Dimensions: 25"L x 9"Dia.

Straps (4): 33"L x 1"W

APPLICATION INSTRUCTIONS:

1. Place the Roll Guard on top of the mattress with the quick-release buckles away from the patient. The Roll Guards should be on top of the attachment straps.
2. Position the Roll Guard approximately 20" from the headboard.
3. Wrap the quick-release straps once around the movable part of the bed frame and secure the buckles.



LAUNDERING INSTRUCTIONS:

Brushed Polyester Cover Only:

100°F / 38°C
WASH LUKEWARM BLEACH AS DIRECTED ON CONTAINER DRY LOW

Vinyl Cover and Internal Bean-Filled Bags:

Wipes Clean with mild disinfectant.



J.T. Posey Company • 5635 Peck Road, Arcadia, CA 91006-0020 USA
Phone: 800-447-6739 • Fax: 800-767-3933 • Web: www.posey.com



MDSS GmbH
Schiffgraben 41
D-30175 Hannover, Germany
© 2007 J.T. Posey Company. All rights reserved.



M5605 041207



Quality Products since 1937

Posey® Roll Guards

BED SAFETY

The Posey Roll Guard is an alternative to side rails and is less restrictive. The Roll Guards help prevent high-risk patients from rolling out of bed, while allowing sufficient room for patient comfort. The Roll Guard easily attaches to the bed with quick-release buckles. The wipe clean anti-microbial vinyl covered guards are easily removed from the outer cover. Available in brushed polyester or vinyl.

- Helps prevent patients from rolling out of bed, while allowing patient to see out.
- Soft, flexible Roll Guard acts as a barrier to protect the patient from banging into side rails when in use.
- Will fit standard twin and hospital style beds.

REF 5700 Roll Guard, brushed polyester cover

REF 5700SC Roll Guard, vinyl cover

REF 5701 Replacement cover, brushed polyester

REF 5701SC Replacement cover, vinyl

Dimensions

Overall Dimensions: 25"L x 43"W

Inner gaps between rolls: 23"W

Roll Dimensions: 25"L x 9"Dia.

Straps (4): 33"L x 1"W



LAUNDERING INSTRUCTIONS:

Brushed Polyester Cover Only:

100°F / 38°C
WASH LUKEWARM BLEACH AS DIRECTED ON CONTAINER DRY LOW

Vinyl Cover and Internal Bean-Filled Bags:

Wipes Clean with mild disinfectant.



J.T. Posey Company • 5635 Peck Road, Arcadia, CA 91006-0020 USA
Phone: 800-447-6739 • Fax: 800-767-3933 • Web: www.posey.com



MDSS GmbH
Schiffgraben 41
D-30175 Hannover, Germany
© 2007 J.T. Posey Company. All rights reserved.



M5605 041207