

Mirage Activa™ LT

NASAL MASK

Fitting Guide



1

- Turn the MicroFit dial to position 24, so that the forehead support is fully extended.



2

- Disconnect the lower headgear clip, hold the mask over the nose and pull the headgear over the head.
- Reconnect the lower headgear clip ensuring it clicks into place.



3

- Keep the bottom straps loose to allow room for the cushion to inflate. Use two fingers as a guide.



4

- Connect one end of the air tubing to the swivel and the other end to the airflow device.




5

- Turn on the airflow device.




6

- While lying down, adjust the dial for comfort and seal.

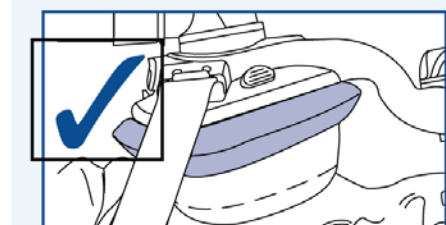

- If the mask is fitted correctly, the cushion should be smooth and fully inflated.

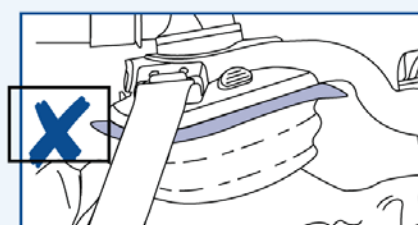

- If fitted incorrectly, the cushion will be creased and underinflated.