

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2012 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

clever-lite Is walker



EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

A large, stylized version of the "drive" logo, oriented vertically. The word "drive" is written in a bold, white, lowercase sans-serif font. A small yellow square is positioned above the letter "i". A trademark symbol (TM) is located at the top right of the word.

REV2.3.26.13

assembly instructions

Do not attempt to adjust or operate walker without carefully reading all instructions in this manual.

1. Remove all contents from carton.
2. Open walker and lower seat to down position.

safety precautions

1. Please make sure the walker is fully opened, seat is down and secured in the down position.
2. Test walker for stability before applying full weight to hand grips.
3. Replace tires when significantly worn or damaged.
4. Do not have anyone push you while you are seated on the walker.
5. To fold walker, lift seat to upright position. Deploy one trigger release and swing brace to closed position. Repeat on the other side.

NOTE: This is a walking aid only. It is not to be used as a transportation device.

WEIGHT CAPACITY: 300 LBS.

push down lock

Always engage and disengage rear push locks on both sides simultaneously.

TO SLOW DOWN OR COME TO A TEMPORARY STOP
Push down slowly by applying weight on the handles until unit slows down or comes to a temporary stop.

TO RELEASE REAR PUSH LOCKS
Remove weight until wheels start moving freely.

