**drive**<sup>TM</sup>

[www.drivemedical.com](http://www.drivemedical.com)

Rev.1.04.03.15

folding steel commode

inodoro de acero  
plegable

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
item # RTL11158KDR

**drive**<sup>TM</sup>

[www.drivemedical.com](http://www.drivemedical.com)

Rev.1.04.03.15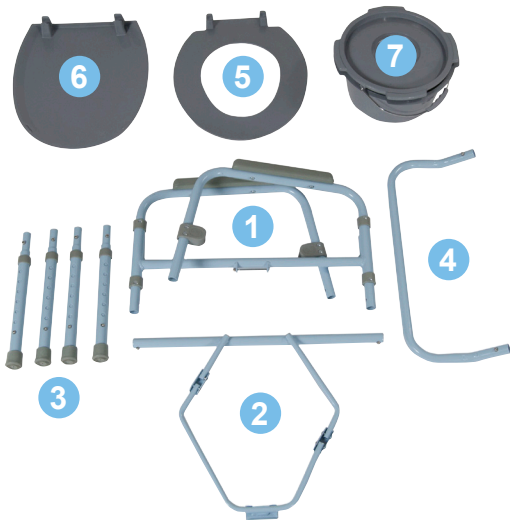


## parts list


- |                  |                |
|------------------|----------------|
| 1. Commode frame | 5. Toilet seat |
| 2. Bucket frame  | 6. Cover       |
| 3. Legs (4)      | 7. Pail & lid  |
| 4. Backrest      |                |

## assembly instructions


1. Open commode frame (1) as shown below and place on the floor.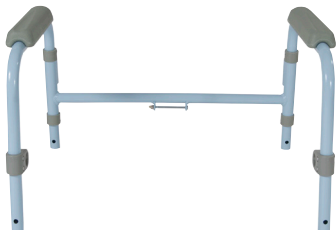

2. Slide the hook on the bucket frame (2) over the receptacle located on the back of the commode frame.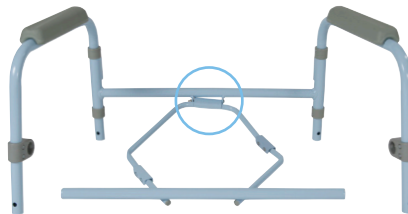

3. Secure bucket frame in place by inserting the front cross brace into the receptacles located on the front of the commode frame. The push buttons will “click” in place when they are secure.

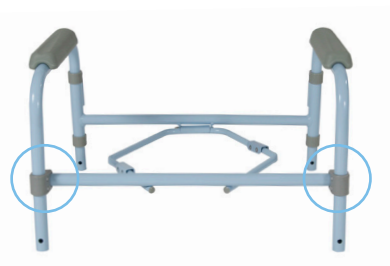4. Turn the frame upside-down. Insert the legs (3) into each receptacle by depressing the push pins and sliding into place. The push buttons will “click” in place when they are secure.

5. Turn the commode right-side up. Insert the backrest (4) into the receptacles located on the back of the commode frame by depressing the push pins and sliding into place. The push buttons will “click” in place when they are secure.

Note: The angle of the backrest should face outwards.

6. Attach the toilet seat (5) and cover (6) to the back of the frame. The toilet seat hooks should fit inside the hooks of the cover.

7. Lower commode bucket into frame and return seat to the down position.

8. Assembled commode

## adjusting leg height

1. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes, per illustration.
2. Be sure legs are locked into place by pushing down on or by pulling legs.

## additional precautions

1. Toilet seat must be in down position before use. Failure to do so may result in breakage of commode and possible injury.
2. Do NOT install commode without first reading and understanding this assembly and instruction sheet. If you cannot understand these instructions, contact a healthcare dealer or technical professional before attempting to install. Otherwise, injury or damage may occur.
3. Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
4. Make certain that the height-adjusting snap buttons fully protrude through the same respective holes of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
5. Ensure ALL screws, nuts and/or bolts are tightened.
6. Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
7. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
8. Users with limited physical strength should be supervised or assisted while using commode.

**Weight Capacity: 350 lbs (158 Kg)**

## lifetime limited warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.


This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2014 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington, N.Y. 11050 USA Made in China

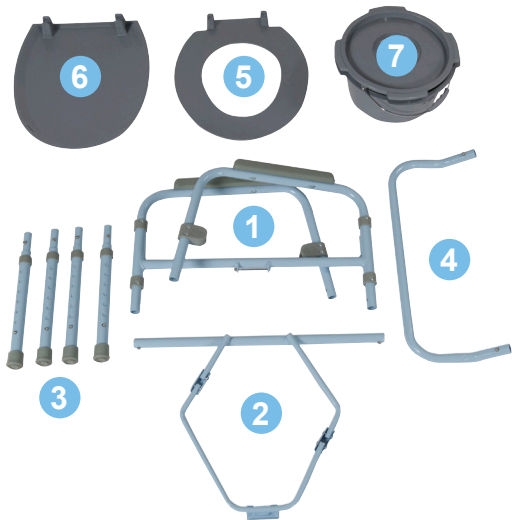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© 2014 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.