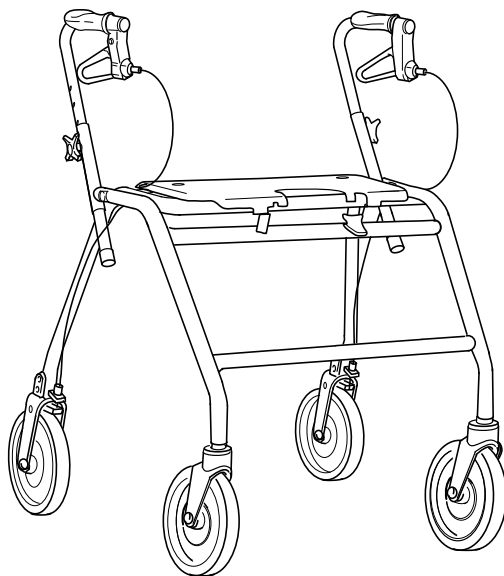
DOLomite MAXI

 Do not use as a wheelchair or transport chair.

OPERATING INSTRUCTIONS

Study the operating instructions carefully before use. The walker is suitable for use both indoors and outdoors.

SAFETY

- Do not use as a wheelchair or transport chair.
- Do not under any circumstances stand on the walker.
- Use seat for stationary seating only.
- Never sit and push with your feet.
- Do not go up or down stairs or use on escalator.
- Both brakes must be in the parking position when the walker is used as a seat.
- The walker must be fully unfolded when used as walking support. N.B.
- The walker may only be used as a walking support.
- Be extra careful when using the walker on a sloping surface.
- Be extra careful when carrying heavy loads in the basket. Loads may only be carried in the basket or on the tray. Max load in basket 20 lbs. (9 kg) and on tray 11 lbs. (5 kg).

ACCESSORY

- A) Mounting of basket.
- B) Secure the tray using the plastic plugs.

FOLDING SEAT







The seat can be folded up to provide a larger walking space, and in order to increase accessibility.

FOLDING

1. Press down the safety catch.
2. Pull the seating tube upwards.

OPERATING POSITION

Unfold the walker by pressing down on the seating tube. Check that the safety catch is in the locked position when the walker is completely unfolded.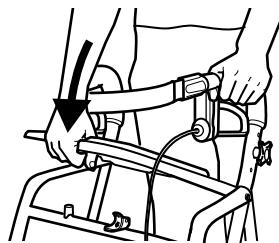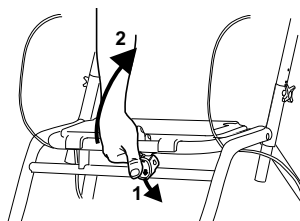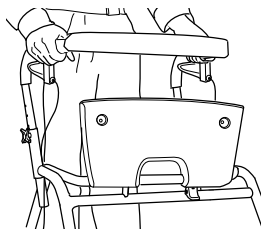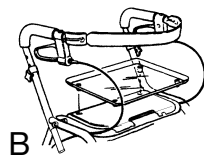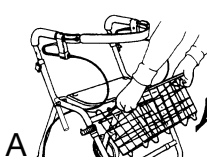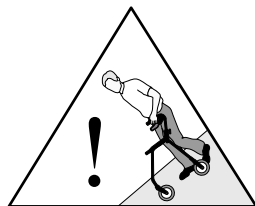

BRAKES

Apply the brakes by pulling **both** brake-handles upwards.

PARKING BRAKE

Press down carefully for parking brake. A clear sound is heard when the brake is placed in correct position. N.b. Always place both brakes in parking position. Pull upwards to loosen the brake.

Both brakes must be in the parking position when the walker is used as a seat.

ADJUSTMENT OF BRAKE EFFECT

For desirable braking strength, adjust the brake wire.







Suitable tools 8 mm and 10 mm spanner (not included).

HANDLEBARS

To adjust the handle to a suitable height, stand behind the walker with shoulders level. The handles should be at the same height as the user's wrist. A therapist may suggest different handle heights to accommodate the special needs of a user.

For adjustment of handles with holes:

Loosen the knob completely, remove the screw. Adjust the handle to suitable height. Fix the screw into the hexagonal hole and fasten the knob properly.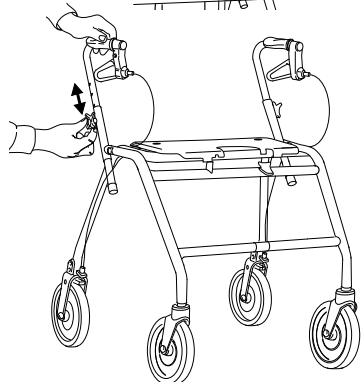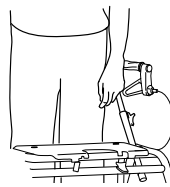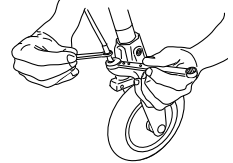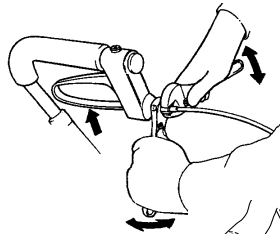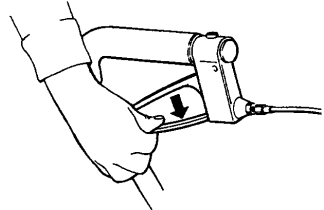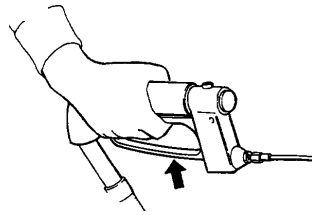

MAINTENANCE – SAFETY

To ensure good performance and safety you should regularly check that the following is carried out:

- Clean the rollator with solvent-free household cleaner. Keep the wheels clean.
- Ensure that screws and adjustments are properly tightened.
- **Immediately contact the nearest dealer if a fault should appear.**
Never use a defective walker. Do not attempt repairs yourself!

N.B. Your walker should only be used as a walking aid!

User's max. weight: DOLOMITE MAXI 200 kg, 440 lbs. (31stones).


DOLOMITE will not accept responsibility for adaptations, faulty repairs etc that are outside their control.

INSTRUCTIONS FOR DESTRUCTION OF PRODUCT AND PACKAGING:

When the product can no longer be used, it and its accessories, together with packaging material, should be sorted properly. The individual parts can be separated and handled according to the materials description.

The label on the **DOLOMITE MAXI** provides all important technical information.

- 1: Article number
- 2: Product name
- 3: Serial number
- 4: Date of manufacture
- 5: Maximum user weight
- 6: NB: read the operating instructions
- 7: Manufacturer's contact information.

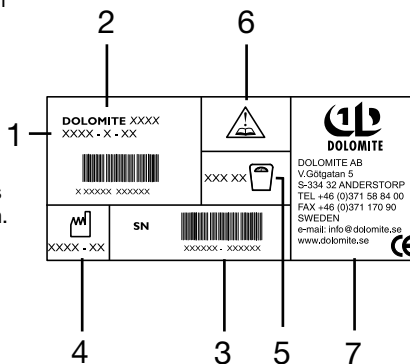DOLOMITE AB, Box 55, 334 21 ANDERSTORP, SWEDEN
TEL +46 (0)371 58 84 00, FAX +46(0)371 170 90
E-mail: info@dolomite.se, www.dolomite.se

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DOLOMITE AB, Box 55, SE-334 21 ANDERSTORP, SVEZIA