


Parts List

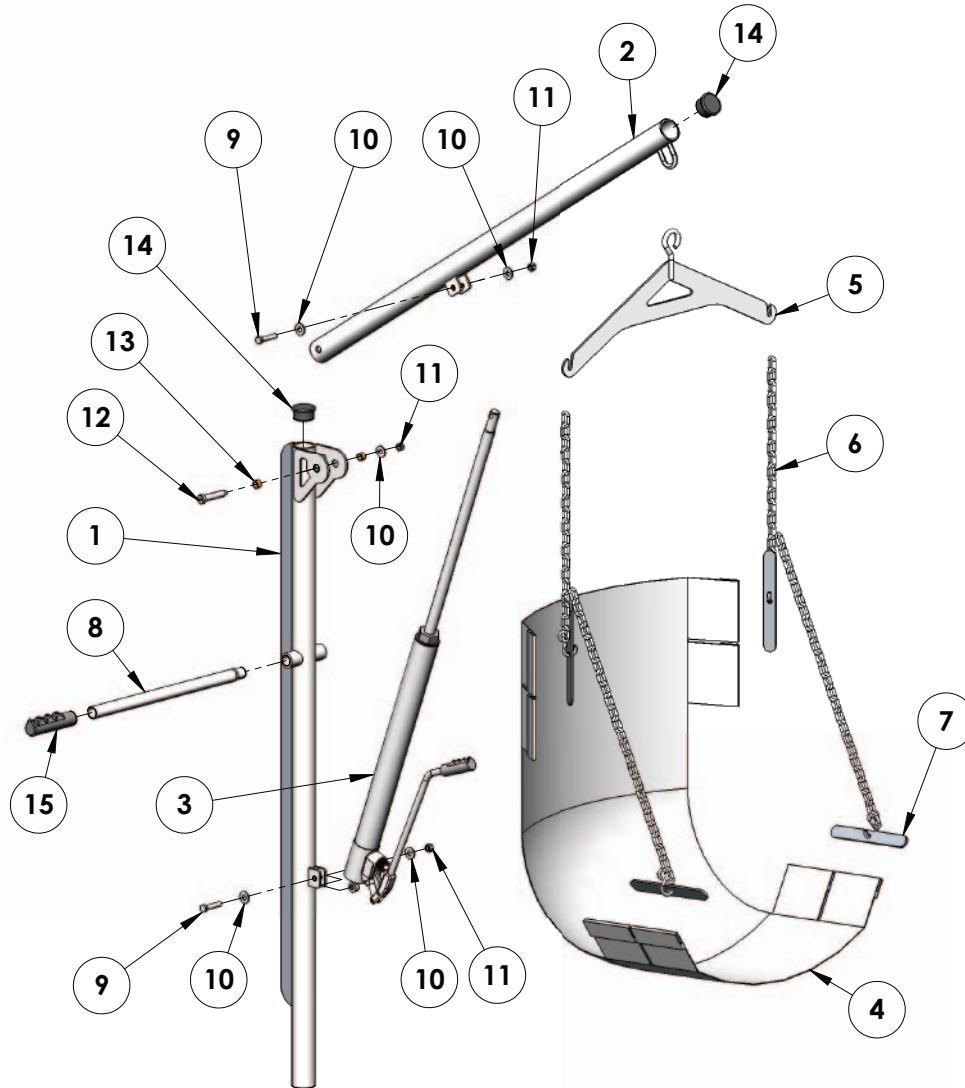

ITEM #	QTY	PART #	DESCRIPTION
1	1	EZWM-0804EZMA	MAST WELDMENT, EZ
2	1	EZWM-0804EZBA	BOOM WELDMENT, EZ
3	1	F-03EZCYC	HYDRAULIC RAM, FOR EZ POOL LIFT
4	1	F-034EZS	SLING ONLY, STANDARD ADULT SIZE
5	1	F-038EZHB	HANGER BAR, STANDARD
6	1	F-031EZCH	CHAIN SET, STANDARD
7	1	F-032EZB	SLING BARS ONLY, SET OF 4
8	1	A-0808EZTB	EZ TURN BAR
9	2	BH 3/8 X 1-3/4	BOLT, 316 SS, 3/8-16 X 1 3/4 HCS
10	5	WF 3/8	WASHER, FLAT, 3/8, 316 SS
11	3	NN 3/8	NUT, 316 SS, NYLOCK, 3/8"-16
12	1	BS 1/2 X 2-1/4 18-8	BOLT, 18-8 SS, 1/2 X 2 1/4 SHOULDER, 3/8-16 THREAD
13	2	BRB SS-1620-3	BUSHING, BRONZE SLEEVE, SS-1620-3, 1/2 X 5/8 X 3/16
14	2	CCF-1 3/4-14-20	PLUG, 1 3/4 ROUND, PLASTIC, BLACK, .065 WALL
15	1	RUBBER HAND GRIP	GRIP, RUBBER HAND GRIP