



The **Wheel**® is a uniquely effective device. It is physician designed to give immediate temporary relief from pain and soreness of muscle related disorders.

Physiology

Strength, speed, flexibility and endurance are known qualities of healthy muscle. A muscle's ability to prolong these desirable qualities is directly dependent on an unimpeded, nutrient-rich and oxygen-laden blood supply.

The **Wheel**® provides the many documented benefits of trigger point therapy and myofascial release. Its purpose is to extinguish myofascial trigger points and augment the qualities of healthy muscle.

Indications


The **Wheel**® is successfully used in the musculotendinous therapy of:
carpal tunnel syndrome • occipital neuralgia
tension headache • TMJ disorder
epicondylitis • tendinitis
and a host of other myofascial lesions.

CPT code 97140 • E 0935



Experience the Magic


- Hold the handlegrip in the palm of your hand...apply pressure with the thumb and index finger.
- The **Wheel**® is rolled lengthwise over the involved muscle, using short back & forth passes.
- Use the **Wheel**® to search and find myofascial trigger points...these areas are tight, knotty, ropy and tender.
- After locating the trigger point, continue the rolling process for about 30 seconds.
- On a pressure-pain scale of 10 – use 5...the procedure should produce a “feel-so-good” pain.
- Chronic, stubborn trigger points may initially require several short, therapeutic sessions per day.
- Your goal is a healthy muscle free of pain, tightness, tenderness and other signs of offending trigger points.

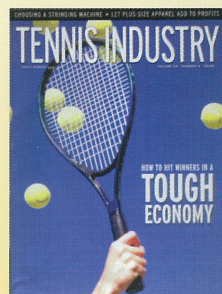

“The **Wheel** can effectively warm up muscles before workouts.”



Relief is just a Roll-A-Way!

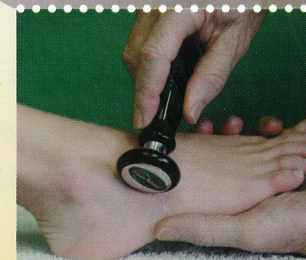



"It works!"

Ask my husband, Robbie."

Genie Windholz • Atlanta, GA

"The Wheel allows me to spend more hours writing and editing than I otherwise could."

John C Lowe

The Fibromyalgia Research Foundation
Houston, TX

"I recommend the Wheel for all my patients with symptoms of carpal tunnel syndrome."

Sydney Wade, PT • Athens, Greece

"I dispense the Wheel for my patients to use at the job site."

R G Hotchkiss, DC • Rockford, IL



Manufactured by

RPI of Atlanta

Atlanta, GA USA
www.intracell.net
Patents Granted and Pending

ICT 04/304

