


After Use

- To deflate, disconnect each of the air hoses from the Camel. **NOTE:** The initial 'rush' of air as the hoses are disconnected from the Camel, can be reduced by first disconnecting the air hose from the Airflo Plus and then pressing each of the buttons on the hand control.
- Roll up the Camel with the hand control in the hollow (A) - fig. 10 secure the straps and place in the carrying bag - fig. 11

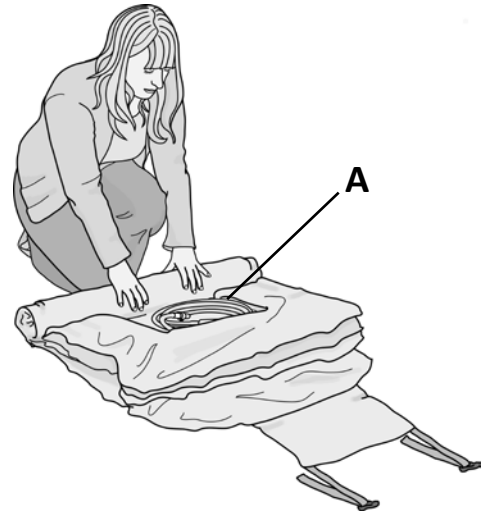

10

Maintenance

Other than routine cleaning, the Camel is maintenance free.

We recommend that a thorough examination of the Camel is carried out regularly; the frequency of examination may need to be increased depending on usage and operating environment.

If required, Mangar International can provide the above inspection within an annual service contract. Please contact our Customer Service Department for further details.

Routine Cleaning

Fully inflate all sections of the Camel, sponge clean with a non abrasive proprietary liquid cleaner or disinfectant and dry thoroughly. **Do not dry using hot air.**

Disinfecting

The Camel should be fully inflated and pre-cleaned by washing with detergent and water, thoroughly rinsed and then disinfected using a 1:10 solution of household liquid bleach and water (1 part bleach to 10 parts cold water). Leave the solution on the Camel for approximately 10 minutes before thoroughly rinsing off with clean cold water. Allow to air dry before storage. **Do not dry using hot air.**



11