

# Quick Start Guide

**NOW WITH  
MORE BATTERY,  
LESS WEIGHT!**

**For Use With Models:  
XYC100B-SPT & XYC100B-PRO**



# GETTING STARTED

This guide will familiarize you with the LifeChoice Activox™ Portable Oxygen Concentrator (POC) and its accessories. For more detailed information and additional instructions, please consult the Operator's Manual included with your unit.

## LIFECHOICE ACTIVOX™ POC STANDARD PACKAGE INCLUDES THE FOLLOWING:




- 1 LifeChoice Activox POC
- 2 Versatile 4-Way Carry Case  
Use as a backpack, shoulder bag, waist pack or briefcase
- 3 Adjustable Straps  
For use with 4-Way Carry Case
- 4 AC Power Adapter
- 5 DC Power Adapter
- 6 Standard Single Lumen 7-Foot-Long Nasal Cannula
- 7 Accessory Bag

Model identified on unit and packaging labels. See **REF**.

An External Battery is available for purchase which provides up to 3 hours of additional portable time.



# CONTROL PANEL

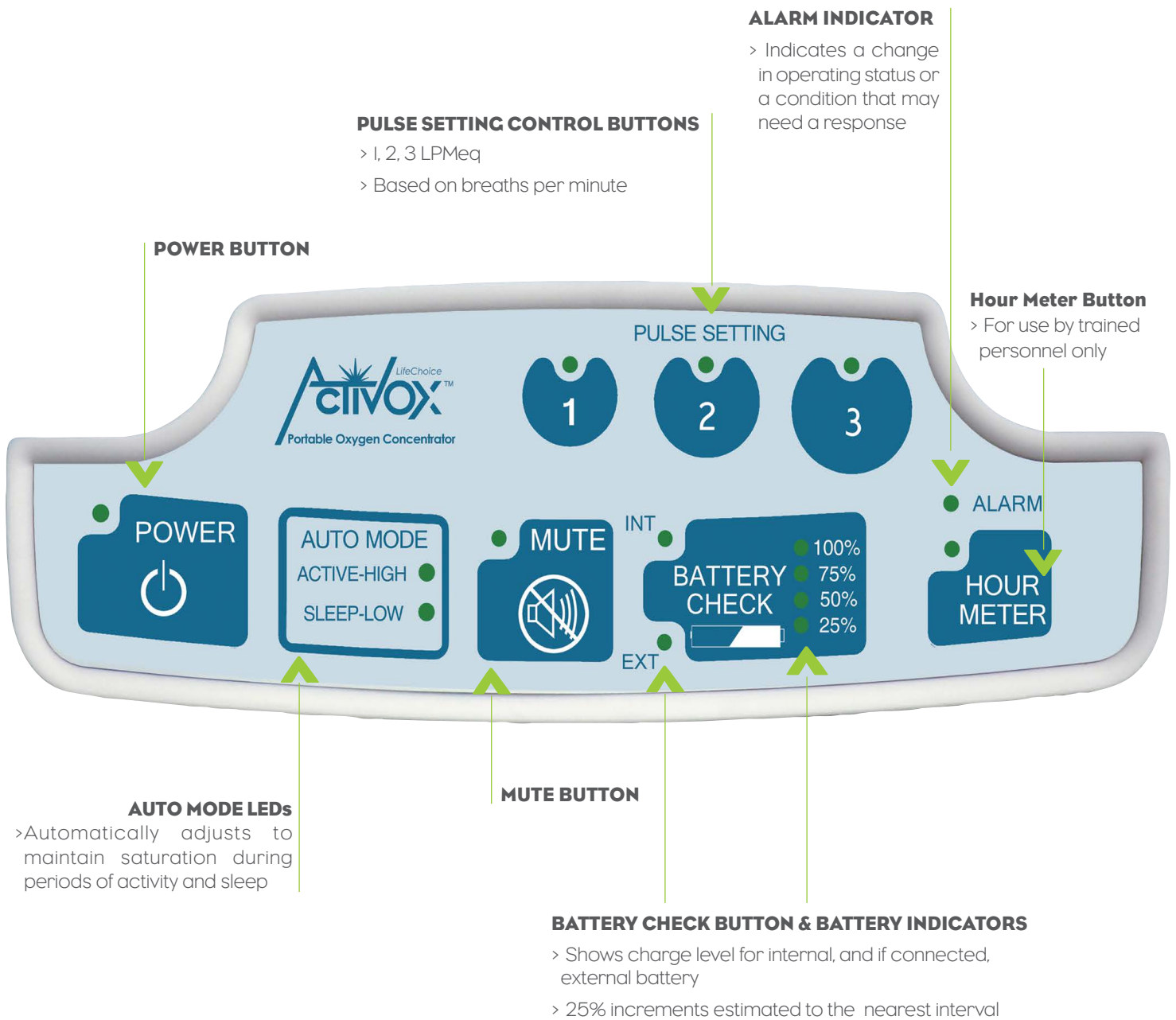

# SETTING UP YOUR LIFECHOICE ACTIVOX POC

Before using your unit for the first time, please review the items below.



1. The LifeChoice Activox POC is recommended and designed to be operated and stored in its carrying case at all times.



2. Charge the unit before the first use. To charge the unit, connect the AC Adapter to the unit as shown in the picture and then connect the AC Charger to an electrical outlet.



3. To charge the External Battery (if purchased), start with the instructions above. Then connect the adapter on the External Battery to the connector port on the unit. It can take up to 7.5 hours to fully charge both the unit and the External Battery.



4. After the initial charge period has ended, press the Battery Check button to verify that the 100% LED is illuminated.



5. Connect the nasal cannula securely to the cannula nozzle fitting on the unit but DO NOT put it in your nose until after the unit has been powered on. Breathing through the cannula during start-up may prevent the system from starting.



6. Always operate your unit in a well-ventilated space. Make sure air inlets and outlets are not obstructed.

# OPERATING YOUR LIFECHOICE ACTIVOX POC

Follow the steps below to power on and operate your unit.



1. Press and hold the Power Button until you hear a short beep and see all LEDs briefly illuminate. Allow 3 minutes for the unit to warm-up.

**CAUTION:** DO NOT place the cannula in your nose until the unit is fully powered on. Breathing through the cannula during start-up may prevent the system from powering on.



2. Press the Pulse Setting (1, 2 or 3 LPMeq) prescribed by your doctor. The green LED above the selected setting will illuminate and flash each time a breath is detected.

3. Place the nasal cannula in your nose and breathe normally. The unit detects when you begin to take a breath and delivers oxygen in PULSE-WAVE™ doses. Because this type of delivery is very gentle, you may not feel the pulse of oxygen. To verify that oxygen is being delivered, reference the LED above your selected Pulse Setting. This LED will flash every time a breath is detected and a bolus of oxygen is delivered.
4. To recharge the unit and/or External Battery\*, you can use the AC or DC Adapter/Charger. Connect the adapter to the connector port on the unit and the charger to a source of electricity.
5. To turn the unit off, press and hold the Power Button for 2 seconds until the green Power LED light and the unit shut off. The internal fans may continue to operate for an additional period of time until the temperature inside the unit is sufficiently cooled.

### Battery Life Timetable

Battery Type		Pulse Setting		
		1 LPMeq	2 LPMeq	3 LPMeq
Internal Battery**	Sport Model	5.5 Hours	4 Hours	3.25 Hours
	Pro Model	12.5 Hours	7.25 Hours	6.25 Hours
External Battery**	All Models	3 Hours	3 Hours	2 Hours

\* If an External Battery was purchased, please refer to the Operator's Manual for detailed charging and operating instructions.

\*\* Based on Breaths Per Minute.



# Technical Support

For technical or emergency support, please contact the local medical equipment company that provided or sold you the LifeChoice Activox POC.



1.800.220.0977

[www.InovaLabs.com](http://www.InovaLabs.com)



Manufactured and Distributed by Inova Labs.  
3500 Comsouth Drive, Suite 100, Austin, TX 78744 USA  
Corporate Office 1.512.617.1700

© 2014 Inova Labs, Inc. All rights reserved. Made in the USA.

I2M-O18 REV B Feb 2014



Follow us @InovaLifeChoice