


Anti-Embolism Stocking Quick Reference Sizing Chart

Albahealth Anti-Embolism Stockings are color-coded to make proper sizing easy. Color coding is found in the following locations:

- Toe Port
- Packaging
- Tape Measures
- Quick Reference Chart Column

- X-Small
- Small
- Medium
- Large
- X-Large
- 2X-Large
- 3X-Large

KNEE HIGH Measurement Guide

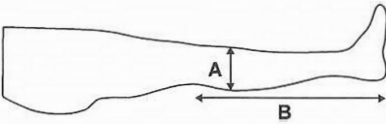

Size
Circumference of "A"

Length
Distance of "B"


CAUTION: DO NOT APPLY these stockings if any measurement exceeds the maximum recommendation. Consult a physician. Federal law restricts this device to sale by or on the order of a physician.

| KNEE LENGTH | | X-Small | Small | Medium | Large | X-Large | 2X-Large | 3X-Large |
|------------------------|--------|-------------|------------|------------|------------|------------|------------|------------|
| SHORT (Yellow Top) | Calf | — | 10" to 12" | 12" to 15" | 15" to 17" | 17" to 19" | 19" to 21" | — |
| | Length | — | 12" to 14" | 13" to 15" | 14" to 16" | 15" to 17" | 16" to 18" | — |
| REGULAR (White Top) | Calf | 8.5" to 10" | 10" to 12" | 12" to 15" | 15" to 17" | 17" to 19" | 19" to 21" | 21" to 24" |
| | Length | 13" to 15" | 14" to 16" | 14" to 16" | 16" to 18" | 16" to 18" | 17" to 19" | 17" to 19" |
| LONG (Blue Top) | Calf | — | 10" to 12" | 12" to 15" | 15" to 17" | 17" to 19" | 19" to 21" | — |
| | Length | — | 16" to 18" | 16" to 18" | 18" to 20" | 18" to 20" | 19" to 21" | — |

INDICATIONS: The use of Anti-Embolism Stockings is indicated in situations where there is likely to be impaired venous return from the lower extremities as in: Surgical patients, Pregnancy, Varicose veins, Geriatric bedfast patients, Chronic venous insufficiency, Edema secondary to cast removal, Patients recovering from phlebitis

CONTRAINDICATIONS: Anti-embolism stockings are not recommended for patients with: severe atherosclerosis or other severe ischemic vascular diseases, including absence of pedal pulses; local skin disorders such as dermatitis, open ulcers, infections, recent vein ligation, skin graft or gangrene in which stockings would interfere, massive peripheral edema associated with pulmonary edema, and severely deformed extremity

THIGH AND WAIST HIGH Measurement Guide

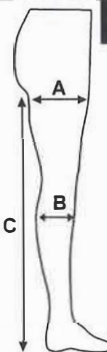

Size
Circumference of "A" and "B"

Length
Distance of "C"

CAUTION: DO NOT APPLY these stockings if any measurement exceeds the maximum recommendation. Consult a physician. Federal law restricts this device to sale by or on the order of a physician.

| THIGH LENGTH | | Small | Medium | Large | X-Large | 2X-Large | 3X-Large |
|------------------------|--------|------------|------------|------------|------------|------------|------------|
| SHORT (Yellow Top) | Calf | 10" to 12" | 12" to 15" | 15" to 17" | 17" to 19" | 19" to 21" | — |
| | Thigh | 20" to 25" | 20" to 25" | 20" to 25" | 25" to 32" | 25" to 32" | — |
| | Length | up to 29" | up to 29" | up to 29" | up to 29" | up to 29" | — |
| REGULAR (White Top) | Calf | 10" to 12" | 12" to 15" | 15" to 17" | 17" to 19" | 19" to 21" | 21" to 24" |
| | Thigh | 20" to 25" | 20" to 25" | 20" to 25" | 25" to 32" | 25" to 32" | 30" to 36" |
| | Length | 29" to 33" | 29" to 33" | 29" to 33" | 29" to 33" | 29" to 33" | 29" to 33" |
| LONG (Blue Top) | Calf | 10" to 12" | 12" to 15" | 15" to 17" | 17" to 19" | 19" to 21" | — |
| | Thigh | 20" to 25" | 20" to 25" | 20" to 25" | 25" to 32" | 25" to 32" | — |
| | Length | 33" to 36" | 33" to 36" | 33" to 36" | 33" to 36" | 33" to 36" | — |
| WAIST LENGTH | | Small | Medium | Large | X-Large | 2X-Large | 3X-Large |
| REGULAR (White Top) | Calf | 10" to 12" | 12" to 15" | 15" to 17" | 17" to 19" | 19" to 21" | — |
| | Thigh | 20" to 25" | 20" to 25" | 20" to 25" | 25" to 32" | 25" to 32" | — |
| | Length | 29" to 33" | 29" to 33" | 29" to 33" | 29" to 33" | 29" to 33" | — |
| LONG (Blue Top) | Calf | 10" to 12" | 12" to 15" | 15" to 17" | 17" to 19" | 19" to 21" | — |
| | Thigh | 20" to 25" | 20" to 25" | 20" to 25" | 25" to 32" | 25" to 32" | — |
| | Length | 33" to 36" | 33" to 36" | 33" to 36" | 33" to 36" | 33" to 36" | — |