

ALLEVYN[®] Heel is conquering new heights in versatility

With its exclusive, highly versatile cup shape and hydrocellular technology, ALLEVYN Heel maintains a moist wound environment for rapid healing of difficult-to-fit body prominences.

In addition, its thick, non-adhesive padding reduces trauma on removal and delivers increased comfort and protection against skin breakdown—all with a friction-reducing, bacteriaproof outer film.

For clinical challenges that demand peak conformability and performance, stake your claim with ALLEVYN Heel.



For heels and other body prominences, ALLEVYN[◇] Heel can fit your need

Unique, specialized shape

Cup shape offers superior anatomical fit and comfort. Designed to fit gently around the heel and other body prominences.

Maintains optimal moisture balance

Unique tri-laminate polyurethane hydrocellular foam technology promotes a moist wound environment for rapid healing.

Breathable outer film layer allows fluid to evaporate from dressing.


Thick, non-adhesive padded protection

5.9-mm-thick hydrocellular foam pad delivers comfort and protection against skin breakdown.

Non-adhesive foam technology minimizes disruption of healing tissue and risk of trauma upon removal.

Friction-reducing, bacteriaproof and waterproof outer film

Contributes to greater patient comfort and protection against infection.

	Moderate to High Exudate
Non-Adhesive	 <p>ALLEVYN Heel For heels, crevices, and other difficult-to-fit or protruding areas requiring form-fitting, padded protection</p>

Respond to the wound healing environment with ALLEVYN.

Go to www.allevyn.com for more information, or call 1-800-876-1261 to request a free sample.

Precautions

Do not use ALLEVYN Heel in conjunction with oxidizing agents such as hypochlorite solutions or hydrogen peroxide solutions as these can break down the absorbent hydrocellular component of the dressing. Reddening of the skin around the wound has been reported rarely following the use of ALLEVYN dressings. In some cases this relates to irritation of fragile skin; in others, wound exudate remaining in contact with periwound skin for prolonged periods may be the cause. Infrequently, cases of sensitivity to the dressing have also been reported. If reddening or sensitization occur, discontinue use and consult a healthcare professional.

Product	Description	Pcs/Pkg	Pkgs/Unit	Order Unit
66007630	Heel	5	6	CA

Indications

The ALLEVYN Heel dressing is ideal for use on granulating wounds on the heel or similar anatomical areas. On necrotic wounds, it can be used in conjunction with INTRASITE[®] gel.

Application

Apply the white surface to heel and secure with a light bandage or with tape (eg, HYPAFIX[®]) or with a dressing retention sheet (eg, OPSITE[®]). ALLEVYN Heel can be left in place for up to 7 days, depending on the level of exudate.

Wound Management

Smith & Nephew, Inc. 970 Lake Carillon Drive Suite 110 St. Petersburg, FL 33716 USA	Customer Care Center 1 800-876-1261 T 727-392-1261 F 727-392-6914	www.smith-nephew.com www.allevyn.com
---	--	--