

FREEDOM Grip® Plus - Model 502

For Home Style Beds - Install This Side Up

Installation Instructions

1. Rotate the "L" leg on the handle until the snap button locks into one of the handle holes. Use same hole for both legs.
2. Insert "L" legs of the handle assembly into the "U" tube attached to the bed board.
3. Slide bed board between the mattress and box spring until the handle is snug against mattress.
4. Position handle at the individuals waist when lying down.
5. Adjust handle to desired height.

Warning: This product is not recommended for individuals with cognitive problems (difficulty in understanding their surroundings). If you are taking medications that affects your balance or your ability to sit or stand you should consult with a physician before using this product. If the product is not properly used, entrapment, serious injury or death could result. This product is not intended to carry the full weight of an individual. The product is only intended to assist in bed mobility and rising to a sitting position. These instructions must be carefully followed to insure that the product is properly assembled. If you have and questions or concerns please contact MTS at 1-800-854-4687. Not for use on hospital style beds.



Recommended weight limit 250 lbs.

Ph: 800-854-4687 www.mtsmedicalsupply.com

Patented 5,400,450

Copyright, Mobility Transfer Systems, Inc. 2008