Catéteres Urinarios, Intermittentes y Masculinos (PVC)

Male Intermittent Urinary Catheters (PVC)

R_x
ONLY

INDICATIONS

Intermittent catheters are indicated for routine short term drainage of the bladder. The catheter is inserted through the urethra.

CAUTIONS, PRECAUTIONS & OBSERVATIONS

- Check the catheter's integrity prior to insertion.
- Select a replacement catheter if required.
- Do not use if package has been previously opened or damaged.
- Do not touch the catheter tip before insertion.
- If there is no urine flow or other complications occur (e.g. pain, difficult insertion or removal, bleeding, signs of urinary tract infection or discharge) please consult a healthcare professional.
- After incorrect insertion, catheters must be discarded and not re-used. A new catheter should be selected. Used catheters should always be discarded.

INSTRUCTIONS FOR MALE USE

NOTE: Healthcare professionals use aseptic technique

1. Wash hands thoroughly with soap and water. (Figure 1)
2. Withdraw the foreskin and wash the penis with soap and water. (Figure 2)
3. Peel open the package. (Figure 3)
4. Lubricate the catheter with water-soluble lubricant. (Figure 4)
5. Take the pre-lubricated catheter out from the package.
6. Choose the position that is most comfortable. (Figure 5,6,7)
7. Hold penis up toward your stomach. Do not squeeze penis, as this can block the opening. (see Figure 8)
- 8A. Straight Tip: Slide catheter slowly into the opening of urethra and into bladder until urine starts to flow. (Figure 9A)
- 8B. Coudé Tip: Keep the angled tip facing the front of you e.g. towards the ceiling, slide catheter slowly into the opening of urethra and into bladder until urine starts to flow. (Figure 9B)
9. Point penis and the funnel end of the catheter down towards container. (Figure 10)
10. When the urine starts to flow, push the catheter in one or two cm to make sure that it is fully inside the bladder.
11. To make sure bladder is emptied completely, remove the catheter slowly and stop if more urine starts to flow. (Figure 11)
12. When the bladder is empty, hold penis up toward stomach again and withdraw the catheter slowly. If uncircumcised, pull foreskin forward. (Figure 12)
13. Discard catheter after use. (Figure 13)
14. Wash hands. (Figure 1)

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© 2011 ConvaTec Inc.
GentleCath is a trademark of ConvaTec Inc.
ConvaTec Inc.