THERMOPHORE[®]

Brand Heat Pack

Deep-Heat[™] Therapy

How to Use **Thermophore[®]**

Heat Pack for Deep-Heat[™] Therapy
to Help Relieve the Pain of:

- ✕ Arthritis
- ✕ Muscle Spasm
- ✕ Sprains
- ✕ Fibromyalgia
- ✕ Joint Pain
- ✕ Sciatica
- ✕ TMJ
- ✕ Bursitis

Please read and follow all instructions in this booklet,
as well as those on the product and packaging.

Enhancing Comfort, Enhancing Treatment, Enhancing Lives![™]

 **BATTLE CREEK[®]**
Equipment

307 West Jackson Street / Battle Creek, MI 49037-2369
800-253-0854 www.BattleCreekEquipment.com

IMPORTANT SAFETY INSTRUCTIONS

DO NOT DESTROY

DANGER

To reduce the risk of burns, electric shock and fire, this product must be used in accordance with the following instructions:

- 1. READ AND FOLLOW ALL INSTRUCTIONS IN THIS BOOKLET, ON THE BOX AND ON THE PAD.**
- 2. DO NOT USE WHILE SLEEPING.**
- 3. DO NOT USE ON AN INFANT.**
- 4. THIS PAD IS NOT TO BE USED ON OR BY AN INVALID, PARALYZED PERSON, SLEEPING OR UNCONSCIOUS PERSON, OR A PERSON WITH POOR BLOOD CIRCULATION UNLESS CAREFULLY ATTENDED.**
- 5. DO NOT USE ON AREAS OF INSENSITIVE SKIN.**
- 6. THE MOMENTARY SWITCH WAS DESIGNED FOR YOUR SAFETY. USE ONLY AS DIRECTED.**
- 7. BURNS CAN OCCUR REGARDLESS OF CONTROL SETTING, CHECK SKIN UNDER PAD FREQUENTLY.**
- 8. NEVER USE PAD WITHOUT COVER IN PLACE.**
- 9. DO NOT USE PINS OR OTHER METALLIC MEANS TO FASTEN THIS PAD IN PLACE.**
- 10. DO NOT SIT ON OR CRUSH PAD-AVOID SHARP FOLDS.**
- 11. NEVER PULL THIS PAD BY THE SUPPLY CORD. DO NOT USE THE CORD AS A HANDLE. LOOP CORD LOOSELY WHEN STORING. TIGHT WRAPPING MAY DAMAGE CORD AND INTERNAL PARTS.**
- 12. CAREFULLY EXAMINE INNER COVER BEFORE EACH USE. DISCARD THE PAD IF INNER COVERING SHOWS ANY SIGN OF DETERIORATION.**
- 13. THIS UNIT IS NOT DESIGNED FOR USE IN CONJUNCTION WITH ANY TYPE OF MASSAGE APPARATUS. SUCH USE MAY CAUSE HAZARDOUS DAMAGE TO INTERNAL COMPONENTS AND WILL VOID THE WARRANTY.**
- 14. SAVE THESE INSTRUCTIONS.**

Using Your Thermophore Classic™ or Thermophore MaxHEAT™ Fleece Cover

Two-Layer Cover with Moist-Sure™ technology - No water to add!

Slide the Thermophore® heating unit into the fleece cover provided with your system. DO NOT use the heating unit without the cover, and do not add additional layers of insulating cover.

DO NOT add water to the Thermophore Classic™ or MaxHEAT™ Systems. The outer cover is designed to absorb moisture from the atmosphere between uses and release the moisture when heat is activated. Your skin should feel slightly damp from the water vapor condensed during the treatment, but will not be dripping wet. If you feel your cover is no longer producing enough moisture, you may simply need to wash it, or to let it rest in an area of high humidity (perhaps near your shower or laundry, where the air contains more moisture) for a few hours.

Caution: Do not put the inner heating unit of the Thermophore® Pack in or near water.

It is very important to keep the heating unit flat inside the cover. Folded or overlapping areas of the heating unit could intensify the heat. This could damage the unit, burn the user or cause a fire. Feel inside the cover with your hand to make sure the heating unit is flat and wrinkle-free. Before each use, check the heating unit to be sure it has remained flat inside the cover.

Position the cover and heating pad assembly over the area of the body needing heat therapy. If you have purchased the Petite 4" x 17" heat pack, it has a Velcro strap to help hold it in place on small joints such as the ankle, elbow, neck or jaw. The Multi-function Muff unit has three Velcro straps, to allow using it flat or forming it into a tube to enclose both hands, a thigh, knee or elbow.


**Please note the UL-required warning:
Do not sit or lay on the Thermophore® heat pack.**

Using Your Thermophore Classic Plus™ or Thermophore MaxHEAT Plus™ with the Moisture-Plus™ Cover

Three-Layer MoisturePlus!™ Cover

The MoisturePlus!™ cover allows you to safely add extra moisture when necessary because of extremely dry atmospheric situations. Many modern homes are so well insulated and sealed so tightly that heating and cooling systems can remove much of the normal moisture. Many areas simply have very dry climates. The cover will absorb moisture from the air naturally, but if more moisture is needed or desired, the steps below will show you how to safely add a damp cloth.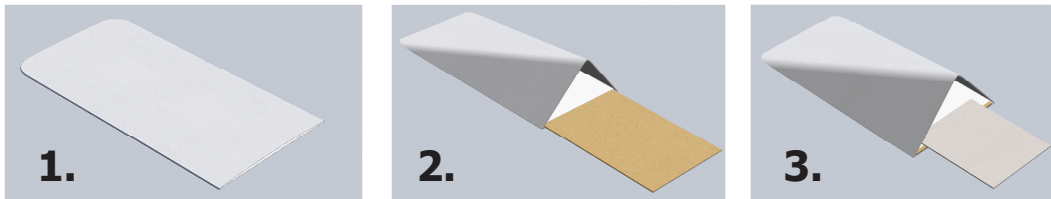

1. Start by laying the MoisturePlus!™ cover out flat and smooth. This type of cover is sometimes called a “stupe” cover (sounds like “stoop”), since it allows you to add moisture, if needed, without getting the fabric of the heat pack wet. Either side can be “Up”.

2. Slide the Thermophore® heat pack into the bottom section of the cover, under the white nylon layer. Make sure the heat pack is kept smooth and flat inside the cover.

3. Slightly dampen a cloth or towel (a fleece square is included for your convenience) and squeeze out the excess moisture. Slide the damp cloth into the top section of the cover, over the white nylon layer. The nylon layer is necessary to protect your heating unit from becoming damp. The cloth should not be wet, just damp. It does not need to fill the entire area of the vinyl pocket, but should be flat and smooth.

It is very important to keep the heating unit flat inside the cover. Folded or overlapping areas of the heating unit could intensify the heat. This could damage the unit, burn the user or cause a fire. Feel inside the cover with your hand to make sure the heating unit is flat and wrinkle-free. Before each use, check the heating unit to be sure it has remained flat inside the cover.

Position the cover and heating pad assembly over the area of the body needing heat therapy. The heat pack layer should be on the outside (farthest away from your body) and the section with the damp cloth on the inside (closest to your body).

If you have purchased the Petite-size 4" x 17" heat pack, it also has a Velcro strap to help hold it in place on small joints such as the ankle, elbow, neck or jaw.

Please note the UL-required warning:

Do not sit or lay on the Thermophore® heat pack.

Using Your Switch:

Classic Easy-Comfort™ Switch

When you are ready to begin your therapy session, position the Thermophore® Deep-Heat™ System over the area you wish to treat. Direct contact with bare skin or over light clothing produces the most benefit. Make sure the unit is lying smoothly on your body, so it can conform to your body's curves and make close contact with painful areas.

Insert the plug into any convenient 110-volt outlet. The pack has a polarized plug (one blade is wider than the other) as a safety feature. This plug will fit into a polarized outlet only one way. If the plug does not fit fully into the outlet, reverse the orientation of the plug in the outlet. If it still does not fit, contact a qualified electrician. Do not attempt to defeat this safety feature.

Press the lever on the hand control and hold it down. **WARNING:** This switch was designed for your safety - USE ONLY AS DIRECTED. The switch is spring-loaded so the Thermophore® Pack is "Off" when the switch is not held "On". You control the heat by holding the switch in your hand. Please do not switch it "On" any other way.

When the Thermophore® Pack gets as hot as you like, release the switch until you begin to notice that the unit feels cool. Repeat the "On and Off" procedure for another quick pulse of intense moist heat, and repeat as needed to stimulate circulation, relax muscle spasms, and give an adequate deep-heat therapy treatment.

Using Your Switch: **MaxHEAT™ One-Touch™ Switch**

The One-Touch™ switch is a user-friendly electronic timer and temperature control. The small red light at the top of the switch will remain lit whenever the heating unit is plugged in properly. When you are ready for a surge of heat, just press the larger red “Power” button in the center of the switch display.

All four green bars should light up in a blinking cascade rotation, to indicate a total time available of 20 minutes, divided into in four 5-minute time segments. One light bar will become a solid green light each time the “Power” or “Resume” button has been activated for a total of five minutes, while the remaining bars continue to blink. This will help you monitor the length of your heat therapy treatment, but you can end the treatment at any time, or extend it longer if you prefer, with the “Power” button.

As soon as the MaxHEAT™ therapy pad becomes as hot as is comfortable for you, just press the yellow “Pause” button, located directly below the “Power” button. This feature allows you to customize the heat level by allowing the heating pad to cool a bit while the timer function pauses, your body adjusts to the heat level, and painful tissues begin to relax. The “Pause” button will blink yellow, and the cascading green lights on the timer will pause. Whenever you feel that more heat is needed, push the yellow button a second time to “Resume” heating. The MaxHEAT™ pad is designed to get very hot, very quickly. Please stay alert, monitor the heat level, and do not let the heat level become uncomfortably hot.

If the “Pause” feature is left in place for 20 minutes, or the heating function has been activated for a total of 20 minutes by using either the “Power” or “Resume” function, the unit will automatically shut off as a safety feature. If you feel that you need extended heat therapy at that point, just press the red “Power” button again to activate a new 20-minute timer cycle.

General Instructions for all models:


Because of individual needs, the length of time for treatment may vary from person to person. We recommend treatments of 30 minutes or less, with a minimum cooling period of 60 minutes between heat therapy sessions. Please consult your physician if you have any questions about using heat therapy, or if you require more intense heat therapy than our product supplies. Should unusual symptoms appear, discontinue use of the product and consult your physician.

When finished with your therapy session, remove the plug from the outlet. Store the Thermophore® Heat Pack in the original box or lay it out flat. Loop the cord loosely – tight wrapping may damage the

cord and internal parts. You may even want to hang the fleece cover in an area of high humidity, so that it can re-absorb moisture to be ready for the next therapy session.

You Are Not Alone!

Arthritis pain is a serious problem, affecting one of every three adults in the US. As our population ages, the CDC estimates these numbers will soon double. Moist heat is also very effective in treating back pain, fibromyalgia, TMJ, muscle spasms, chronic pain, or speeding injury recovery.

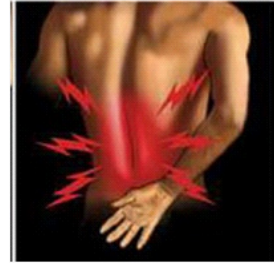Many people you know can benefit from using the Thermophore® Deep-Heat™ System. Of course, we hope you will encourage them to purchase their own Thermophore® Pack – but, if you do lend your unit to a friend, please include this booklet and urge the user to read it to assure safety, satisfaction and maximum therapeutic benefit.

WARNING-Please stay alert and use your Thermophore® Pack carefully. For it to work properly, it must be capable of becoming very hot. Do not sit or lay on your Thermophore® Pack. Do not use over areas of the body with poor circulation or insensitive skin. Diabetics should be especially careful when applying heat. Those who are incapacitated should be closely supervised at all times during their use of the Thermophore® Pack. Do not use a Thermophore® Pack as a bed- or foot-warmer. It gets too hot to be used for this purpose. Do not use your Thermophore® Pack on animals or small children. When in doubt about the advisability of using heat therapy because of any situation or health condition, please consult your personal physician.

Trouble-shooting:

If you have any questions regarding the use of your Thermophore® Pack, please contact our Customer Service department at 800-253-0854 or go to BattleCreekEquipment.com.

If your Thermophore® Pack fails to heat, we suggest the following:

1. Make sure the plug fits properly and firmly into the electric outlet, and that the outlet is functioning properly. Try plugging the unit into another outlet which is on a different electrical circuit.
2. Inspect the electric cord. Make sure there is no break in the wire of the cord. The areas of highest stress are normally next to the plug, switch, or where it is attached to the Thermophore® Pack.
3. Check the heavy fabric cover of the heating unit. It may eventually become somewhat discolored, especially near the wires, but excessive scorching or isolated darker areas may be caused if the unit is folded during use, creating damage to internal components. If there is evidence that this has occurred, the unit should be replaced.

WARRANTY –

During the first two years after purchase or three years after manufacture (whichever is longer), we will replace this product free of charge to the original owner if it has a manufacturing defect or, if in use as directed, it does not perform as stated.

Please be sure to save your sales receipt showing the date of purchase. If no receipt is available, we will use the serial number stamped on each heating unit to establish date of manufacture and determine warranty eligibility.

If you believe your Thermophore® unit has failed to work properly within this warranty period because of a manufacturing defect, please contact our office for assistance in confirming warranty eligibility.

In order to obtain warranty service, you will be requested to send the unit to us for evaluation. Be sure to include your name, address and daytime phone number, along with a brief description of nature of the problem. We may also request a copy of your purchase receipt.

To save yourself time and money, please contact us before sending anything to our office.

Phone: 800-253-0854 8am to 5pm EST
Email: CustomerService@BattleCreekEquipment.com
Mail: Battle Creek Equipment Company
307 West Jackson Street
Battle Creek, MI 49037

*Please note that this warranty does not cover incidental or consequential damages or costs connected with any such defect. While this warranty gives you specific legal rights, you may also have other rights which vary from state to state.

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Email : CustomerService@BattleCreekEquipment.com

Adresse Postale : Battle Creek Equipment Company
307 West Jackson Street
Battle Creek, MI 49037

Warranty Registration Card

Please note: Warranty registration is not proof of purchase date. Please save your receipt showing date of purchase. If a warranty issue arises and no receipt is available, we will determine warranty eligibility based on the serial number of the unit.

Name: _____

Address: _____

City: _____

State and zip code: _____

Purchased at: _____

If this product was recommended by a healthcare professional, please provide their name and address:

Comments? _____

Please return to:

Battle Creek Equipment Co
307 West Jackson Street
Battle Creek, MI 49037

307 West Jackson Street / Battle Creek, MI 49037-2369
800-253-0854 www.BattleCreekEquipment.com

Try these other fine products from Battle Creek Equipment®!


Joint-Specific and Multi-Use styles, as well as individual cold packs

Gentle overnight warmth can help to relieve pain for up to 48 hours.

Replacement covers are also available for all Thermophore® styles!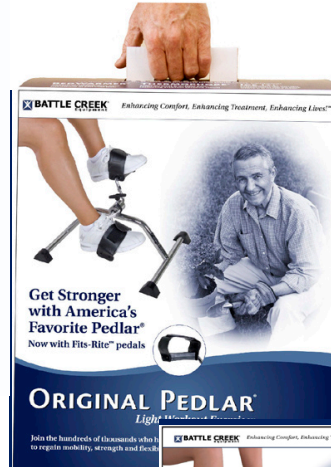

Convenient, portable exercise for arms or legs, right in your own living room.

Enhancing Comfort, Enhancing Treatment, Enhancing Lives!™

©2010 Battle Creek Equipment Company
Part#2859 Universal Thermophore HTUB