seca

Technical data

Dimensions	
Width:	44 cm
Depth:	55 cm
Height:	146.5 cm
Weight:	approx. 19 kg

Scale	
Max. load	160 kg /400 lbs
Fine graduations	0.1 kg / 0.25 lbs

Measuring rod	
Measuring range	89 – 210 cm / 36 – 83 "
Graduations	5 mm / 1/4 "

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GB

Warranty

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be honoured only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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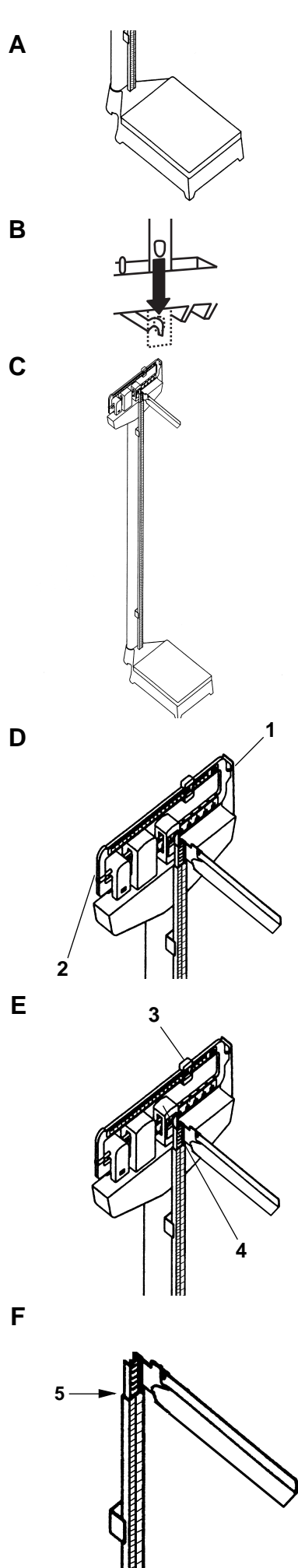
seca ltd.
Medical Scales and
Measuring Systems
40 Barn Street
Birmingham B5 5QB
England
Tel. +44 121 643 9349
Fax +44 121 633 3403
<http://www.seca.co.uk>

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seca corporation
Medical Scales and
Measuring Systems
1352 Charwood Road
Suite E
Hanover, MD 21076, USA
Tel. +1 410 694 9330
Fax +1 410 694 9680
<http://www.secacorp.com>

17-10-07-358b / 02

GB

Congratulations!

By purchasing the seca 700 personal scale you have acquired a highly accurate and sturdy piece of equipment.

For more than 150 years, seca has been putting its experience at the service of health, and as market leader in many countries of the world, is always setting new standards with its innovative developments for weighing and measuring.

The seca 700 is designed for use in hospitals, doctor's surgeries or nursing facilities.

The personal scale has a very sturdy design and will serve you well for a long time.

Safety

Before using the new scale, please take a little time to read the following safety instructions.

- Follow the instructions in the manual.
- Do not drop the scale or subject it to heavy shocks.
- Have repairs carried out by authorised persons only.

Before starting...

Unpacking

- Remove all packaging.

The following components are supplied:

- Weighing platform
- Load leverwork
- Column
- Measuring rod
- 4 fixing screws

Assembly

- Place the column on the weighing platform (A).
- Screw the column to the weighing platform using 4 screws.
- Carefully tilt the scale to one side and hook the load lever onto the tie rod (B).
- Stand the scale back up again (C).
- Insert the small measuring rod into the fixed rod (F).
- Place the scale on a safe, even surface.

Setting the scale's zeropoint

Before the scale is used for the first time and whenever the scale is moved to another place, the scale's zeropoint must be set.

- First slide the two sliding weights to zero.
- With the scale unloaded, turn the adjuster screw **2** until the tip of the measuring indicator **1** is in the middle of the window.

Operation

Correct weighing

- Step on the platform and stand still.

First of all make the **coarse adjustment** in steps of 20 kg or 50 lbs:

- Lift the sliding weight **4** on the lower scale, slide it to the right and let it latch below the roughly estimated weight.

For **fine adjustment** proceed as follows.

- Slide the upper sliding weight **3** to the right until the measuring indicator settles in the middle of the window.
- Add the two values to determine the **total weight**.

Measuring height

- Fold the measuring tongue up, it will latch into the horizontal position.
- Push the measuring tongue up so that the person to be measured can easily stand below it.
- Ask the person to be measured to stand on the platform with his/her back to the measuring rod. The heels should be at the rear end of the platform; back and head must be straight.

- Now slide the measuring slide downwards until the measuring tongue rests on the head.
- For heights below 126.5 cm slide the measuring rod down until you feel resistance.
- Do not bend the measuring tongue.
- Read off the height at marking **5** or at the tip of the measuring tongue

Cleaning

Clean the outside of the scale as required with a household cleaning agent or a commercially available disinfectant. Follow the manufacturer's instructions.

What to do if...

... the beam does not move if a person steps on the scale?

Check that the load lever is hooked in correctly.

... a weight is indicated which is obviously wrong?

Check whether the zeropoint of the scale has been properly set.

... measuring errors or deviations occur when the height is determined?

Please pay attention that the patient does not lean against the measuring rod. Deflection of the measuring rod will cause measuring errors.

Disposal

Disposal of the scale and of the measuring rod

If the scale or the measuring rod can no longer be used, your local refuse disposal service will be glad to provide information about professional disposal in accordance with regulations.

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