

# Tranquility® Product Family

## Incontinence Catalog and Care Guide



  
**TRANQUILITY**®  
INCONTINENCE CARE

**SELECT**®  
Incontinence Products

**ComfortCare**™  
Incontinence Products

**swimmates**®  
Disposable Swimwear

**PBE** Principle  
Business  
Enterprises, Inc.

# Tranquility® Product Family Incontinence Catalog and Care Guide

## About this Guide

The purpose of this guide is to provide help for navigating the best path of care, for individuals or care providers, dealing with incontinence that can challenge normal, productive, fulfilled living and/or pose healthcare risks. Selecting just the right product for each person is vital.

This guide will answer most of your questions and provide you with a peace of mind about incontinence. If you still have questions, please contact our experienced staff at 1-800-467-3224.

## Contents

Company Introduction and Mission .....	2
Product Positioning .....	3
Understanding Absorbency .....	4
The Tranquility Advantage .....	5
Peach Mat Guarantee .....	5
Tranquility® Product Family Listing	
Tranquility Incontinence Products .....	6
Select Incontinence Products .....	16
ComfortCare Incontinence Products .....	21
Swimmates Disposable Swimwear .....	22
Useful Reference Materials	
Moisture Management Standards .....	23
Frequently Asked Questions .....	23
Product Selection Chart .....	24
Sizing Chart .....	25
Quick Order Reference Guide .....	26



## Welcome to PBE

Principle Business Enterprises, (PBE) is a woman-owned, family enterprise celebrating over 55 years of service to the healthcare field and over 30 years as a pioneer in the development of incontinence and moisture management solutions. Founded by Lee and Jim Mitchell in 1961, the company's name and corporate charter clearly speak to its ethical foundation and purpose:

"to operate its business affairs in such a manner as will accord with that fixed principle which will destine it to do nothing but that which is honorable and will promote a high standard of business ethics in its own affairs and may tend to raise and leaven the standard of business ethics of the community."

PBE is the acknowledged innovative leader in high performance niche products especially in the area of incontinence care, wound care and fall prevention. Its products have been designed to address risk factors and care protocols that can materially affect the quality of life for individuals in hospitals, nursing homes, clinics and homecare settings while providing exceptional customer service.

## Our Mission

"To be an example of the right idea of business that operates and prospers from a principled base and tends towards **uplifting, enlightening** and **enriching** the lives of those it serves; that they might have life and that they might have it more abundantly."



# Tranquility® Product Family

## Product Positioning

**TRANQUILITY®**  
INCONTINENCE CARE

**When Performance Matters Most.**

Maximum Protection for greatest needs.

The highest performing incontinence products available anywhere.

**SELECT®**  
Incontinence Products

Heavy Protection, Excellent Value.

High quality products with proven performance.

**ComfortCare™**  
Incontinence Products

Moderate Protection, Quality Design.

"Best in Class" of mass retail products.

**swimmates®**  
Disposable Swimwear

Providing adults with affordable, discreet disposable swimwear designed for bowel containment.





# Understanding Absorbency

Capacity claims can be confusing. Some brands show “drops of fluid” or use terms, such as; “super,” “extra,” or “supreme” to indicate product capacity. Two measurable indicators are C.U.P. and ISO. The Tranquility Product Family uses C.U.P. and displays actual product fluid capacity in standard 8 ounce measuring cups.


**ISO (International Standards Organization)** is a test method that measures capacity that only records the amount of fluid a product can absorb without any pressure applied (free-swell). The product is placed into saline solution for 30 minutes. After removal from the saline, it is allowed to drip for 5 minutes. The level of fluid that is retained is the ISO capacity.

**C.U.P. (Capacity Under Pressure) is the recommended way PBE tests because it gives a more “real world” indication of what the product will hold in actual use.**

PBE tests all absorbent products using the ISO method and also tests using the C.U.P. which takes it a step further. After the product has been soaked in synthetic urine, pressure is applied to simulate the kind of pressure a wearer would exert on the saturated product. The amount of fluid remaining after pressure has been applied is defined as the C.U.P. capacity.

The reality is that wearers of incontinence products are sitting, standing, walking, or lying down – all of which exerts pressure on the product. Experienced caregivers are acutely aware that a product’s **RETENTION** is the key factor in incontinence management.

Easy to understand graphics on packaging help in selecting the right product and capacity to meet the challenge.

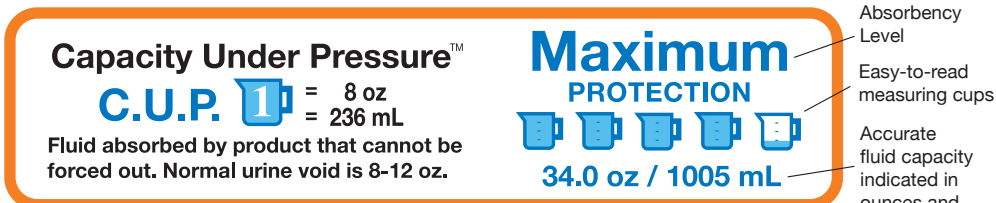

*Absorbent capacity indicator as shown on packaging.*



Experienced caregivers are acutely aware that a product’s **RETENTION** is the key factor in incontinence management.



“The fact that Tranquility actually states the number of ounces it absorbs says it all. Other companies that advertise ‘maximum’ or ‘ultimate’ don’t compare ... I have not seen any other brand diaper that can boast a 34 oz. capacity for their overnight underwear! Dry, plus cost effective – so glad we tried them!”

– D.K., Caregiver, Whippany, NJ



# The Tranquility® Advantage

## Quality of Life

- Confidence of no leakage and no odor ends isolation and restores normal socialization.
- Uninterrupted sleep patterns contribute to healing and well-being.
- Effective core design eliminates the feeling of wetness, coldness and discomfort that prompt nighttime toileting.
- Caregivers experience less fatigue especially with complex cases-dementia, mobility limitation, bowel incontinence and obesity.
- Ending leakage means less laundry of clothing and bedding and less frustration and caregiver burnout.

**TRANQUILITY**  
INCONTINENCE CARE



## Clinical Benefits

- Tranquility technology protects fragile skin from irritation and rashes.
- By inhibiting bacterial growth, urinary tract infections and skin breakdown are reduced in most cases.
- Reduces injury from incontinence related slips and falls.
- By reducing the number of changes needed, the risk of strain/injury to caregivers is reduced.

## Economic Benefits

- Total cost savings: high absorbent capacity means fewer changes, fewer units purchased, fewer units to dispose and less labor required.
- Laundry cost savings: fewer clothing/bedding changes due to leakage, less detergent and water usage, longer linen life and less labor.

## Peach Mat Guarantee®

Exclusive to Tranquility Premium Protection Products

**Tranquility products** are carefully engineered to provide premium protection. The Peach Mat construction uses a proprietary composite structure of tissues, superabsorbent polymers, cellulose and liquid dispersing materials. The peach-colored core assures the product is the Tranquility brand. The clinically proven design guarantees four key factors:

- Unsurpassed capacity: protects against leakage and skin irritation.
- Odor reduction: due to polymer selection and product construction odor is reduced.
- Urine pH neutralization: protects delicate skin from the caustic effect of urine.
- Inhibition of bacterial growth: deprives bacteria of a place or conditions to thrive.

You can rely on Tranquility's promise of unsurpassed overall performance in the areas of...

- skin dryness
- odor reduction
- urine neutralization
- inhibition of bacterial growth.

*In fact, we guarantee it!*

**Peach Mat Guarantee®**

## CONSUMER

Uninterrupted sleep & reduced health risks for wounds and falls



## CAREGIVERS

Improved morale & productivity



## FACILITIES

Reduced product usage, laundry & odors



# WHEN PERFORMANCE MATTERS MOST.

Caregivers have a peace of mind that they are giving their loved ones the best care and a sense of relief from daily household chores. Wearers have confidence to pursue normal daily activities.

Tranquility products guarantee unsurpassed, overall performance in the areas of skin dryness, odor reduction, urine neutralization and inhibition of bacterial growth.

**TRANQUILITY**  
INCONTINENCE CARE

## Look for these high performance features in Tranquility products:

### PMG Peach Mat Guarantee®

The promise of unsurpassed overall performance in the areas of skin dryness, odor reduction, urine pH neutralization and inhibition of bacterial growth.

### BR Breathable

Allows air to circulate around the body, promoting skin integrity.

### KG Kufguards®

Inner leg cuffs channel fluids into the core, helping to contain high volume episodes.

### CL Cloth-Like Backsheet

Product outer layer is soft and rustle free.

### MH Micro-Hook Closure Tabs

Refastenable tabs grip the soft, cloth-like outer layer to securely fasten the brief.

### WI Wetness Indicator

Yellow indicator lines turn to blue when moisture is present eliminating unnecessary changes.

### RT Refastenable Tapes

Advanced tape system allows multiple openings of the briefs.

### AS Adhesive Strip

Secures product in place.

### LF Latex-Free

### Absorbency

Product fluid capacity represented by standard 8-ounce measuring cup.



NEW!

## EliteCare™

Highest capacity brief available featuring the ultra-thin PowerSorb™ Micro-layer. When Performance Matters Most.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	WAIST/HIP	CAPACITY	PACKAGING	HCPC
2412	Medium	32" - 44"	44 oz/1300 mL	4 bags of 12 (48)	T4522
2413	Large	45" - 58"	44 oz/1300 mL	4 bags of 12 (48)	T4523
2414	X-Large	56" - 64"	44 oz/1300 mL	4 bags of 12 (48)	T4524

Absorbency: 44 oz/1300 mL, size Large



Breathable  
Sides

## SmartCore®

Breathable sides help eliminate heat and moisture, providing an ideal micro-climate for skin integrity. Color coded by size.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	WAIST/HIP	CAPACITY	PACKAGING	HCPC
2311	Small (Green)	24" - 32"	22 oz/650 mL	10 bags of 10 (100)	T4521
2312	Medium (White)	32" - 44"	28 oz/828 mL	8 bags of 12 (96)	T4522
2313	Large (Blue)	45" - 58"	34 oz/1005 mL	8 bags of 12 (96)	T4523
2314	X-Large (Beige)	56" - 64"	34 oz/1005 mL	6 bags of 12 (72)	T4524
2315	2XL (White)	60" - 80"	34 oz/1005 mL	4 bags of 8 (32)	T4543

Absorbency: 34 oz/1005 mL, size Large



TOP  
SELLER

## ATN™ (All-Through-the-Night)

A full-fit poly-backed brief with expanded surface area providing longer wear-time by absorbing multiple voids.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	WAIST/HIP	CAPACITY	PACKAGING	HCPC
2183	X-Small*	18" - 26"	18.5 oz/547 mL	10 bags of 10 (100)	T4533
2184	Small	24" - 32"	22.2 oz/657 mL	10 bags of 10 (100)	T4521
2185	Medium	32" - 44"	27.5 oz/812 mL	8 bags of 12 (96)	T4522
2186	Large	45" - 58"	33 oz/975 mL	8 bags of 12 (96)	T4523
2187	X-Large	56" - 64"	34 oz/1005 mL	6 bags of 12 (72)	T4524

Absorbency: 33 oz/975 mL, size Large



\* Cloth-like backsheet, Micro-hook closure tabs



HEAVY

PROTECTION



**Absorbency:** 21.4 oz/632 mL, size Large



## SlimLine® Breathable

Breathable sides help promote skin integrity.  
Slim profile makes it ideal for everyday use.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	WAIST/HIP	CAPACITY	PACKAGING	HCPC
2305	Medium (Peach)	32" - 44"	20.4 oz/602 mL	8 bags of 12 (96)	T4522
2306	Large (Peach)	45" - 58"	21.4 oz/632 mL	8 bags of 12 (96)	T4523
2307	X-Large (Peach)	56" - 64"	23.7 oz/702 mL	6 bags of 12 (72)	T4524

HEAVY

PROTECTION



**Absorbency:** 21.4 oz/632 mL, size Large



## SlimLine® Original

Classic full fit poly-backed brief with a slimmer profile makes it versatile for everyday use.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	WAIST/HIP	CAPACITY	PACKAGING	HCPC
2112	Junior*	28 - 42 lbs	10.2 oz/302 mL	10 bags of 12 (120)	T4530
2166	X-Small*	18" - 26"	10.9 oz/322 mL	10 bags of 10 (100)	T4533
2120	Small*	24" - 32"	12.7 oz/374 mL	10 bags of 10 (100)	T4521
2122	Medium	32" - 44"	20.4 oz/602 mL	8 bags of 12 (96)	T4522
2132	Large	45" - 58"	21.4 oz/632 mL	8 bags of 12 (96)	T4523
2134	X-Large	56" - 64"	23.7 oz/702 mL	6 bags of 12 (72)	T4524

\* Cloth-like backsheet, Micro-hook closure tabs



“

“We recommend Tranquility because it works!  
We tell customers ‘you get what you pay for’ and  
it really applies to incontinence challenges.”

– P.W., Home-Caring Dealer, Roanoke, VA



TOP  
SELLER

## AIR-Plus™

100% breathable allowing for heat and moisture vapor to escape enhancing skin integrity. Fits up to 106" waist.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	WAIST/HIP	CAPACITY	PACKAGING	HCPC
2195	4XL (Peach)	70" - 106"	34 oz/1005 mL	4 bags of 8 (32)	T4543

Absorbency: 34 oz/1005 mL



PROTECTION

MAXIMUM



## HI-Rise™

Full-fit brief with higher coverage in the front and back for a secure fit. Fits up to 96" waist. Breathable, stretchy side panels.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	WAIST/HIP	CAPACITY	PACKAGING	HCPC
2192	3XL	64" - 96"	34 oz/1005 mL	4 bags of 8 (32)	T4543

Absorbency: 34 oz/1005 mL



PROTECTION

MAXIMUM



## Bariatric

Full-fit brief with breathable stretchy side panels comfortably fits up to a 96" waist.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	WAIST/HIP	CAPACITY	PACKAGING	HCPC
2190	3XL	64" - 96"	34 oz/1005 mL	4 bags of 8 (32)	T4543

Absorbency: 34 oz/1005 mL



PROTECTION

MAXIMUM

MAXIMUM

PROTECTION



TOP  
SELLER

Absorbency: 34 oz/1005 mL, size Large



## Premium OverNight™

Maximum absorbency provides extended wear-time for a full night of uninterrupted sleep.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	WAIST/HIP	WEIGHT	CAPACITY	PACKAGING	HCPC
2113	X-Small	17" - 28"	65-85 lbs	20.3 oz/600 mL	4 bags of 22 (88)	T4534
2114	Small	22" - 36"	80-125 lbs	20.3 oz/600 mL	4 bags of 20 (80)	T4525
2115	Medium*	34" - 48"	120-175 lbs	34 oz/1005 mL	4 bags of 18 (72)	T4526
2116	Large*	44" - 54"	170-210 lbs	34 oz/1005 mL	4 bags of 16 (64)	T4527
2117	X-Large*	48" - 66"	210-250 lbs	34 oz/1005 mL	4 bags of 14 (56)	T4528
2118	2XL*	62" - 80"	250+ lbs	34 oz/1005 mL	4 bags of 12 (48)	T4544

\* Blue leg elastics denote Premium OverNight from Premium DayTime.

HEAVY

PROTECTION



Absorbency: 25 oz/740 mL, size Large



## Premium DayTime™

Discreet daytime protection for active individuals with heavier urine voids.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	WAIST/HIP	WEIGHT	CAPACITY	PACKAGING	HCPC
2105	Medium	34" - 48"	120-175 lbs	25 oz/740 mL	4 bags of 18 (72)	T4526
2106	Large	44" - 54"	170-210 lbs	25 oz/740 mL	4 bags of 16 (64)	T4527
2107	X-Large	48" - 66"	210-250 lbs	25 oz/740 mL	4 bags of 14 (56)	T4528
2108	2XL	62" - 80"	250+ lbs	25 oz/740 mL	4 bags of 12 (48)	T4544

For sizes X-Small and Small, try Premium OverNight.



“

As a full time caregiver to my wife, I have painfully witnessed a steady cognitive and physical decline over the past six years. Last fall she started with incontinence. This winter, I finally gave in to not getting up with her to go to the toilet because it was literally wearing me out. I discovered your underwear, which I put on her at bedtime. They work! I am able to get a reasonable night's sleep, have more patience and don't have to change the sheets!"

– M.J., Husband and Caregiver, Belvidere, NJ





## AIR-Plus™

100% breathable backsheet allows for heat and moisture vapor to escape. Ideal for use with low-air-loss bed systems.

Product Features:

PMG

BR

KG

CL

MH

WI


RT

AS

LF

ITEM #	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2709	23"x 36"	29 oz/850 mL	4 bags of 10 (40)	T4541/A4554
2710	30"x 36"	34 oz/1005 mL	4 bags of 10 (40)	T4541/A4554

Absorbency: 34 oz/1005 mL

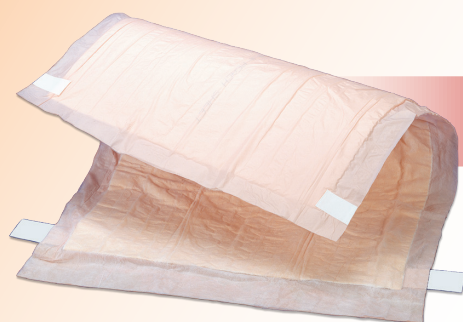

## Peach Sheet

Poly-backed underpad has four adhesive tapes to secure it to wheelchairs, bedding and furniture.

Product Features:

PMG

BR

KG

CL

MH

WI


RT

AS

LF

ITEM #	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2074	21.5"x 32.5"	34 oz/1005 mL	8 bags of 12 (96)	T4541/A4554

Absorbency: 34 oz/1005 mL

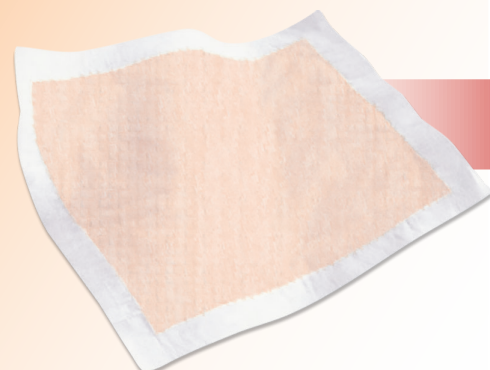

## Heavy-Duty

Strong cloth-like backsheet resists tearing as an individual moves on the underpad.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2088	30"x 36"	34 oz/1005 mL	6 bags of 10 (60)	T4541/A4554

Absorbency: 34 oz/1005 mL



HEAVY

PROTECTION



**Absorbency:** 27.5 oz/813 mL



## Super-Plus Liner

Moisture-proof backing without adhesive is worn in snug fitting underwear. Accommodates plus sizes.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2079	Super-Plus	32"x 14"	27.5 oz/813 mL	8 bags of 12 (96)	T4535

MODERATE

PROTECTION



**Absorbency:** 15.5 oz/458 mL



## Adult Liner

Thin, moisture-proof backed liner is worn in regular underwear for a discreet appearance.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2078	Adult	24"x 9"	15.5 oz/458 mL	4 bags of 30 (120)	T4535

MODERATE

PROTECTION



**Absorbency:** 17.6 oz/519 mL



## Adjustable Belted Undergarment

Worn in place of regular underwear. Self-gripping straps are adjustable. One pair of straps included in each bag.

Product Features:

PMG

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	LENGTH/WIDTH	WAIST/HIP	CAPACITY	PACKAGING	HCPC
2150	28"x 8"	28" - 58"	17.6 oz/519 mL	4 bags of 30 (120)	T4535



## Personal Care Pads

Features two embossed channels to direct fluid into the pad and elastics to form a cup shape. Worn in regular underwear.

Product Features:



ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2380	Super*	10.5" x 5.5"	13.5 oz/400 mL	4 bags of 24 (96)	T4535
2381	Ultimate*	13.5" x 6.5"	15.2 oz/450 mL	4 bags of 24 (96)	T4535
2382	OverNight	16.5" x 7.25"	16.9 oz/500 mL	4 bags of 24 (96)	T4535

\* Individually packaged for hygienic protection

Absorbency: 15.2 oz/450 mL, size Ultimate



## TrimShield® Pads

Unique pleated core construction keeps product thin and very absorbent. Moisture barrier. Worn in regular underwear.

Product Features:



ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2080	Mini	10.5" x 3"	7 oz/199 mL	8 bags of 25 (200)	T4535
2083	Super-Plus	11.75" x 4"	8 oz/236 mL	5 bags of 25 (125)	T4535

Absorbency: 8 oz/236 mL, size Super-Plus



"After looking in drug stores and medical equipment supply stores for several years, I have settled on these two products to meet my various needs. The Overnight Pad works well when I play tennis and the Super-Plus Liner is used when I know I will be standing for prolonged periods. Thanks for the variety you offer."


– J.K., Wearer, Houston, TX



## What is a booster pad?

Booster pads have a flow-through design that allows the pad to fill to capacity first and then pass additional fluid to the host (primary) garment. Booster pads are placed inside any disposable undergarment with a moisture barrier, boosting absorbency to extend wear-time, reduce changes and help contain bowel incontinence. Strategically placed over the hip helps prevent leaking for side sleepers.

For additional placement ideas, see page 20.

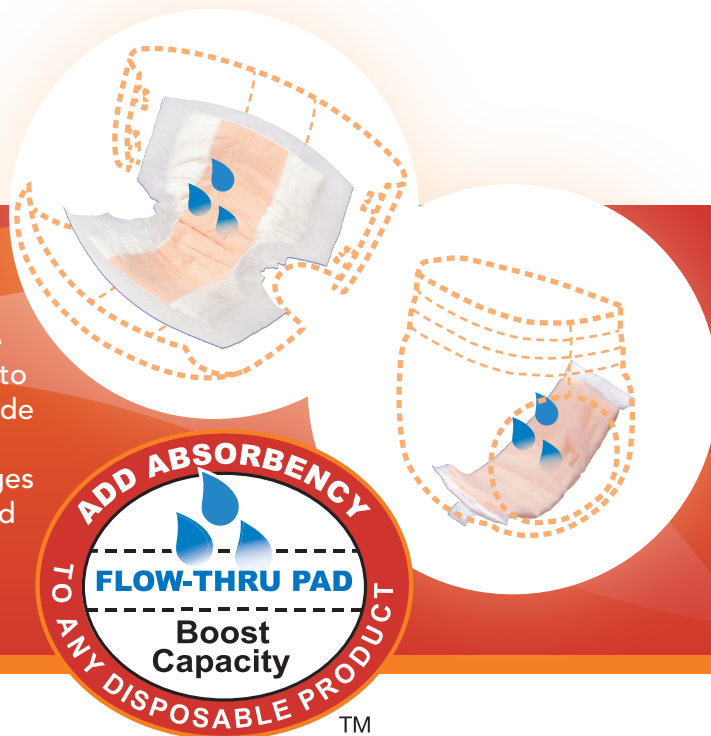

ADDED

PROTECTION



TOP  
SELLER

Absorbency: 14.2 oz/420 mL, size Super



## TopLiner® Booster Pads

Thin design allows for placement in target areas where additional absorption is needed.

Product Features:



ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2072	Mini	10.5" x 2.75"	5.7 oz/169 mL	8 bags of 25 (200)	T4535
2070	Regular	14" x 4"	10.9 oz/322 mL	8 bags of 25 (200)	T4535
2060	Super	15" x 4.25"	14.2 oz/420 mL	8 bags of 25 (200)	T4535

ADDED

PROTECTION



TOP  
SELLER

Absorbency: 13.6 oz/403 mL, size Contour



## TopLiner® Booster Contours

Wide hourglass shape helps contain bowel. Soft texture and no adhesive strip makes cleanup easier.

Product Features:



ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
3096	Contour	21.5" x 13.5"	13.6 oz/403 mL	10 bags of 12 (120)	T4535
3097	Super-Plus*	32" x 14"	27.5 oz/813 mL	8 bags of 12 (96)	T4535

\* Ideal for use with sizes XL and greater.

## ThinLiner Moisture Management® Sheets

100% breathable, soft cloth-like sheets reduce skin-on-skin friction. Collects, absorbs and retains moisture.

Product Features:



ITEM #	LENGTH/WIDTH	CAPACITY*	PACKAGING	HCPC
3090	6" x 10"	5 oz/150 mL*	8 bags of 25 (200)	T4535
3091	6" x 14"	7.7 oz/228 mL*	4 bags of 25 (100)	T4535
3092	20" x 22"	15 oz/450 mL*	4 bags of 25 (100)	T4535

\* Moisture capacity (not C.U.P.)

## Cleansing Wipes

Paraben and alcohol free, hypoallergenic with a mild scent. Generous 9" x 13" size allows for quicker cleanup.

Product Features:



ITEM #	SIZE	LENGTH/WIDTH	PACKAGING	HCPC
3101	Full Case	9" x 13"	12 packs of 56 (672)	A4335
3121	Mini Case	9" x 13"	4 packs of 56 (224)	A4335



"Haven't had any leaks since using these products. Had to get up 2-3 times a night with my elderly mother in the past. Now we can both get an uninterrupted night's sleep and not have to change bedding, etc. Thank you. TopLiners are inexpensive and save having to change disposable underwear every time. Thanks for giving us back our rest."

– B.T., Daughter and Caregiver, Rowe, NM





# EXCELLENT CAPACITY, OUTSTANDING VALUE.



Heavy Protection, Excellent Value.  
High quality products with proven performance.

**SELECT**  
Incontinence Products

## Look for these high performance features in Select products:

- BR Breathable**  
Allows air to circulate around the body, promoting skin integrity.
- KG Kufguards®**  
Inner leg cuffs channel fluids into the core, helping to contain high volume episodes.
- CL Cloth-Like Backsheet**  
Product outer layer is soft and rustle free.
- MH Micro-Hook Closure Tabs**  
Refastenable tabs grip the soft, cloth-like outer layer to securely fasten the brief.
- WI Wetness Indicator**  
Yellow indicator lines turn to blue when moisture is present eliminating unnecessary changes.
- RT Refastenable Tapes**  
Advanced tape system allows multiple openings of the briefs.
- AS Adhesive Strip**  
Secures product in place.
- LF Latex-Free**
- Absorbency**  
Product fluid capacity represented by standard 8-ounce measuring cup.





## Briefs

A full fit, poly brief with color coding for easy size identification.

Product Features:



ITEM #	SIZE	WAIST/HIP	CAPACITY	PACKAGING	HCPC
3665	Youth* (White)	28 - 42 lbs	9.8 oz/289 mL	10 bags of 10 (100)	T4530
3666	X-Small* (White)	18" - 26"	10.4 oz/308 mL	10 bags of 10 (100)	T4533
2620	Small (Green)	24" - 32"	11.8 oz/348 mL	10 bags of 10 (100)	T4521
2624	Medium (White)	32" - 44"	15.8 oz/468 mL	8 bags of 12 (96)	T4522
2634	Large (Blue)	45" - 58"	19.3 oz/571 mL	6 bags of 12 (72)	T4523
2635	X-Large (Beige)	56" - 64"	23.0 oz/681 mL	8 bags of 8 (64)	T4524

\* Cloth-like backsheets, Micro-hook closure tabs

Absorbency: 19.3 oz/571 mL, size Large



## Soft n' Breathable

Contoured core with breathable sides allow for air circulation. Color coded for easy size identification.

Product Features:



ITEM #	SIZE	WAIST/HIP	CAPACITY	PACKAGING	HCPC
2627	Medium (White)	32" - 44"	15.8 oz/468 mL	8 bags of 12 (96)	T4522
2628	Large (Blue)	45" - 58"	19.3 oz/571 mL	6 bags of 12 (72)	T4523
2629	X-Large (Beige)	56" - 64"	23.0 oz/681 mL	8 bags of 8 (64)	T4524

Absorbency: 19.3 oz/571 mL, size Large



## Disposable Absorbent Underwear

Highly absorbent underwear with breathable sides and full-rise waist panel for a comfortable, secure fit.

Product Features:



ITEM #	SIZE	WAIST/HIP	WEIGHT	CAPACITY	PACKAGING	HCPC
2602	Youth	15" - 25"	38-65 lbs	14.5 oz/429 mL	8 bags of 12 (96)	T4534
2603	X-Small	17" - 28"	65-85 lbs	14.5 oz/429 mL	4 bags of 24 (96)	T4534
2604	Small	22" - 36"	80-125 lbs	14.5 oz/429 mL	4 bags of 22 (88)	T4525
2605	Medium	34" - 48"	120-175 lbs	19.0 oz/562 mL	4 bags of 20 (80)	T4526
2606	Large	44" - 54"	170-210 lbs	19.0 oz/562 mL	4 bags of 18 (72)	T4527
2607	X-Large	48" - 66"	210-250 lbs	19.0 oz/562 mL	4 bags of 14 (56)	T4528
2608	2XL	62" - 80"	250+ lbs	19.0 oz/562 mL	4 bags of 12 (48)	T4544

Absorbency: 19 oz/562 mL, size Large



ADDED

PROTECTION



Absorbency: 8.4 oz/248 mL



## Ultra-thin

Ultra-thin, composite mat is fluff-free (no shake-out). Consistently smooth, uniform construction resists bunching during use.

Product Features:



ITEM #	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2717	22.5" x 30"	8.4 oz/248 mL	5 bags of 30 (150)	T4541/A4554

ADDED

PROTECTION



Absorbency: 18.5 oz/550 mL, size Ultra Large



## Standard

Perfect for protecting bedding, furniture and wheelchairs from leakage.


Product Features:



ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2675	Large (Blue)	23" x 36"	8.1 oz/239 mL	6 bags of 25 (150)	T4541/A4554
2677	Extra Large (Green)	28" x 30"	10.6 oz/314 mL	10 bags of 10 (100)	T4541/A4554
2679	Ultra Large (Beige)	36" x 36"	18.5 oz/550 mL	5 bags of 10 (50)	T4541/A4554

ADDED

PROTECTION

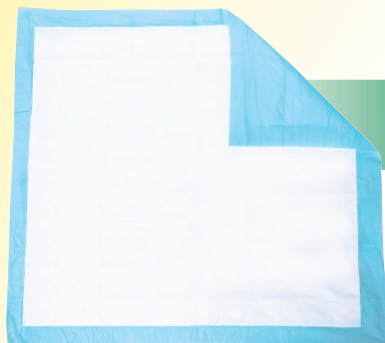

Absorbency: 1.7 oz/50 mL



## Procedure Pad

Compact and smaller design for hygienic protection of surfaces. No superabsorbent. No fluff.

Product Features:



ITEM #	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2707	22.5" x 22.5"	1.7 oz/50 mL	10 bags of 30 (300)	T4541/A4554



Absorbency: 12.8 oz/378 mL



## Belted Undergarment

A discreet product featuring a pair of wide, two button-elastic straps. Worn in place of regular underwear.

Product Features:

BR

KG

CL

MH

WI

RT


AS

LF

ITEM #	LENGTH/WIDTH	WAIST/HIP	CAPACITY	PACKAGING	HCPC
2650	28" x 8"	28" - 44"	12.8 oz/ 378 mL	4 bags of 30 (120)	T4535

PROTECTION

MODERATE

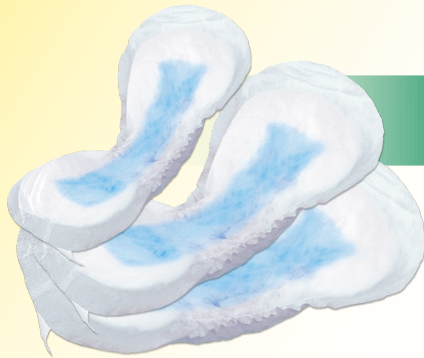

Absorbency: 11.8 oz/349 mL, size Regular



## Personal Care Pads

Contoured pads with moisture barrier fit securely into regular underwear for easy changes.

Product Features:

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2880	Light*	10.5" x 5.5"	10.1 oz/299 mL	4 bags of 24 (96)	T4535
2881	Regular*	13.5" x 6.5"	11.8 oz/349 mL	4 bags of 24 (96)	T4535
2882	Extra	16.5" x 7.25"	12.6 oz/375 mL	4 bags of 24 (96)	T4535

\* Individually packaged for hygienic protection

PROTECTION

MODERATE



Absorbency: 8 oz/236 mL



## Shield/Pad

Ideal for providing skin dryness and odor reduction. Moisture barrier. Worn in regular underwear.

Product Features:

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2648	Shield/Pad	11.75" x 3.75"	8 oz/236 mL	5 bags of 20 (100)	T4535

PROTECTION

LIGHT





Absorbency: 6.7 oz/198 mL, size Medium



## Booster Pads

A soft, flow-through pad designed to fill to capacity and then pass additional fluid into the primary garment.

Product Features:

BR

KG

CL

MH

WI

RT


AS

LF

ITEM #	SIZE	LENGTH/WIDTH	CAPACITY	PACKAGING	HCPC
2770	Small	12" x 3.25"	5.5 oz/163 mL	6 bags of 30 (180)	T4535
2760	Medium	12" x 4.25"	6.7 oz/198 mL	8 bags of 25 (200)	T4535
2762	Extended	15" x 4.25"	8.0 oz/236 mL	8 bags of 25 (200)	T4535


## What is a booster pad?

Booster pads have a flow-through design that allows the pad to fill to capacity first and then pass additional fluid to the host (primary) garment. Booster pads are placed inside any disposable undergarment with a moisture barrier, boosting absorbency to extend wear-time. They are also easy to change and dispose.

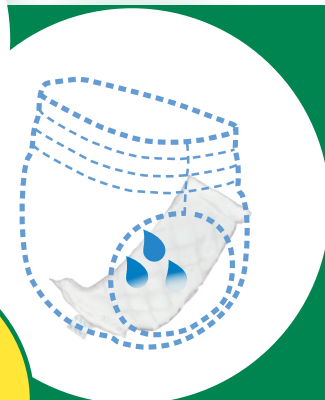


## Alternative Placements for Booster Pads

- Position booster pad horizontally across the stomach, adhesive side up. Apply absorbent undergarment as usual and adhere pad.
- Strategically place the booster pad over the hip to help prevent leaking for side sleepers.
- For women, position the pad toward the back. For men, position the pad toward the front.
- For men, DO NOT remove adhesive backing. Fold booster pad over top of penis with release paper to the outside. Apply absorbent undergarment as usual.

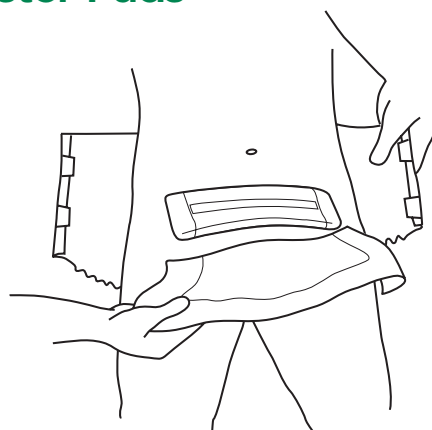

Tummy pad placement

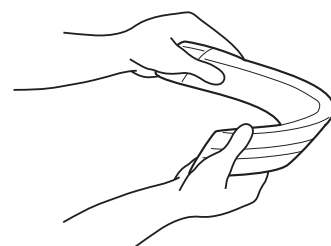

Fold pad in half



Moderate Protection,  
Quality Design.  
"Best in Class" of  
mass retail products.



Absorbency: 13.5 oz/400 mL, size Large



## Briefs

Provides breathable comfort in a value brief where cost containment is the key focus.

Product Features:



ITEM #	SIZE	WAIST/HIP	CAPACITY	PACKAGING	HCPC
#2965-100	Medium (White)	32" - 44"	13.5 oz/400 mL	4 bags of 25 (100)	T4522
#2966-100	Large (Blue)	45" - 58"	13.5 oz/400 mL	4 bags of 25 (100)	T4523
#2967-100	X-Large (Beige)	56" - 64"	13.5 oz/400 mL	4 bags of 25 (100)	T4524

PROTECTION

MODERATE



Absorbency: 13.5 oz/400 mL, size Large



## Disposable Absorbent Underwear

Ideal for active individuals offering the best absorption in the economy class.

Product Features:




ITEM #	SIZE	WAIST/HIP	WEIGHT	CAPACITY	PACKAGING	HCPC
#2974-100	Small	22" - 36"	80-125 lbs	10.1 oz/299 mL	4 bags of 25 (100)	T4525
#2975-100	Medium	34" - 48"	120-175 lbs	13.5 oz/400 mL	4 bags of 25 (100)	T4526
#2976-100	Large	44" - 54"	170-210 lbs	13.5 oz/400 mL	4 bags of 25 (100)	T4527
#2977-100	X-Large	48" - 66"	210-250 lbs	13.5 oz/400 mL	4 bags of 25 (100)	T4528

PROTECTION

MODERATE

For Aqua Therapy or  
Any Day in the Pool

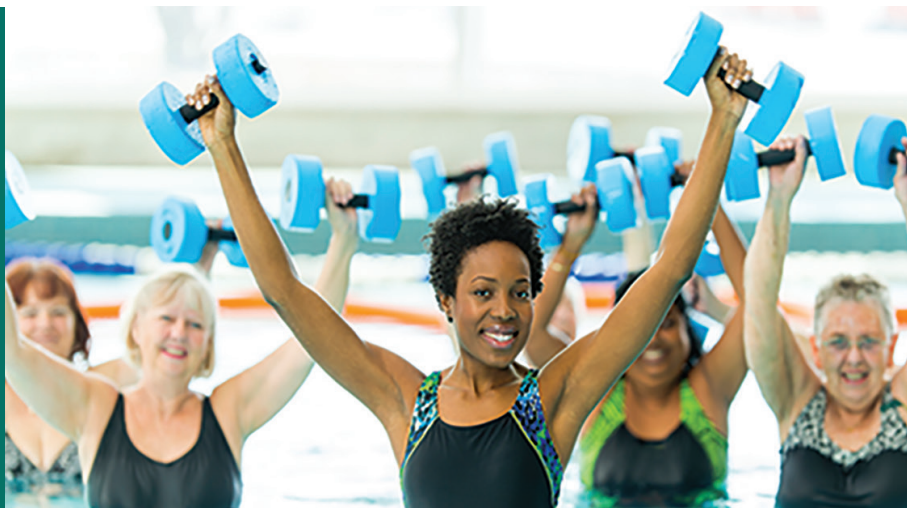

## Disposable Swimwear

This discreet unisex disposable underwear is worn under a bathing suit. Features Kufguards (inner leg cuffs) and moisture barrier to help contain bowel incontinence. Designed for minimal swelling and does not break apart when wet.

Product Features:

BR

KG

CL

MH

WI

RT

AS

LF

ITEM #	SIZE	WAIST/HIP	WEIGHT	PACKAGING	HCPC
2844	Small	22" - 36"	80-125 lbs	4 bags of 22 (88)	A4335
2845	Medium	34" - 48"	120-175 lbs	4 bags of 20 (80)	A4335
2846	Large	44" - 54"	170-210 lbs	4 bags of 18 (72)	A4335
2847	X-Large	48" - 66"	210-250 lbs	4 bags of 14 (56)	A4335
2848	XXL-Plus	62" - 80"	250+ lbs	4 bags of 12 (48)	A4335



BR Breathable

KG Kufguards

CL Cloth-Like

MH Micro-Hook

WI Wetness Indicator

RT Refastenable Tapes

AS Adhesive Strip

LF Latex-Free

= 8 oz.



# Moisture Management Standards



Effective products dealing with moisture need to meet exacting performance criteria. The objectives are always to maintain or restore the ideal “micro-climate” for skin, the safety and quality of life for the individuals and caregivers involved. **The following are fundamental principles and criteria that need to be considered:**

1. Sufficient absorbent capacity to provide 6-8 hours of protection to allow uninterrupted activity (such as sleep, working, shopping, etc.).
2. Protection from risks to the skin caused by sheer, friction, heat, wetness and unbalanced pH.
3. Protection from the risk of falls associated with incontinence and/or wound exudate that leaves the feeling of wetness/cold from the absorbent products.
4. Design and absorbent capacity that eliminates the waste and costs incurred arising from product leakage and the resulting excessive cost and labor demands resulting from wet bedding and/or wet clothing.
5. Ease of use and effectiveness that minimizes the burden and frustration for caregivers or professional staff that leads to “burnout.”
6. The high performance that allows care-providers to focus on individual-centered therapies and life-enhancement strategies for care recipients.

## Frequently Asked Questions

### 1. Will drinking less fluid decrease my challenges with incontinence?

No. Cutting back on fluid intake can lead to dehydration, constipation and irritation of the bladder. It is true that certain types of fluids should be avoided such as alcohol and caffeine that irritate the bladder.

### 2. What can I do to stop leaking with my current incontinence products?

The main factors to take into consideration when choosing the correct incontinence product are:

- Sizing and Fit: Make sure you have the right size product. Leg openings should be snug fitting (no gaps) and should not create redness around the leg/groin area.
- Style and absorption capacity required: Make sure that you are wearing the correct product style and absorbency level to meet your needs. Refer to the Product Selection Guide (page 24) to assist in selection of the right style of product to meet ones mobility and incontinence level.

### 3. Can two briefs/pads be used at the same time (also called “diaper doubling”)?

No, doubling up on products that contain a moisture barrier will not increase absorption. Briefs, disposable underwear and pads/liners

are typically designed with a moisture barrier that stops the urine from passing through. This approach to adding more capacity or addressing leakage concerns works only if the second product is a “flow-through” design with no moisture barrier (Refer to booster pads on pages 14 & 20).

### 4. Why are some incontinence products so much more expensive?

Higher performing incontinence products (that may cost more) typically contain more super absorbent capacity and are made with superior quality materials. The Tranquility Product Family keeps skin dryer, neutralizes urine pH, and reduces odors. It is often possible to save money by using the more expensive products because they require fewer changes, result in fewer clothing/ bedding changes, less waste, less laundry due to leakage, and better quality of life for both the care provider and the care recipient.

### 5. How is bowel incontinence managed?

Booster pads are designed to contain the bowel, assist in clean-up and can extend the wear time of the “host garment.” The focus for bowel incontinence is on containment not absorption. (Refer to booster pads on page 14).

# Product Selection Chart

Refer to the chart below for product selections/recommendations based on type of incontinence, frequency and absorbency level needed. Incontinence types are defined below for reference.

## Incontinence Level:

	1	2	3	4	5
Incontinence Type (see below for descriptions)	Stress	Stress Urge Mixed	Overflow Reflex Functional	Overflow Reflex Functional	Overflow Reflex Functional
Incontinence (Urine/Bowel)	Occasionally Incontinent Partial voids No Fecal	Occasionally Incontinent Partial Voids Occasional Fecal Smears	Frequently Incontinent Full Void Fecal Incontinent	Frequently Incontinent Multiple Voids Fecal Incontinent	Frequently Incontinent Multiple Voids Fecal Incontinent
Mental Orientation/ Mobility Level	Oriented/Clear Active No Assistance	Oriented/Clear Active Minimal Assistance	Oriented/Clear to Slightly Confused Active; Semi-Active Minimal to Moderate Assistance	Oriented/Clear to Confused Semi-Active; Inactive Moderate Assistance to Immobile	Oriented/Clear to Confused Inactive Moderate Assistance to Immobile

## Absorbency Level:

Light






Moderate

Heavy

Maximum

Maximum-Plus

## Products

<div>Pads/Liners</div> <div></div>	<div>Tranquility® TrimShield®</div> <div>Select® Shield/Pad</div>	<div>Tranquility® Personal Care Pads</div> <div>Select® Personal Care Pads</div> <div>Tranquility® Adult Liner</div> <div>Tranquility® Belted Undergarment</div> <div>Select® Belted Undergarment</div>	<div>Tranquility® Super-Plus Liner</div>		
<div>Briefs</div> <div></div>		<div>ComfortCare™</div>	<div>Tranquility® SlimLine® Original</div> <div>Tranquility® SlimLine® Breathable</div> <div>Select® Soft n' Breathable</div> <div>Select® Briefs</div>	<div>Tranquility® SmartCore®</div> <div>Tranquility® ATN™</div> <div>Tranquility® Bariatric</div>	<div>All Maximum Options Shown at Left, PLUS</div> <div>Tranquility® EliteCare™</div>
<div>Underwear</div> <div></div>		<div>ComfortCare™</div>	<div>Tranquility® DayTime™</div> <div>Select® Underwear</div>	<div>Tranquility® OverNight™</div>	
<div>Booster Pads</div> <div></div>	<div>Booster pads can be added to any pad, liner, brief or underwear to assist with bowel incontinence or to increase absorption.</div>				
<div>Underpads</div> <div></div>	<div>Select® Procedure Pad</div>	<div>Select® Large</div> <div>Select® XL</div> <div>Select® Ultra-Thin</div>	<div>Select® Ultra Large</div>	<div>Tranquility® AirPlus™</div> <div>Tranquility® Peach Sheet</div> <div>Tranquility® Heavy Duty</div>	

## Definitions: Incontinence types

### Stress:

Leakage of small amounts of urine due to sudden pressure on the bladder, such as coughing, sneezing, laughing or exercising.

### Urge:

A sudden urge to urinate that is so strong it cannot be held long enough to reach the bathroom.

### Mixed:

A combination of the symptoms of stress and urge incontinence.

### Overflow:

Involuntary urination. Bladder muscles are unable to contain urine under pressure. Feeling of inability to empty bladder completely.

### Reflex:

Involuntary urination without sensation of a full bladder. Unaware of the need to urinate.

### Functional:


Inability to get to the bathroom fast enough due to mobility or cognitive impairments.

# Sizing Chart

Suggested sizes are a guideline. When in doubt, use a tape measure and obtain an accurate waist/hip measurement. Use the larger of the two measurements to find the correct size. **For sizes greater than an XL, obtain an accurate hip measurement for best fit.**



## Disposable Briefs

Size Description	Waist/Hips (inches)	Weight (pounds)	Women's Pant Size	Men's Pant Size
Youth	N/A	28-42 lbs	N/A	N/A
X-Small	18"-26"	42-90 lbs	0-3	18-26
Small	24"-32"	85-100 lbs	4-6	24-32
Medium	32"-44"	100-150 lbs	8-12	32-44
Large	45"-58"	150-200 lbs	14-18	44-58
X-Large	56"-64"	200-250 lbs	18-24	56-64
2XL	60"-80"	250+ lbs	26-28	60+
3XL	64"-96"	250+ lbs	28+	64+
4XL	70"-106"	250+ lbs	28+	70+

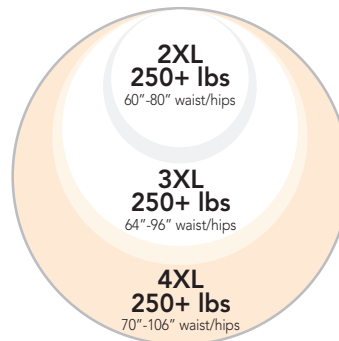

## Disposable Absorbent Underwear

Size Description	Waist/Hips (inches)	Weight (pounds)	Women's Pant Size	Men's Pant Size
Youth	15"-25"	38-65 lbs	N/A	N/A
X-Small	17"-28"	65-85 lbs	0-3	17-28
Small	22"-36"	80-125 lbs	4-6	22-36
Medium	34"-48"	120-175 lbs	6-12	34-48
Large	44"-54"	170-210 lbs	12-18	44-54
X-Large	48"-66"	210-250 lbs	18-24	48-66
2XL	62"-80"	250+ lbs	24+	62+

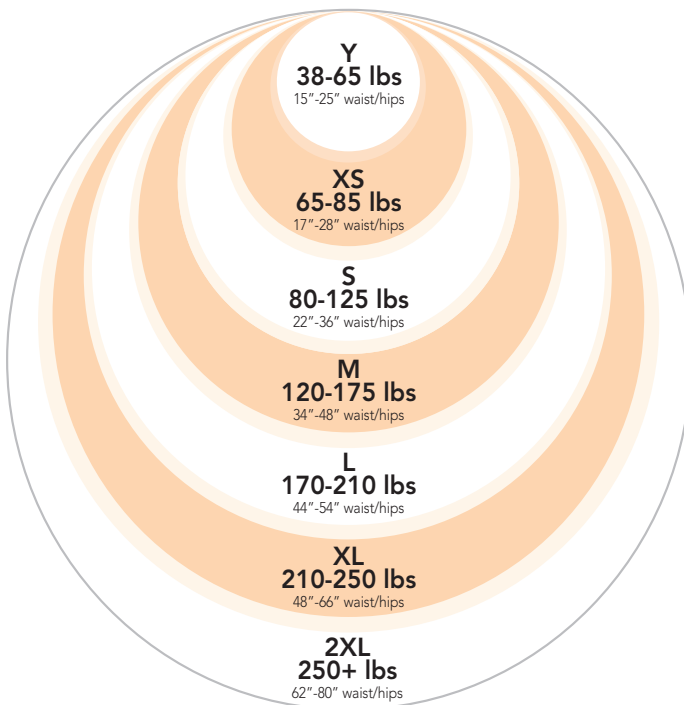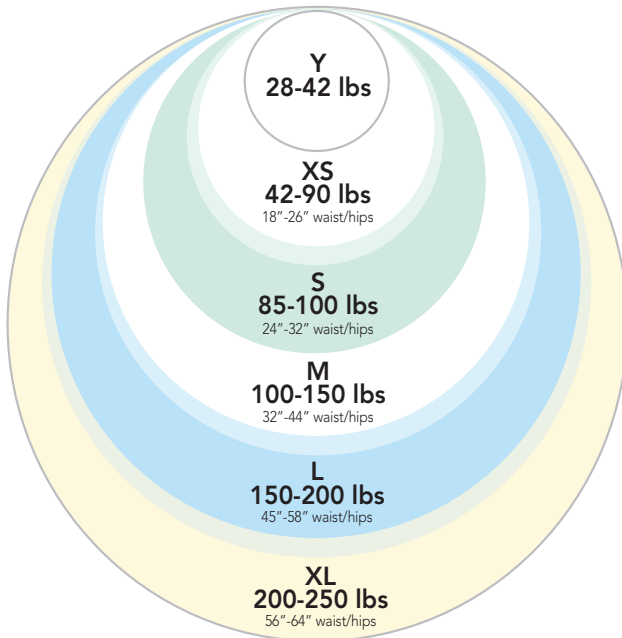



# Quick Order Reference Guide

Item #	Product Description	Unit/ Case	Pkg/ Case	HCPC	Printed Pkg	Page #
2060	Tranquility® TopLiner® Booster Pad- Super	200	8/25	T4535	Yes	14
2070	Tranquility® TopLiner® Booster Pad- Regular	200	8/25	T4535	Yes	14
2072	Tranquility® TopLiner® Booster Pad- Mini	200	8/25	T4535	Yes	14
2074	Tranquility® Peach Sheet Underpad	96	8/12	T4541/A4554	Yes	11
2078	Tranquility® Adult Liner	120	4/30	T4535	Yes	12
2079	Tranquility® Super-Plus Liner	96	8/12	T4535	No	12
2080	Tranquility® TrimShield® Pad- Mini	200	8/25	T4535	No	13
2083	Tranquility® TrimShield® Pad- Super-Plus	125	5/25	T4535	Yes	13
2088	Tranquility® Heavy-Duty Underpad	60	6/10	T4541/A4554	No	11
2105	Tranquility® Premium DayTime™ Underwear- Medium	72	4/18	T4526	Yes	10
2106	Tranquility® Premium DayTime™ Underwear- Large	64	4/16	T4527	Yes	10
2107	Tranquility® Premium DayTime™ Underwear- X-Large	56	4/14	T4528	Yes	10
2108	Tranquility® Premium DayTime™ Underwear- 2XL	48	4/12	T4544	Yes	10
2112	Tranquility® SlimLine® Original Brief- Junior	120	10/12	T4530	Yes	8
2113	Tranquility® Premium OverNight™ Underwear- X-Small	88	4/22	T4534	Yes	10
2114	Tranquility® Premium OverNight™ Underwear- Small	80	4/20	T4525	Yes	10
2115	Tranquility® Premium OverNight™ Underwear- Medium	72	4/18	T4526	Yes	10
2116	Tranquility® Premium OverNight™ Underwear- Large	64	4/16	T4527	Yes	10
2117	Tranquility® Premium OverNight™ Underwear- X-Large	56	4/14	T4528	Yes	10
2118	Tranquility® Premium OverNight™ Underwear- 2XL	48	4/12	T4544	Yes	10
2120	Tranquility® SlimLine® Original Brief- Small	100	10/10	T4521	Yes	8
2122	Tranquility® SlimLine® Original Brief- Medium	96	8/12	T4522	Yes	8
2132	Tranquility® SlimLine® Original Brief- Large	96	8/12	T4523	Yes	8
2134	Tranquility® SlimLine® Original Brief- X-Large	72	6/12	T4524	Yes	8
2150	Tranquility® Belted Undergarment	120	4/30	T4535	Yes	13
2166	Tranquility® SlimLine® Original Brief- X-Small	100	10/10	T4533	Yes	8
2183	Tranquility® ATN™ (All-Through-the-Night) Brief- X-Small	100	10/10	T4533	Yes	7
2184	Tranquility® ATN™ (All-Through-the-Night) Brief- Small	100	10/10	T4521	Yes	7
2185	Tranquility® ATN™ (All-Through-the-Night) Brief- Medium	96	8/12	T4522	Yes	7
2186	Tranquility® ATN™ (All-Through-the-Night) Brief- Large	96	8/12	T4523	Yes	7
2187	Tranquility® ATN™ (All-Through-the-Night) Brief- X-Large	72	6/12	T4524	Yes	7
2190	Tranquility® Bariatric Brief- 3XL	32	4/8	T4543	Yes	9
2192	Tranquility® HI-Rise™ Bariatric Brief- 3XL	32	4/8	T4543	Yes	9
2195	Tranquility® AIR-Plus™ Bariatric Brief- 4XL	32	4/8	T4543	Yes	9
2305	Tranquility® SlimLine® Breathable Brief- Medium	96	8/12	T4522	Yes	8
2306	Tranquility® SlimLine® Breathable Brief- Large	96	8/12	T4523	Yes	8
2307	Tranquility® SlimLine® Breathable Brief- X-Large	72	6/12	T4524	Yes	8
2311	Tranquility® SmartCore® Brief- Small	100	10/10	T4521	Yes	7
2312	Tranquility® SmartCore® Brief- Medium	96	8/12	T4522	Yes	7
2313	Tranquility® SmartCore® Brief- Large	96	8/12	T4523	Yes	7
2314	Tranquility® SmartCore® Brief- X-Large	72	6/12	T4524	Yes	7
2315	Tranquility® SmartCore® Brief- 2XL	32	4/8	T4543	Yes	7
2380	Tranquility® Personal Care Pad- Super	96	4/24	T4535	Yes	13
2381	Tranquility® Personal Care Pad- Ultimate	96	4/24	T4535	Yes	13
2382	Tranquility® Personal Care Pad- OverNight	96	4/24	T4535	Yes	13
2412	Tranquility® EliteCare™ Disposable Brief- Medium	48	4/12	T4522	Yes	7
2413	Tranquility® EliteCare™ Disposable Brief- Large	48	4/12	T4523	Yes	7
2414	Tranquility® EliteCare™ Disposable Brief- X-Large	48	4/12	T4524	Yes	7

Item #	Product Description	Unit/ Case	Pkg/ Case	HCPC	Printed Pkg	Page #
2602	Select® Absorbent Underwear - Youth	96	8/12	T4534	Yes	17
2603	Select® Absorbent Underwear - X-Small	96	4/24	T4534	Yes	17
2604	Select® Absorbent Underwear - Small	88	4/22	T4525	Yes	17
2605	Select® Absorbent Underwear - Medium	80	4/20	T4526	Yes	17
2606	Select® Absorbent Underwear - Large	72	4/18	T4527	Yes	17
2607	Select® Absorbent Underwear - X-Large	56	4/14	T4528	Yes	17
2608	Select® Absorbent Underwear - XXL-Plus	48	4/12	T4544	Yes	17
2620	Select® Disposable Brief - Small	100	10/10	T4521	Yes	17
2624	Select® Disposable Brief - Medium	96	8/12	T4522	Yes	17
2627	Select® Soft n' Breathable Brief - Medium	96	8/12	T4522	Yes	17
2628	Select® Soft n' Breathable Brief - Large	72	6/12	T4523	Yes	17
2629	Select® Soft n' Breathable Brief - X-Large	64	8/8	T4524	Yes	17
2634	Select® Disposable Brief - Large	72	6/12	T4523	Yes	17
2635	Select® Disposable Brief - X-Large	64	8/8	T4524	Yes	17
2648	Select® Shield/Pad	100	5/20	T4535	No	19
2650	Select® Belted Undergarment	120	4/30	T4535	Yes	19
2675	Select® Underpads - Large	150	6/25	T4541/A4554	Yes	18
2677	Select® Underpads - Extra Large	100	10/10	T4541/A4554	No	18
2679	Select® Underpads - Ultra Large	50	5/10	T4541/A4554	No	18
2707	Select® Procedure Pad	300	10/30	T4541/A4554	No	18
2709	Tranquility® AIR-Plus™ Underpad 23 x 36	40	4/10	T4541/A4554	Yes	11
2710	Tranquility® AIR-Plus™ Underpad 30 x 36	40	4/10	T4541/A4554	Yes	11
2717	Select® Ultra-Thin Underpad	150	5/30	T4541/A4554	No	20
2760	Select® Booster Pad - Medium	200	8/25	T4535	Yes	20
2762	Select® Booster Pad - Extended	200	8/25	T4535	No	20
2770	Select® Booster Pad - Small	180	6/30	T4535	Yes	20
2844	Swimmates® - Small	88	4/22	A9999	Yes	22
2845	Swimmates® - Medium	80	4/20	A9999	Yes	22
2846	Swimmates® - Large	72	4/18	A9999	Yes	22
2847	Swimmates® - X-Large	72	4/14	A9999	Yes	22
2848	Swimmates® - XXL-Plus	48	4/12	A9999	Yes	22
2880	Select® Personal Care Pads - Light	96	4/24	T4535	No	19
2881	Select® Personal Care Pads - Regular	96	4/24	T4535	No	19
2882	Select® Personal Care Pads - Extra	96	4/24	T4535	No	19
3090	Tranquility® ThinLiner Moisture Management® Sheets 6 x 10	200	8/25	A9999	Yes	15
3091	Tranquility® ThinLiner Moisture Management® Sheets 6 x 14	100	4/25	A9999	Yes	15
3092	Tranquility® ThinLiner Moisture Management® Sheets 20 x 22	100	4/25	A9999	Yes	15
3096	Tranquility® TopLiner® Booster Contour Pad	120	10/12	T4535	Yes	14
3097	Tranquility® TopLiner® Booster Contour Pad- Super-Plus	96	8/12	T4535	Yes	14
3101	Tranquility® Cleansing Wipes 9 x 13	672	12/56	A4335	Yes	15
3121	Tranquility® Cleansing Wipes 9 x 13	224	4/56	A4335	Yes	15
3665	Select® Disposable Brief- Youth	100	10/10	T4533	Yes	17
3666	Select® Disposable Brief- X-Small	100	10/10	T4533	Yes	17
2965-100	ComfortCare™ Briefs - Medium	100	4/25	T4522	Yes	21
2966-100	ComfortCare™ Briefs - Large	100	4/25	T4523	Yes	21
2967-100	ComfortCare™ Briefs - X-Large	100	4/25	T4524	Yes	21
2974-100	ComfortCare™ Disposable Underwear - Small	100	4/25	T4525	Yes	21
2975-100	ComfortCare™ Disposable Underwear - Medium	100	4/25	T4526	Yes	21
2976-100	ComfortCare™ Disposable Underwear - Large	100	4/25	T4527	Yes	21
2977-100	ComfortCare™ Disposable Underwear - X-Large	100	4/25	T4528	Yes	21



# Tranquility® Product Family

WHEN PERFORMANCE MATTERS MOST.

## Try it!

### Product Sampling/Promotional Programs

PBE offers a generous sample program, allowing the customer to experience the high quality performance of these unique incontinence products.



### Dealers/Distributors

Dealers and distributors can contact PBE at 1-800-467-3224 for information about Tranquility products' wide range of marketing services, consumer sampling programs, and support materials.

### Care Center

The Tranquility Care Center, 1-800-467-3224 ext. 7, provides FREE product samples, private consultation, and product information to consumers.

### Website

The Tranquility website, [www.TranquilityProducts.com](http://www.TranquilityProducts.com), features technical articles, product information, dealer locator assistance and much more.



P.O. Box 129  
Dunbridge, OH 43414  
419.352.1551  
[www.PrincipleBusinessEnterprises.com](http://www.PrincipleBusinessEnterprises.com)



Incontinence products  
produced in the U.S.A.  
(except item #2602)



PBE is the acknowledged leader in the following fields of care...

- **Incontinence Care** – Tranquility's family of absorbent products providing a scientific approach to moisture and risk management. [www.TranquilityProducts.com](http://www.TranquilityProducts.com)
- **Fall Prevention** – Pillow Paws® brand slippers deliver protection from the risk of slips and falls as well as exceptional hygienic protection and comfort. [www.PillowPaws.com](http://www.PillowPaws.com)
- **Wound Care** – Family of products designed to accelerate wound healing by removing biofilms and protecting the wound environment. [www.NovaGranProducts.com](http://www.NovaGranProducts.com)



© Tranquility, Select and Swimmates are registered trademarks of Principle Business Enterprises, Inc. | © Copyright 2017 | 2TRQ-802-SL17

800-467-3224 [www.TranquilityProducts.com](http://www.TranquilityProducts.com)