

BD Twinpak™

Dual Cannula Device



Helping all people
live healthy lives

Simplifying Needleless IV Access with a Truly Universal Device

The BD Twinpak is a truly universal IV access device, offering two methods of performance over traditional stand-alone devices. It features two cannulas in one shield: one red hub cannula for syringe filling and one clear blunt plastic cannula for safe split-septum access. The BD Twinpak Device is designed to help streamline IV work flow and reduce the number of individual pieces, thereby saving time and reducing costs.



BD Twinpak™

Dual Cannula Device

Working Together
for Simplified,
Convenient IV Access

The Self-Contained Solution



**BD™ Red Hub
Cannula Syringe
Filling Device:**
Provides Access to Vials



BD Twinpak Shield:
Holds Both Cannulas Securely



**BD™ Blunt
Plastic Cannula:**
*Allows for Safe, Needleless
Access to Split-Septum
Injection Sites*



Green Cap:
*Protects / Covers
BD Blunt Plastic
Cannula Prior to Use*

Red Hub Cannula

(Syringe Filling Device):

- **20 G Metal Cannula**
provides versatility in syringe filling from vials, and for other multiple vial access needs (e.g., reconstitution)
- **Blunt Tip Design**
provides a safer alternative to a traditional, sharp needle for syringe filling
- **Red Hub**
easily identifies the syringe filling device

Blunt Plastic Cannula

(with Green Cap):

- **Dual Side Ports**
provide greater fluid dispersion in flushing and drug delivery
- **Center-point Design**
features tapered tip for less resistance during insertion
- **Universally Compatible**
with split-septum cannula access devices, such as Baxter Interlink®, Hospira LifeShield®, B. Braun SafeLine®, and Alaris VersaSafe® injection sites as well as vials designed for needleless access
- **Green Cap**
designed to provide easy identification and a protective cover for clear BD Blunt Plastic Cannula Hub, and can later be used to cover the BD Red Hub Cannula



Secure Red Hub Cannula to the syringe. Remove shield.



After filling, recap the Red Hub Cannula into standing BD Twinpak Shield or use "scoop" method.



Flip the BD Twinpak and attach the clear BD Blunt Plastic Cannula for split-septum IV access.

BD Medical
9450 South State Street
Sandy, Utah 84070
1.888.237.2762
www.bd.com

| Reference No. | Description | Case Quantity | |
|---------------|-------------------------------|---------------|-----------|
| 303390 | BD Twinpak | 100/Box | 1000/Case |
| 303391 | 3 mL Syringe with BD Twinpak | 100/Box | 800/Case |
| 303392 | 5 mL Syringe with BD Twinpak | 100/Box | 400/Case |
| 303393 | 10 mL Syringe with BD Twinpak | 100/Box | 400/Case |

Interlink is a registered trademark of Baxter International, Inc.
LifeShield is a registered trademark of Hospira.
SafeLine is a registered trademark of B. Braun Medical, Inc.
VersaSafe is a registered trademark of Alaris Medical Systems, Inc.

BD, BD Logo and BD Twinpak are trademarks of Becton, Dickinson and Company. ©2006 BD.

BD 2/06-0320 D15729-1