



MC-002938

AsthmaMD™ Peak Flow Meter

Empower your asthma patients.

The AsthmaMD™ Peak Flow Meter helps patients measure their lung performance and anticipate attacks.

- Measures peak expiratory flow
- Lightweight, portable, and accurate
- Can be easily cleaned without disassembly
- Compatible with a convenient physician-developed mobile app



AsthmaMD™ Peak Flow Meter Fast Facts

Compatible with a convenient app for tracking peak expiratory flow*


Universal 60–810 LPM range

Colorful, integrated sliding zone indicators

Does not contain natural rubber latex or DEHP¹

Meets all ATS standards for portable Peak Flow Meters²



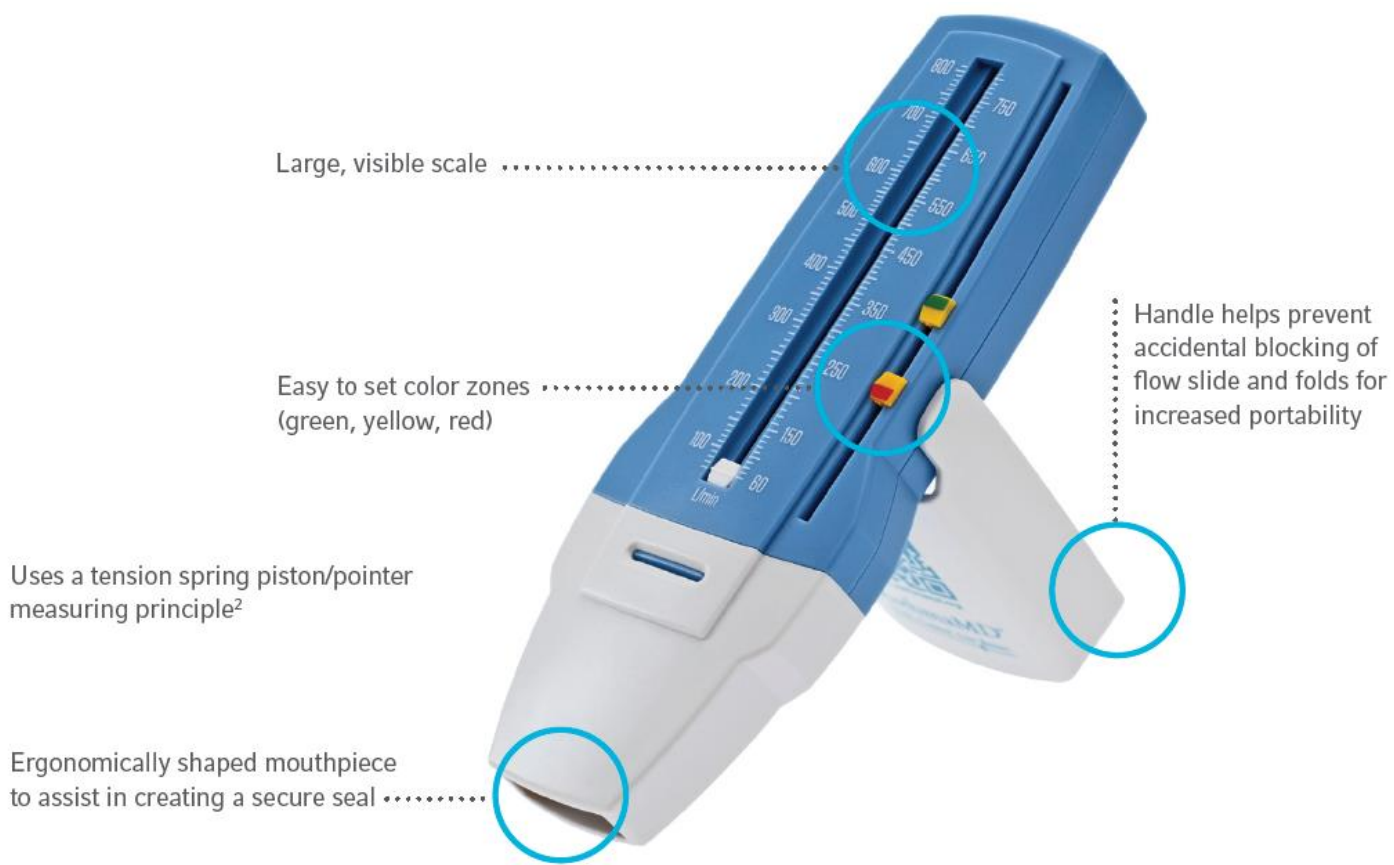


Large, visible scale

Easy to set color zones (green, yellow, red)

Uses a tension spring piston/pointer measuring principle²

Ergonomically shaped mouthpiece to assist in creating a secure seal

Handle helps prevent accidental blocking of flow slide and folds for increased portability

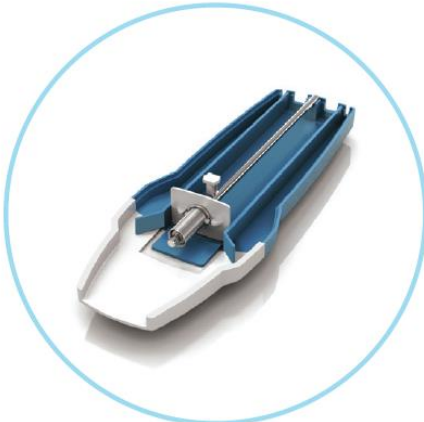

Parabolic side channels and a three-channel design help keep the device consistent¹

AsthmaMD™ Peak Flow Meter Mobile App*



Helps maintain a comprehensive asthma journal*



Tracks triggers, symptoms, medications, and notes*



Logs peak flow meter readings*



Visualize asthma activity on a 3-zone color graph*



Create customized alerts and reminders*




Easy for patient to share journal with their care team*

AsthmaMD™ Peak Flow Meter Mobile App*




Step 1

Forcefully blow into PFM 3 times, taking note of HIGHEST value

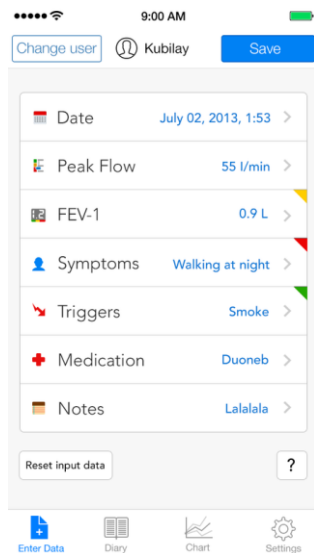


Step 2

Log Peak Flow, Symptoms, Triggers, Medication taken, and any notes

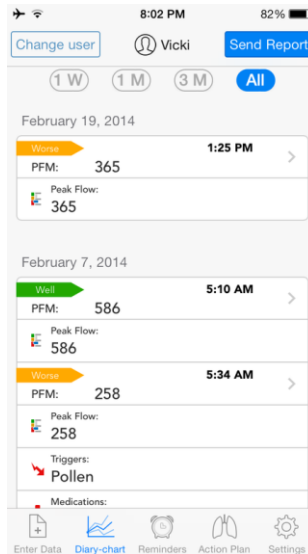


Step 3

Review of past Peak Flows via journal

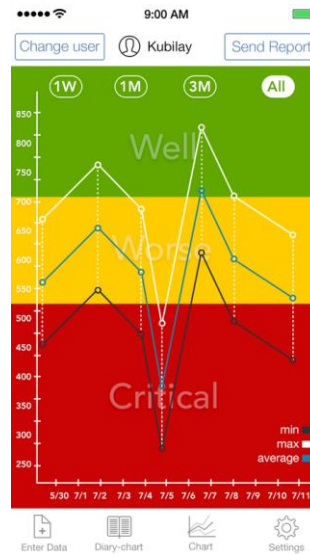

Step 4

Turn phone to view chart

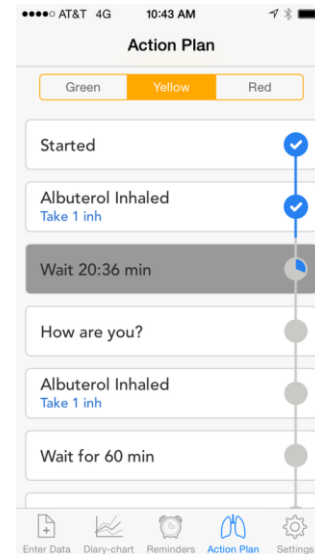


Step 5


If necessary, app can guide the patient through their predetermined action plan that was set up with their care provider

AsthmaMD™ Peak Flow Meter Mobile App*

Additional Features

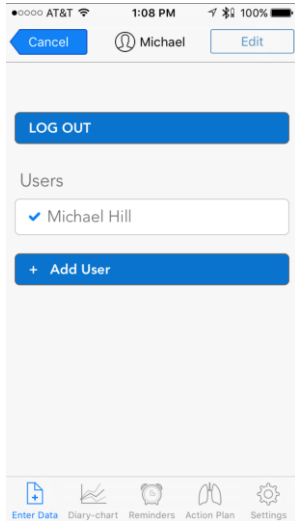


Easily switch
between multiple
users

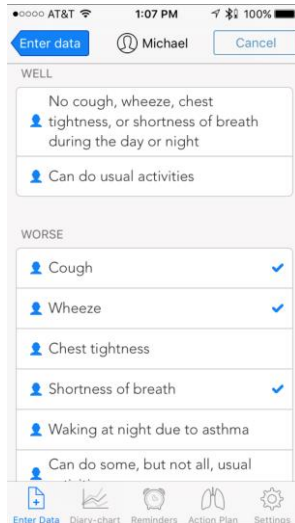


Symptoms List



Trigger List

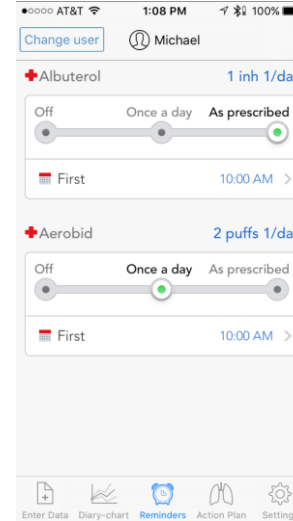

Integrated reminder system

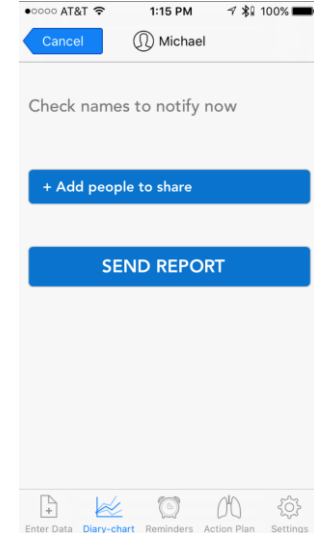

Easily share report
with care team

Benefits



The Clinician

Accurate, precise,
and designed for
children and adults¹



Your Institution

Meets all ATS standards
for portable Peak
Flow Meters²



The Patient

Durable, lightweight,
and easy to use.

AsthmaMD™ Peak Flow Meter

ITEM CODE	SIZE	CASE QUANTITY
1802	AsthmaMD™ Peak Flow Meter	10

Specifications

Range	60–810 l/m (ISO23747, ATS)
Scale Increment	10 l/m
Accuracy	<+/- 10% or 10 l/m
Design	Single-Patient Use
Standard Compliance	ISO 23747, ATS 26 Flow-Time Waveform Testing, FDA 820
Cleaning Instructions	Clean at least once every two weeks. Hand wash: wash with warm water and mild liquid soap. Rinse thoroughly and air dry completely prior to use. Do not boil AsthmaMD™ Peak Flow meter. With proper care the AsthmaMD™ Peak Flow meter can last up to 2 years ¹



*DISCLAIMER: The AsthmaMD™ mobile app is a third-party app offered by AsthmaMD, Inc. Teleflex has no control over such third-party app, nor does Teleflex review, approve, monitor, endorse, or make any representations or warranties with respect to such third-party app. Phone also not included.



To place an order call 1.866.246.6900

Teleflex®

References

1. Data on File, Noble International, Inc.
2. 510(k): K101380

[Click for price and product description](#)



*DISCLAIMER: The AsthmaMD™ mobile app is a third-party app offered by AsthmaMD, Inc. Teleflex has no control over such third-party app, nor does Teleflex review, approve, monitor, endorse, or make any representations or warranties with respect to such third-party app. Phone also not included.

Any Questions?

Thank You

Thank You