

# Mepilex® Border


Self-adherent, absorbent foam dressing with Safetac® technology

## Backing film with dynamic permeability

- Advanced fluid handling
- Viral proof, bacterial proof & waterproof\*
- Thin and conformable with low friction coefficient for secure fixation

## Safetac® soft silicone layer

- Secure fixation
- Atraumatic to the wound bed and the surrounding skin on removal
- Reduces risk of maceration

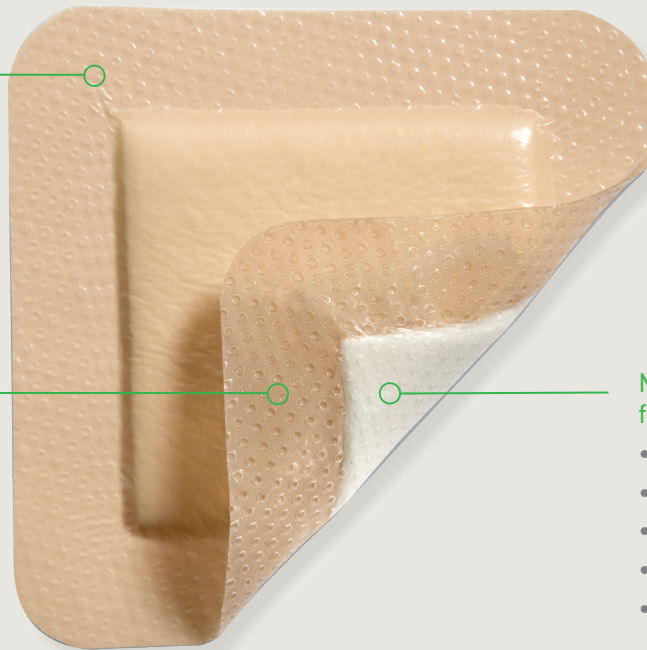

## Multi-layered "moisture control" foam pad

- Advanced multi-layered construction
- Provides optimal fluid uptake
- Maintains integrity
- Conforms to body contours
- Promotes optimal moisture balance

\* microbes larger than 25 nm

**Safetac**  
TECHNOLOGY

- Advanced fluid handling promotes optimal moisture balance and minimizes the risk of maceration
- Minimizes trauma and pain at dressing change
- Viral proof, bacterial proof and waterproof film to protect the patient and the clinician

## Proven choice for a better outcome

Safetac\*, pioneered by Mölnlycke, delivers above and beyond the ordinary. Proven to help optimize the wound healing journey and even prevent wounds, dressings with Safetac are the safe choice for patients and a champion for higher standards in wound care.

In fact, we have a wealth of evidence that supports the clinical and economic benefits of dressings with Safetac, including Mepilex®, Mepitel®, Mepiform® and Mepitac®. To date, these dressings have helped millions of patients worldwide<sup>7-9</sup>.


\* A unique proprietary technology exclusive to Mölnlycke Health Care

**Safetac**  
TECHNOLOGY


**Mepilex**® Border

  
**Mölnlycke**®

## Directions for use

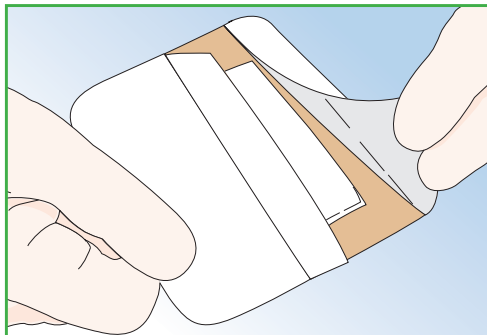

**1.** Gently cleanse the wound; dry surrounding skin. Choose a dressing size that will allow the adherent border to overlap the wound area by at least 2cm. Remove the release film.

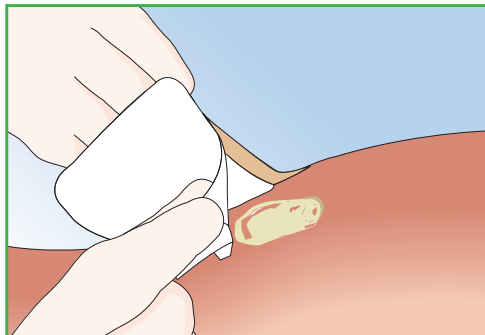

**2.** Lay absorbent foam pad over wound area. Gently smooth adherent border in place. Do not stretch. Mepilex® Border can remain in place for up to 7 days depending on drainage and wound condition.

## How Mepilex® Border works

Mepilex® Border is a self-adherent foam dressing with advanced fluid handling capabilities, dynamic permeability and all the advantages of Safetac® soft silicone technology. It adheres gently to surrounding skin but doesn't adhere to moist wound tissue, thereby minimizing trauma to the wound and peri-wound area and pain to the patient. It is waterproof, viral proof and bacterial proof with a multilayered wound pad construction designed to promote optimal moisture balance and fluid uptake.

## Benefits of Mepilex® Border

- No secondary fixation required
- Viral/bacterial/waterproof
- Does not adhere to moist wound surfaces
- Can remain in place for up to 7 days depending on wound condition and exudate level
- Maintains a moist wound environment
- Conforms to body contours

## Indications for use

Mepilex® Border is indicated for use on a wide range of wounds including:

- Leg ulcers
- Pressure injuries
- Traumatic wounds
- Surgical wounds
- Painful wounds
- Wounds with compromised peri-wound skin
- Mepilex® Border can be used as a primary dressing on shallow wounds or as a cover dressing for deeper wounds. Ideal for wounds with moderate to heavy exudate.

## Mepilex® Border Assortment (Sterile packed)

Art. no	Size cm	Pcs/Box	Pcs/Case
295200	7.5 x 7.5	5	70
295300	10 x 10	5	50
295000	12.5 x 12.5	5	50
295400	15 x 15	5	50
295600	15 x 20	5	50
295800	10 x 20	5	25
295900	10 x 30	5	25

