

## two year limited warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for 2 years from the date of purchase for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Limited 2 Year Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2014 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

## dining tray

item # FC 2024 & FC 4024

for use with first class chairs  
(item # FC 2000N & FC 4000N)



EU Authorized Representative  
Drive Medical LTD  
Ainley's Industrial Estate  
Elland, West Yorkshire,  
United Kingdom HX5 9JP



## set up instructions

1. Move armrests on chair to the down position and slide tray onto armrests to the appropriate depth. Cam handle attaches to the underside of the armrest.



2. To tighten the tray to armrests, rotate the cam handle approximately 180°.



Item # FC 2024  
Shown