

## Cleaning, Care & Routine Maintenance

### Routine Maintenance

Servicing of the internal components inside the cabinet of the Eclipse must be conducted by a CAIRE trained and qualified service technician as needed.

Preventative Maintenance (PM) to be completed on an annual or yearly basis. Consult your provider to arrange PM schedule. Your home care provider or qualified technician will perform inspections of the alarms, internal 9-volt battery, battery, internal filters and other internal parts only as needed.

### User Care and Cleaning of the Device

#### Cannula Replacement

Replace your supply tubing and cannula on a regular basis as recommended by your home care provider. Your physician or home care provider will provide you with cleaning, disinfection and replacement information.

Note: Always follow the cannula manufacturer's instructions for proper use. Replace the disposable cannula as recommended by the cannula manufacturer or your Equipment Provider. Additional supplies are available from your Equipment Provider.

#### Air Inlet Filter

Ambient air is drawn into the device through the air inlet located on the top rear of the device. Cleaning the air inlet filter is the most important maintenance activity you will perform as a user. Cleaning the air inlet filter will keep your Eclipse performing properly. Check this air inlet filter daily and clean it at least once a week using the following procedure:

1. Remove the air inlet filter
2. Wash the filter in warm tap water using a mild soap detergent solution
3. Rinse the filter thoroughly with clear tap water and squeeze out the excess water
4. Allow the filter to air dry
5. Reinsert the filter in the cabinet



Cleaning tip: Keep a clean second filter as a replacement for use when one filter is drying.



**CAUTION:** Do not operate the Oxygen Concentrator without the air intake filter in place. If a second filter is provided, insert the "replacement" filter before you clean the dirty filter. Clean the dirty filter in a warm soap and water solution then dry thoroughly prior to use.

Note: The Manufacturer does not recommend the sterilization of this equipment.

Note: Do not operate the Oxygen Concentrator without the air intake filter in place. If a second filter is provided, insert the "replacement" filter before you clean the dirty filter. Clean the dirty filter in a warm soap and water solution then dry thoroughly prior to use.

The air inlet filter should be replaced annually. If the Eclipse is used in a dusty environment, the filter may need to be replaced more frequently. You should not operate the Eclipse without an air inlet filter installed for more than 5 minutes.