

5 Quick Steps to Recharge Your Battery.

INOGENONE[®] G2 External Battery Charger

Your Equipment Provider:

Your Physician:




326 Bollay Drive,
Goleta, CA 93117
1-877-4-INOGEN (Toll Free)
1-877-446-6436 (Toll Free)
1-805-562-0515 (International)
info@inogen.net
www.inogenone.com
www.OxygeNation.com

96-02857-00-01A



INOGEN ONE[®]G2 External Battery Charger

Your Inogen One G2 External Battery Charger (BA-203) will charge your Inogen One G2 12 and 24 cell batteries. It takes approximately 3 hours to charge a fully depleted 12 cell battery, and approximately 6 hours to charge a fully depleted 24 cell battery. Note: the battery will also recharge in the Inogen One G2 when powered by the Universal Power Supply.

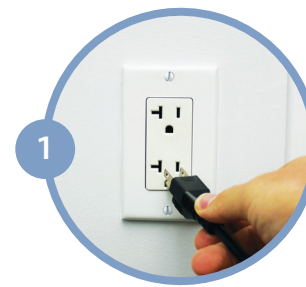

1 Plug the External Battery Charger AC power supply cord into the nearest electrical outlet.



2 Plug the External Battery Charger AC power supply into the battery charger.



3 Slide your charger onto the Inogen One G2 Battery by clicking and locking into the charger.



4 When the battery is in the correct position, a solid red light will indicate that the battery is charging.



5 When the green light illuminates, the battery is fully charged.*

