

limited two year warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for two years from the date of purchase for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Limited Two Year Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2014 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

Rev.3.04.18.14

hip guide

item# FC 8027N

(for use with item# FC 2000N & FC 4000N)



EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

WenzeliteRe/habTM
drive

assembly instructions

1. Insert hip guides as in figure 1.
2. Insert knobs (A). Push/ pull to adjust width.


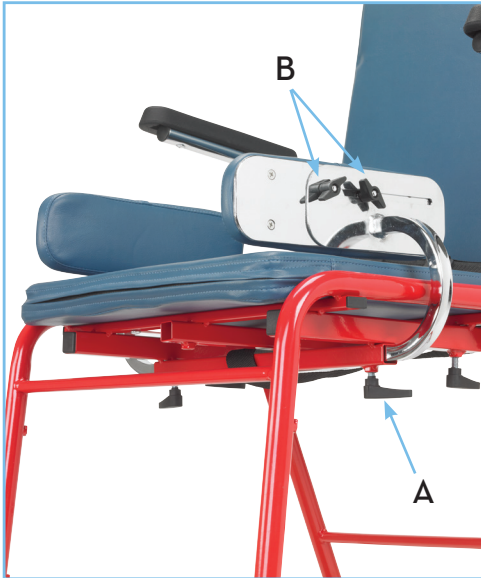


figure 1

adjustments

1. Loosen knobs (B) and move plates forward or backward to adjust depth of hip guides. (figure 1)

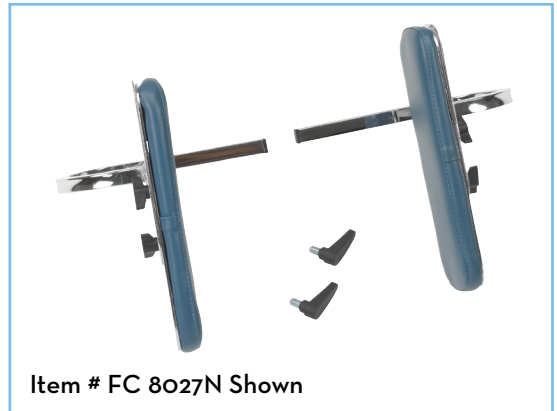

Item # FC 8027N Shown



Item # FC 8027N
with FC 4000N Shown