


Dedicated to Delivery
Nasco
Life/form®



**Patient Education
Tracheostomy Care Set
LF01159U
Instruction Manual**



Life/form® Products by Nasco

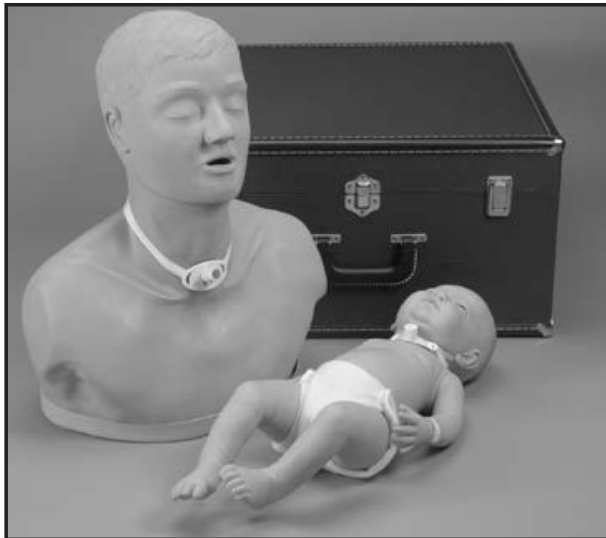

About the Manikins

The **Life/form**® Patient Education Tracheostomy Care Set is ideal for teaching patients and caregivers the skills they will need to perform at home. The adult manikin includes oral and nasal passages, the pharynx, epiglottis, trachea, esophagus, tracheostomy site, and cricoid cartilage. The mouth and jaw are flexible to allow oral suctioning. The trachea branches into the left and right bronchial trees. Students can practice suctioning techniques, proper cuff inflation, dressing changes and other techniques. Cleansing and maintenance of the external tracheal site can be performed just as on an actual patient. The infant manikin can be used to teach patients and caregivers basic tracheostomy care skills.

List of Components

- Adult Tracheostomy Care Manikin
- Infant Tracheostomy Care Manikin
- Adult Tracheostomy Tube
- Infant Tracheostomy Tube
- Carrying/Storage Case
- **Life/form**® Lubricant

General Information About the Manikins

The manikins may be used in all methods of tracheostomy care. Individual methods and procedures may differ. Therefore, this booklet does not propose to outline a particular procedure to be followed. Rather, it is recommended that a procedural manual and instructor's directions be followed. This will eliminate confusion and best serve the individual instructor and curriculum.

Procedures That May Be Performed on the Adult Manikin

- Oral-pharyngeal suctioning
- Nasal tracheal suctioning
- Tracheal suctioning via tracheostomy

Procedures That Can Be Performed on the Adult and Infant Manikin

- Dressing changes
- Stoma cleansing
- Changing of tracheostomy tube
- Cuff inflation
- Changing tracheostomy ties

Anatomy Present on the Adult Manikin

- Oral Passages
- Nasal Passages
- Pharynx
- Epiglottis
- Trachea
- Esophagus
- Stoma
- Right and Left Bronchi

Care of the Manikin

Normal surface soil can be removed with a mild soap and lukewarm water. Use Nasco Cleaner (LF09919U) to remove stubborn stains. Simply apply the Nasco Cleaner to the soiled area and wipe with a clean, soft paper towel.

Cautions

1. Do not cleanse the stoma area with Betadine® or any other similar, colored disinfectant. NEVER place the trainer on any kind of printed paper or plastic. These materials, as well as ball-point pens, will transfer indelible stains. Do not use any cosmetics on the simulator.
2. With repeated use, the suctioning catheter will become soft and pliable, making it difficult to insert. You must either use a new catheter, or place the old one in a refrigerator or on ice for a few minutes to stiffen it.
3. Heavy lubrication must be used while practicing procedures. The manikin will not function properly unless heavy lubrication is used. Use **Life/form**® Lubricant only. Do not use a water soluble lubricant.
4. Exercise the same care with this manikin as would be used with an actual patient. Excessive pressure and abuse will damage and shorten the unit's life.

Supplies/Replacement Parts for the Patient Education Tracheostomy Care Set

LF09919U Nasco Cleaner

LF00985U Lubricant Kit

LF01170U Adult Tracheostomy Tube

LF01171U Infant Tracheostomy Tube

Other Available *Life/form*® Simulators

- LF00698U** Adult Injectable Arm (White)
- LF00855U** Male Catheterization
- LF00856U** Female Catheterization
- LF00901U** Prostate Examination
- LF00906U** Ostomy Care
- LF00929U** Surgical Bandaging
- LF00957U** Enema Administration
- LF00958U** Pediatric Injectable Arm
- LF00961U** Intramuscular Injection
- LF00984U** Breast Examination
- LF00995U** Arterial Puncture Arm
- LF00999U** Pediatric Injectable Head
- LF01005U** First Aid Arm
- LF01008U** Intradermal Injection Arm
- LF01012U** Heart Catheterization (TPN)
- LF01019U** Ear Examination
- LF01027U** Peritoneal Dialysis
- LF01028U** Suture Practice Arm
- LF01034U** Suture Practice Leg
- LF01036U** Spinal Injection
- LF01037U** Hemodialysis Practice Arm
- LF01038U** Episiotomy Suturing Set
- LF01042U** Suture Kit
- LF01062U** Pelvic, Normal & Abnormal
- LF01063U** Stump Bandaging, Upper
- LF01064U** Stump Bandaging, Lower
- LF01069U** Cervical Effacement
- LF01070U** Birthing Station
- LF01082U** Cricothyrotomy
- LF01083U** Tracheostomy Care
- LF01084U** Sigmoidoscopic Examination
- LF01087U** Central Venous Cannulation
- LF01095U** Blood Pressure Arm
- LF01108U** Infant Intraosseous Infusion
- LF01121U** Advanced IV Arm
- LF01131U** Venipuncture and Injection Arm
- LF01139U** Advanced IV Hand
- LF01142U** Auscultation Trainer
- LF01143U** Testicular Exam
- LF01152U** Male & Female Catheter
- LF01155U** Advanced CPR Dog
- LF01162U** Venatech IV Trainer
- LF01174U** NG Tube & Trach Skills
- LF01184U** Venatech IM & Sub Q
- LF01193U** Special Needs Baby
- LF03000U** **CPARLENE**® Series
- LF03601U** Adult Airway Management Trainer with Stand
- LF03602U** Adult Airway Management Manikin
- LF03609U** Child Airway Management Trainer with Stand
- LF03616U** Child **CRiSis**™ Manikin
- LF03617U** Deluxe Child **CRiSis**™ Manikin with Arrhythmia Tutor
- LF03620U** PALS Update Kit
- LF03623U** Infant Airway Management Trainer with Stand
- LF03632U** Child Intraosseous Infusion/ Femoral Access Leg on a Stand
- LF03633U** Child Airway Management Trainer Torso
- LF03693U** **Basic Buddy**® CPR Manikin
- LF03699U** "Airway Larry" Airway Management Trainer
- LF03709U** Infant **CRiSis**™ Manikin
- LF03720U** **Baby Buddy**™ Infant CPR Manikin
- LF03750U** Fat Old Fred
- LF03760U** Airway Management/Cricoid Pressure Trainer
- LF03770U** Chest Tube
- LF03953U** **CRiSis**™ Manikin, Complete
- LF03955U** Deluxe **CRiSis**™ Manikin
- LF03956U** Deluxe "Plus" **CRiSis**™ Manikin
- LF03965U** Adult **CRiSis**™ Auscultation Manikin
- LF03966U** Adult **CRiSis**™ Auscultation Manikin with ECG Simulator
- LF04000U** **GERi**™/**KERi**™ Manikin Series
- LF04200U** Adult Sternal Intraosseous Infusion
- LF06001U** CPR Prompt® Adult/Child Manikin
- LF06012U** CPR Prompt® Infant Manikin
- LF06200U** CPR Prompt® Keychain Rescue Aid
- LF06204U** CPR Prompt® Rescue and Practice Aid

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1-800-558-9595

eNasco.com • E-mail: lifeform@eNasco.com

COPYRIGHT © NASCO 1999 PRINTED IN U.S.A. NP 33-00/RV 8-12