

CORPAK MedSystems

INSTRUCTIONS FOR NG AND J-TUBE USE

1.

CLOG ZAPPER® Applicator

Pump Set

- Turn feeding pump OFF and close roller clamp on administration set.

NG Tube

EXCESSIVE FORCE

2.

A

10 cc syringe

Water

B

- Fill syringe to 10 cc mark with water (7.1 cc).

3.

Cap

LUMPS

SHAKE! SHAKE! SHAKE!

4.

NG Tube

- Fill tube with a recommended initial dosage of 2-6 cc of the CLOG ZAPPER solution.

5.

Remove applicator from tube.

PLUG tube.

- Keep APPLICATOR and SYRINGE together with patient if procedure must be repeated.

6.

KEEP SOLUTION IN TUBING FOR 60 MINUTES

30 MINUTES MINIMUM

7.

A

6 cc syringe

Water

B

Fill syringe completely.

8. FLUSH! with WATER!

NG Tube

9.

- If tubing cannot be flushed, REPEAT steps #4. thru #8.

Always check with physician to determine how many times the process can be repeated.

10.

After tube clears ...

- Dispose per institution's protocol
- and RESUME FEEDING.

DISPOSAL

CAUTION

- Protective eyewear is strongly suggested. CLOG ZAPPER can cause irritation if it gets in contact with the eyes.

STORAGE REQUIREMENTS

- CLOG ZAPPER, once reconstituted, should be used within 24 HOURS.
- REFRIGERATE mixture while not in use.

INSTRUCTIONS FOR G-TUBE USE

- Cut APPLICATOR at dot.
- Discard excess tubing.
- Place APPLICATOR into G-TUBE's Administration Set port.
- Continue with NG TUBE INSTRUCTIONS above.

INSTRUCTIONS FOR LOW PROFILE DEVICES

- Place FEEDING SET into LOW PROFILE DEVICE following manufacturer's instructions.
- Place APPLICATOR into FEEDING SET.
- Continue with NG TUBE INSTRUCTIONS above.