
## MIC\* EXTENSION TUBING

### Instructions for Use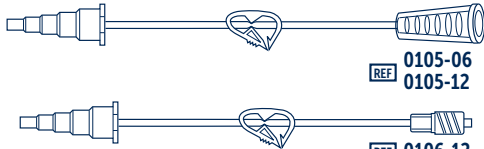

REF 0105-06  
0105-12

REF 0106-12

Nonsterile	Not made with natural rubber latex	Product is NOT made with DEHP as a plasticizer
Rx Only	Caution	Consult instructions for use

Distributed in the USA by Halyard Sales, LLC, Alpharetta, GA 30004  
In USA, please call 1-844-425-9273 • halyardhealth.com

Halyard Health, Inc., 5405 Windward Parkway, Alpharetta, GA 30004 USA

Halyard Belgium BVBA, Leonardo Da Vinciilaan 1, 1930 Zaventem, Belgium

Sponsored in Australia by Halyard Australia Pty Limited, 52 Alfred Street, Milsons Point, NSW 2061

\*Registered Trademark or Trademark of Halyard Health, Inc., or its affiliates. © HYH 2015-07-13

15-H1-267-0-00 / 70168788

## MIC\* Extension Tubing

### Instructions for Use

Rx Only: Federal Law (USA) restricts this device to sale by or on the order of a physician.

- Attach a feeding syringe or set to the MIC\* Extension Set and flush.
- Insert the MIC\* Extension Set connector into the open feeding port. Use a FIRM twist to make a secure connection. Begin feeding. After feeding, flush the MIC\* Extension Set with 5–10 ml of warm water. During continuous feeding, flush approximately every 4 hours.
- Disconnect the feeding set from the feeding port with a FIRM twist.

**Note:** Always hold the universal connector firmly when handling the MIC\* Gastrostomy Tube.

- Irrigate the MIC\* Gastrostomy Tube with warm water using 10 ml for children or 20 ml for adults, injected through the feeding port.
- Close the feeding port securely.

#### Single patient use only.

Not made with natural rubber latex.

Product is not made with DEHP as a plasticizer.

**Warning: For Enteral Nutrition and/or Medication Only.**

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