

Visit us online
at Skil-Care.com

Skil-Care[™]
CORPORATION

29 Wells Avenue
Yonkers, NY 10701

914-963-2040
800-431-2972

SLIP-GRIP

Re-Order #	Color	Size	Unit
☐909250	Blue	12" x 144"	Roll

Skil-Care[™]
CORPORATION

Advanced design for nursing and therapy.[™]

Purpose

Slip-Grip is a nonslip matting that may be used to prevent cushions from slipping off chairs; hold plates so that unsteady eaters can feed themselves; and position instruments on uneven surfaces. Sold in 12: x 144" rolls, Slip-Grip may be cut to required size.

Important Safety Information

Slip-Grip loses its nonslip quality on wet surfaces. Do not use Slip-Grip on shower or bathroom floors. To avoid skin shearing, do not place Slip-Grip in direct contact with resident's skin.

Cleaning

Slip-Grip may be washed with soap and water and air dried. Do not machine wash and dry. Do not autoclave. Make certain that Slip-Grip is completely dry before using. Slip-Grip can EO sterilized.

[Click here to view detailed product descriptions and prices.](#)

Call 800-397-5899 to order today!