

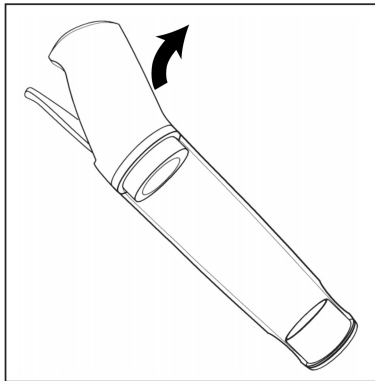


How to Load the Tension Ring Using the Easy-Action® Ring Applicator

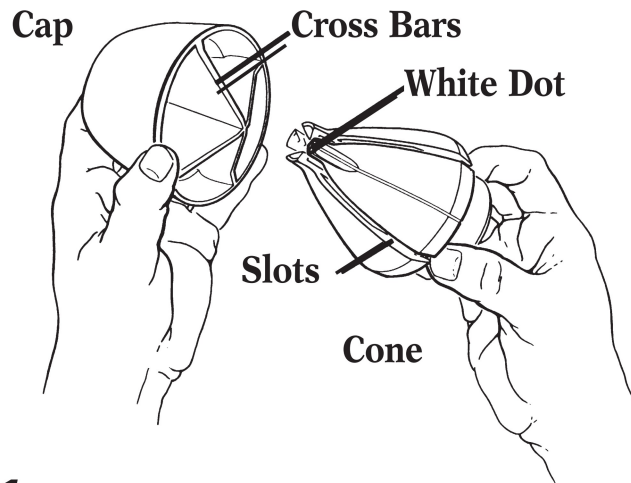


Step 1

Remove the pump and the insert from the cylinder. Put a towel on a table or counter top and stand the cylinder on top of the towel with the large end of the cylinder face down. The towel will keep the cylinder from slipping.

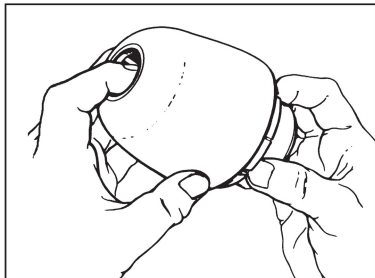


Step 2

Separate the Easy-Action ring applicator by holding the base of the cone in your hand and then pushing the button on the top of the cap. Place the base of the ring applicator cone in the small open end of the cylinder. Twist the cone so that the white dot on top of the cone lines up with the cylinder groove **opposite** the cylinder label.

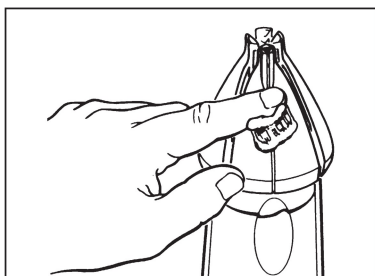


Step 3

Lubricate the entire cone surface with Osbon Personal Lubricant.™

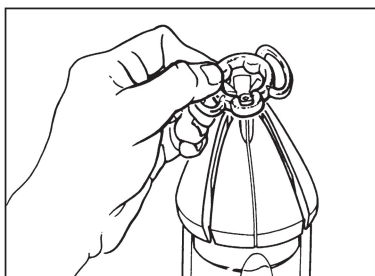


Step 4

Place the tension ring on the cone, making sure to align the **U-notch** on the tension ring with the **white dot** on the cone.

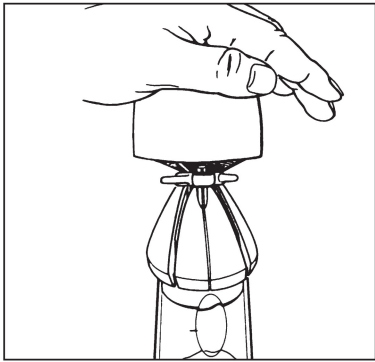


Step 5

Place the cap back on the cone and tension ring, lining up the cross bars of the underside of the cap with the slots on the top of the cone.

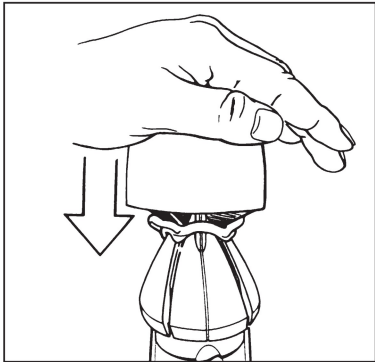

Step 6

Then quickly and firmly press straight down on the cap of the ring applicator with one hand while holding the cylinder in an upright position on the towel with your other hand to help stabilize it.

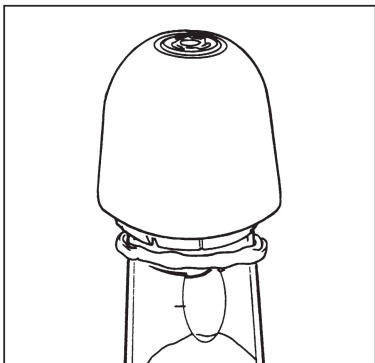

Step 7

The tension ring should now be transferred to the cylinder in the correct position. If you need to add additional rings to the cylinder, slide the first ring 1/4" towards the pump end of the cylinder to allow space for the additional ring(s). Then, choose another ring and repeat steps 2 through 6.



Step 8

After you have loaded your tension ring(s), remove your ring applicator from the cylinder and replace the insert. Place the pump handle back on to the cylinder. You are now ready to create an erection with your ErecAid® Esteem® System.