

MODEL NO. IRC \_\_\_\_\_ SERIAL NO. \_\_\_\_\_



## PREVENTIVE MAINTENANCE RECORD

ON EACH INSPECTION

Yes, you can:

Record Date of Service																				
Record Elapsed Hours On Hour Meter																				
Clean Cabinet Filter(s)																				
Check Prescribed L/min. Flow Rate																				
<b>EVERY 4,380 HOURS</b>																				
Check Oxygen Concentration (UNITS WITHOUT SensO <sub>2</sub> )																				
<b>EVERY 26,280 HOURS OR 3 YEARS, WHICHEVER COMES FIRST</b>																				
Check Oxygen Concentration (UNITS WITH SensO <sub>2</sub> )																				
<b>DURING PREVENTIVE MAINTENANCE SCHEDULE, OR BETWEEN PATIENTS</b>																				
Clean/Replace Cabinet Filter(s)																				
Check Outlet HEPA Filter*																				
Check Compressor Inlet Filter*																				
Check Power Loss Alarm																				
<b>AS REQUIRED</b>																				
Clean Heat Exchanger																				
Replace Exhaust Muffler																				
Rebuild Top End of Compressor																				

\*NOTE: Refer to Preventive Maintenance section of Service Manual.

NOTE:

4,380 hours are equivalent to usage 24 hours per day, 7 days per week, for 6 months.

26,280 hours are equivalent to usage 24 hours per day, 7 days per week, for 3 years.

FIGURE 6.2 Preventive Maintenance Record