

Lock 'n Roll Microseal Closure

Purpose


- A drainable pouch is open at the bottom to allow emptying. It requires a closure on the bottom to keep it closed
- Empty your pouch when it is 1/3 to 1/2 full of discharge or gas
- The **Lock 'n Roll** microseal closure is an integrated closure on a drainable pouch

Before you apply your new pouch, close the pouch tail (see Figure 4 and Figure 5)



Opening

Lift from the center of the transparent flap

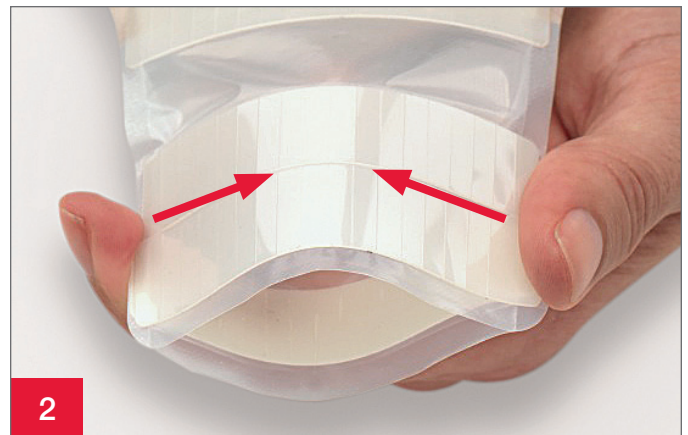

Draining

Position fingers on both sides of the pouch tail, between the ribbed strips, and squeeze from the sides. Continue to hold the pouch open for cleaning




Cleaning

Clean and dry the inside of the pouch tail. Wipe the outside edge of the pouch tail



Closing

Roll the pouch tail up tightly three times. Fold flap down aligning the fasteners together (Figure 4)

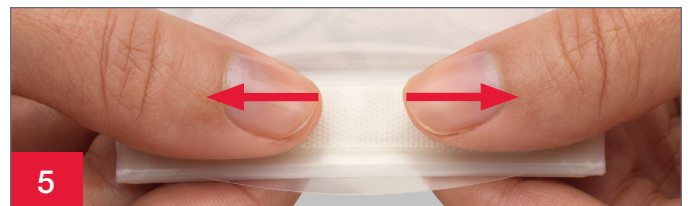

To close, start at the center and press. Continue pressing to the outer edges. You will feel the fasteners seal together (Figure 5)

Lock 'n Roll Microseal Closure

FREQUENTLY ASKED QUESTIONS

Q: How do I know my pouch is securely fastened?

A: Aligning and pressing the center of the security flap to the outer edges will allow you to feel the interlocking fasteners seal and ensure your pouch is securely closed.

Q: Will the closure work when it is wet?

A: The material used on the closure is water-resistant and can be securely closed even when it is wet, but will work best when dry.

Q: Will the closure show through my clothes?

A: The closure is designed to be discreet with its slim profile, and is not noticeable underneath most clothes.

Q: Is the closure comfortable against my body?

A: The slim profile of the curved outlet is designed to conform to the body's natural curves providing the utmost comfort.

Q: Will the closure open accidentally?

A: Laboratory testing has shown that the interlocking fasteners are secure when closed properly.

Q: My output is very liquid. Are there any suggestions to make emptying easier?

A: With the closure's unique design, you may want to control the draining of the liquid contents by unrolling the pouch tail two times and holding the last flap up. You can then unroll this flap slowly.

Q: Will people hear me open the pouch when I need to empty it in a public restroom?

A: The material used on the closure is quiet and will not make a "ripping" noise when the security flap is opened.

For product questions, sampling needs, or detailed clinical questions concerning our products in the US, call **1.888.808.7456**. In Canada call **1.800.263.7400**.

Routine follow-up with your healthcare professional is recommended.

Prior to using any ostomy products/accessories be sure to read all product inserts and labels.

The Hollister logo, Lock 'n Roll and "Healthy skin. Positive outcomes." are trademarks of Hollister Incorporated.
© 2017 Hollister Incorporated. 923068-1117 US-00195

Hollister Incorporated
2000 Hollister Drive
Libertyville, Illinois 60048
1.800.323.4060

www.hollister.com

Hollister Limited
95 Mary Street
Aurora, Ontario L4G 1G3
1.800.263.7400