

AIRIAL CHAMBER

HOLDING CHAMBER FOR METER DOSE INHALERS


Model # MQ8000

1




Remove MDI canister mouthpiece cover. Insert MDI canister mouthpiece into the elastic opening at the back of the AIRIAL CHAMBER.

2

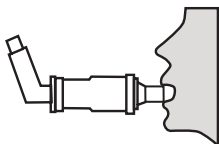


Insert CHAMBER mouthpiece between teeth, seal lips around mouthpiece.

3

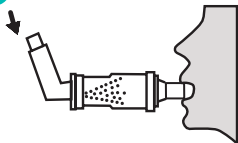

Sit up straight and exhale. Release one puff of MDI Medication into CHAMBER.

4

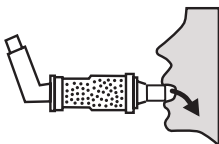

Take a slow, deep breath through the mouthpiece. Hold breath for 5-10 seconds.

driveTM

AEROCÁMARA AIRIAL

AEROCÁMARA PARA INHALADORES DE MEDICAMENTO


Artículo # MQ8000

1




Retire MDI cubierta de la boquilla bote. Inserte MDI bote boquilla en la abertura elástica en la parte posterior de la AIRIAL CÁMARA.

2

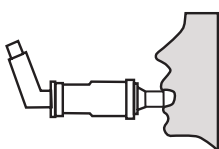


Inserte la boquilla CÁMARA entre los dientes, cerrar los labios alrededor de la boquilla.

3

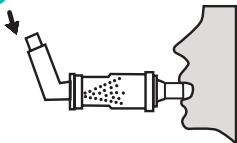

Siéntese derecho y exhala. Suelte una bocanada de MDI medicación dentro de la cámara.

4

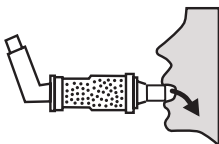

Tome una respiración lenta y profunda a través de la boquilla. Mantenga la respiración durante 5 a 10 segundos.