



OPERATING INSTRUCTIONS

MDF® 747 Dual Head Stethoscope (also for MDF® 747C)
MDF® 747XP Acoustica™ XP Deluxe Dual Head Stethoscope
MDF® 777 MD One™ Stainless Steel Dual Head Stethoscope (also for MDF® 777C and MDF® 777I)
MDF® 787XP Infant and Neonatal Stethoscope (also for MDF® 787)
MDF® 797 Classic Cardiology™ Stethoscope
MDF® 797X ProCardial ERA™ Stethoscope
MDF® 797DD ER Premier™ Stethoscope
MDF® 797CC ProCardial™ C3 Stethoscope

Read instructions carefully before using and save for future reference.


General Safety: This MDF® stethoscope is designed exclusively as a non-invasive medical auscultation instrument to detect heart and lung sounds on healthy skin around the external thoracic region. Due to its sound amplification function, it may cause damage to user's ears if used other than its intended medical diagnostic purpose. Use only if you are a medical professional and do not leave stethoscope unattended around children.



General Usage: MDF® stethoscope with ErgonoMax™ headsets are ergonomically pre-angled. Models with AccuFit™ headsets are anatomically adjustable for best fit, comfort, and sound transmission. A headset adjusted to a 15° forward angle is standard, and should be worn with the eartips pointed away from you if it is held directly in front of you. The dual-head chestpiece contains a diaphragm side for detecting high-frequency sounds, and a bell side for detecting low-frequency sounds. According to the auscultation need, simply turn the fully-rotational acoustic valve stem to select the proper side of the chestpiece. Hold the chestpiece firmly against the desired auscultation location on the patient's external thoracic region. Prior to each use, make sure all parts are securely fastened and properly connected.

General Cleaning & Care: Clean stethoscope thoroughly before each use. Use clean cotton cloth or swab and isopropyl alcohol solution to cleanse all soiled parts. Avoid prolonged exposure of acoustic tubing to excessive heat, cold, solvents, moisture and oils/lipids (secreted by human skin).

Our commitment to high-quality instruments is backed by our lifetime warranty on stethoscopes.

SYMBOL DEFINITION:

 Batch Code
 Date of Manufacture
 Conforms to European Standard

 Authorized Representative in the European Community
 Registered Trademark of Medifriend and MDF® Instruments USA

AUTHORIZED REPRESENTATIVE IN EUROPEAN COMMUNITY
 Medical Technology Promedt Consulting GmbH
Altenhofstrasse 80, 66386 St. Ingbert, Germany
e: info@mt-procons.com



Distributed by:

MDFinstruments.com

- ☐ MDF® Instruments USA, Inc. 26665 Seagull Way Suite 117 Malibu CA 90265 USA
- ☐ MDF® Instruments Direct, Inc. 5016 Chesebro Road, Suite 100 Agoura Hills, CA 91301 USA