

Don't replace clogged  
feeding devices...

# ZAP 'EM with CLOG ZAPPER™

- Safely zaps clogs in all types of feeding devices:  
NG Tubes, G Tubes, PEGs, J Tubes and Low  
Profile Devices.
- Saves valuable time by utilizing this pre-mixed,  
pre-measured, and pre-loaded system.
- Eliminates costly reinsertion and placement  
confirmation charges.

- CLOG ZAPPER is a patented, food-grade  
powder, designed to break up formula  
clogs in most enteral feeding devices.
- CLOG ZAPPER combines an "enzyme  
cocktail," acids, buffers, anti-bacterial  
agents, and metal inhibitors in an  
all-inclusive, ready-to-use system.
- CLOG ZAPPER ingredients all work  
together to loosen, break down and  
dislodge formula clogs.
- In clinical trials\* CLOG ZAPPER was 100%  
successful in restoring patency to feeding  
tubes clogged with enteral formulas.

| <b>Ingredient</b>     | <b>Benefit</b>   |
|-----------------------|--|
| Maltodextrin*         | Density enhancer                                       |
| Papain                | Breaks down proteins                                   |
| Alpha-Amylase         | Breaks down starches                                   |
| Cellulase             | Breaks down cellulose (fiber)                          |
| Citric Acid*          | Lowers pH, helps enzymes,<br>antibacterial agent       |
| Disodium Phosphate    | Enzyme enhancer  |
| Ascorbic Acid         | Keeps enzymes fighting                                 |
| Potassium Sorbate*    | Antibacterial agent                                    |
| Sodium Lauryl Sulfate | Helps enzyme mixture<br>penetrate clogs                |
| Disodium Edta         | Keeps minerals from damaging<br>enzymes in Clog Zapper |

All ingredients are food grade.

\*G.R.A.S. Generally Recognized As Safe by the Food  
and Drug Administration

CLOG ZAPPER IS PATENTED UNDER U.S. PATENT NO. 5,424,299  
©CORPAK MedSystems 2009 5839H 06-11

## Nutrition Facts

|                          |               |
|--------------------------|---------------|
| Serving Size (1.5 g)     |               |
| Servings Per Container 1 |               |
| Amount Per Serving       |               |
| Calories                 | 0             |
|                          | % Daily Value |
| Total Fat 0 g            | 0%            |
| Sodium 118 mg            | 5%            |
| Potassium 65 mg          | 2%            |
| Total Carbohydrate 0 g   | 0%            |
| Protein 0 g              | 0%            |
| Vitamin C                | 2%            |

Not a significant source of calories from fat, saturated  
fat, cholesterol, dietary fiber, sugars, vitamin A,  
calcium and iron.

Percent Daily Values are based on a 2,000 calorie diet.

## Kit Contents

- 1 ea. 10 mL Oral Syringe with  
Declogging Powder
- 1 ea. 10 mL Oral Syringe
- 1 ea. 12" Applicator
- 2 ea. 4" x 4" Gauze Pads

## Ordering Information

REORDER #: 20-0002 (10 Kits/Box)

**CORPAK®**  
MEDSYSTEMS

1001 Asbury Drive • Buffalo Grove, IL 60089  
1-800-323-6305 • 1-847-403-3400  
Fax: 1-847-541-9526  
[www.corpak.com](http://www.corpak.com)

\*Clinical trial results are available  
upon request.