



INSTALLING/REMOVING THE BATTERY

Hold the battery vertically with the connection to the bottom, and install it into the back of the device as shown below.

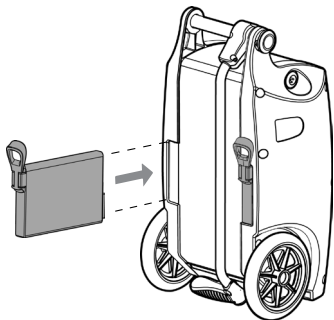

INSTALL

SLIDE INTO BACK OF DEVICE;
BATTERY WILL DROP DOWN SLIGHTLY
AND CLICK INTO POSITION.

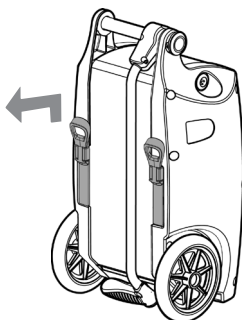

REMOVE

LIFT BATTERY SLIGHTLY UP; THEN
SLIDE STRAIGHT OUT.

TYPICAL BATTERY OPERATION TIMES

(2 Batteries @ 20 Breaths/Minute)

Single battery operation times are approximately half of
2 battery operation times.

FLOW RATE SETTING	CONTINUOUS FLOW	PULSE FLOW
0.5	5.75 hours	6.25 hours
1.0	5.25 hours	6.00 hours
1.5	3.50 hours	6.00 hours
2.0	2.50 hours	5.75 hours
2.5	2.00 hours	4.75 hours
3.0	1.50 hours	4.00 hours
3.5	N/A	3.50 hours
4.0		3.00 hours
4.5		2.50 hours
5.0		2.25 hours
5.5		2.00 hours
6.0		1.75 hours

OPERATING TIME WILL DEGRADE WITH BATTERY USE AND AGE.