


TROUBLESHOOTING GUIDE

ALERT/ALARM CONDITION	DEVICE STATUS	SOLUTION
LOW BATTERY ALERT	When remaining battery operation time reaches approximately 6 minutes, the screen will turn yellow and "ALERT LOW BATTERY" will be displayed in the System Status window. The device will beep every 30 seconds and then go into Low Battery Alarm mode.	Plug into an external power source. OR Replace depleted battery (batteries) with a charged battery.
LOW PURITY ALERT	If Oxygen Purity drops below 85% the screen will turn yellow and "ALERT LOW PURITY" will be displayed in the System Status window. The device will beep once every 30 seconds until the condition is corrected.	Clean or replace air inlet filter. Contact your authorized dealer.
NO BREATH DETECTED	If no breath is detected for 45 seconds the screen will turn yellow and "ALERT NO BREATH" will be displayed in the System Status Window. The device will beep once. If no breath is detected for an additional 15 seconds, the device will default to the previous Continuous Flow Setting selected.	Ensure cannula tubing does not exceed 7 feet (2.1m). Note: No Breath Detected is used when in Pulse Flow Mode ONLY.
LOW BATTERY ALARM	When the battery is depleted, the screen will turn red and "LOW BATTERY" will be displayed in the System Status window. The device will beep for 15 seconds before shutting down.	Plug into an external power source. Replace depleted battery (batteries) with a charged battery.
LOW PURITY ALARM	If Oxygen Purity drops below 72% the screen will turn red and "LOW PURITY" will be displayed in the System Status window. The device will beep for 15 seconds before shutting down.	Repair or replace tubing or humidifier. Clean or replace air inlet filter. Contact your authorized dealer.
NO EXTERNAL POWER ALARM	If no external power source is detected, the device will beep once before shutting down.	Ensure all charging connections are intact. In the case of an extended power outage, utilize an alternate, non-powered, source of oxygen.
NO FLOW ALARM	If no oxygen flow is detected, the screen will turn red and "NO FLOW" will be displayed in the System Status window. The device will beep for 15 seconds before shutting down.	Check cannula connection. Repair or replace tubing or humidifier. Clean or replace air inlet filter. Move device to ensure adequate air flow. Contact your authorized dealer.
OVER TEMPERATURE ALARM	If the device is too hot to operate safely, the screen will turn red and "OVER TEMPERATURE" will be displayed in the System Status window. The device will beep for 15 seconds before shutting down.	Allow the device to reach operating temperature. Utilize an alternate source of oxygen if necessary.
SERVICE REQUIRED ALARM	If service is required for the device to operate safely, the screen will turn red and "SERVICE REQUIRED" will be displayed in the System Status window. The device will beep for 15 seconds before shutting down.	Contact your authorized dealer and utilize an alternate supply of oxygen.
UNAUTHORIZED BATTERY ALARM	If the device detects that an unauthorized battery has been installed, the screen will turn red and "UNAUTHORIZED BATTERY" will be displayed in the System Status window. The device will shut down.	Remove unauthorized battery and replace with an authorized O2 Concepts battery.


ALERT AND ALARM SCREENS

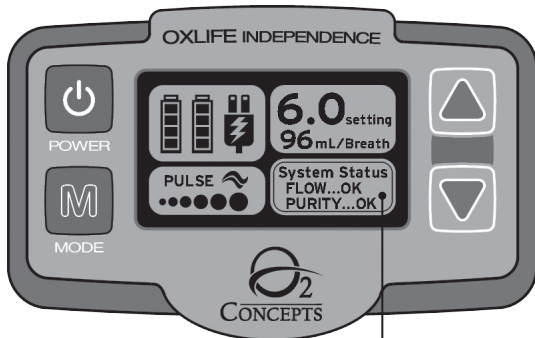

ALERTS AND ALARMS WILL BE INDICATED IN THE LOWER RIGHT SECTION OF THE LCD DISPLAY.

ALARM SCREENS

(RED SCREEN)

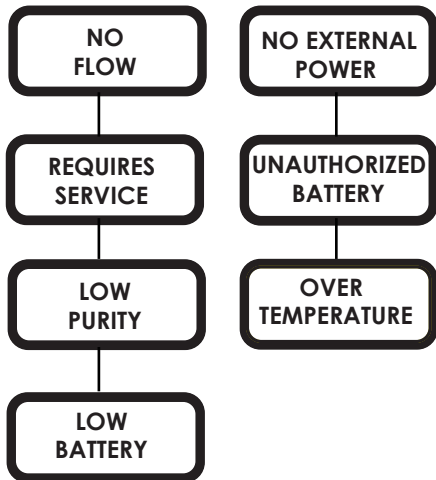

ALERT SCREENS

(YELLOW SCREEN)

