

## THANK YOU FOR USING THE VERTALOC® FLEX FIT BACK BRACE

Your Lumbar-Sacral Orthosis (LSO) has been designed to maximize support of your lower back while providing you with the comfort you require.

### YOU HAVE RECEIVED:

- ONE (1), ONE-PIECE LUMBAR-SACRAL ORTHOSIS BRACE WITH ATTACHED CLOSURE SYSTEM.
- ONE (1) ABDOMINAL SUPPORT PANEL (REMOVABLE).
- ONE (1) BACK INSERT (REMOVABLE).

## FIT YOUR BRACE IN 3 EASY STEPS



1. OPEN ALL STRAPS. CENTER THE POSTERIOR PANEL OF THE BRACE IN THE MIDDLE OF LOWER BACK.



2. FASTEN THE RIGHT END OF THE ABDOMINAL SUPPORT PANEL ON THE TOP OF THE LEFT PANEL. TIGHTEN TO ALLOW OPTIMAL SUPPORT.



3. PULL THE COMPRESSION STRAPS TO ACHIEVE OPTIMAL SUPPORT. FASTEN THEM TO THE ABDOMINAL PANEL SURFACE.

**CARING FOR YOUR FLEX FIT LSO:** *Hand wash in warm water using mild soap. Rinse thoroughly. Lay flat to dry. Do not machine wash or dry. Do not expose to direct heat sources such as heaters, radiators, direct sunlight, etc.*

### VERTALOC® FLEX FIT WARRANTY:

- ✓ YOUR VERTALOC® FLEX FIT IS WARRANTED TO BE FREE OF MATERIAL OR WORKMANSHIP DEFECTS.
- ✓ ANY DEFECTIVE COMPONENTS OF YOUR VERTALOC® FLEX FIT MAY BE REPLACED OR CREDITED AT THE DISCRETION OF VERTALOC, INC.
- ✓ THIS PRODUCT IS WARRANTED FOR ONE YEAR FROM THE DATE OF PURCHASE.



**QUESTIONS OR COMMENTS? WE ARE ALWAYS HERE TO HELP YOU.  
CALL US TOLL FREE 855-VERTALOC (855-837-8256)**