


Sound Amplifier

- High sensitivity microphone amplifies environmental sounds (eg. amplifying distant staff members or visitor voices, amplifying telephone, radio, and TV sounds, attending activities with a distant speaker, etc.).
- Adjustable volume control.
- Soft ear buds.
- Fits in pocket or clips to your belt.
- Battery included.

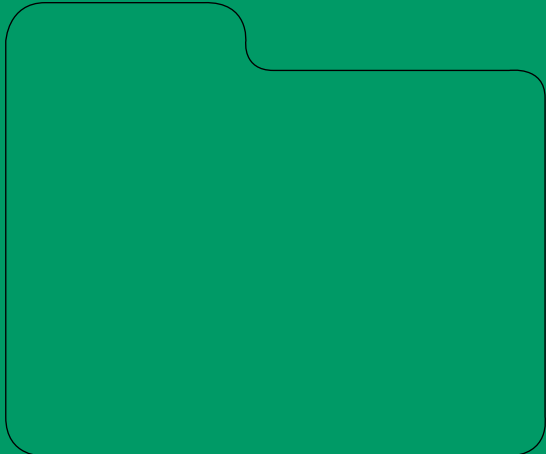

POSEY[®]







8274

Sound Amplifier


To Use:

1. Plug the earphones into the Sound Amplifier (fig. 1).
2. Use the volume control dial to turn the unit on and adjust the volume (fig. 2).

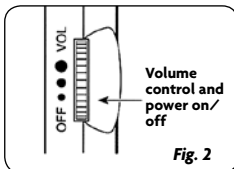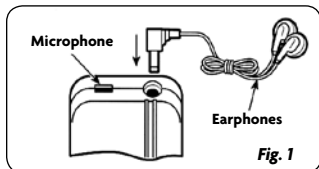

To Change Battery:

1. Press down on arrow and slide the belt clip off (fig. 3).
2. With belt clip removed, press down on arrow and slide the battery compartment door off (fig. 3).
3. Insert a new battery (fig. 3) (LR44, 1.5V Alkaline).
4. Reattach the battery door and belt clip (fig. 3).

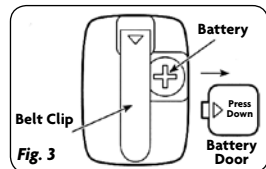

ALWAYS verify Sound Amplifier is working properly and volume is not too loud to cause discomfort or injury when placing the Sound Amplifier on another person.

Cleaning:

Wipe clean with disinfectant liquids that DO NOT contain Phenol. DO NOT submerge in liquid or sterilize with heat.

Storage Instructions:

- This device is designed for use in normal indoor environments.
- This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

Disposal:

WARNING The battery in this product contains mercury and must be recycled or disposed of according to local, state and federal laws. Dispose of per facility policy for BIOHAZARDOUS material. Be sure to follow all laws that apply.

Warranty: This product is warranted for 30 days.

Distributed By Posey

5635 Peck Road • Arcadia, CA 91006-0020 USA • Tel: 1.800.447.6739 or 1.626.443.3143

Fax: 1.800.767.3933 or 1.626.443.5014 • www.posey.com