
RESPIRONICS

SideStream[®] Plus

Reusable High Efficiency Nebulizer

Breath enhanced • Fast, efficient treatments • Easy to use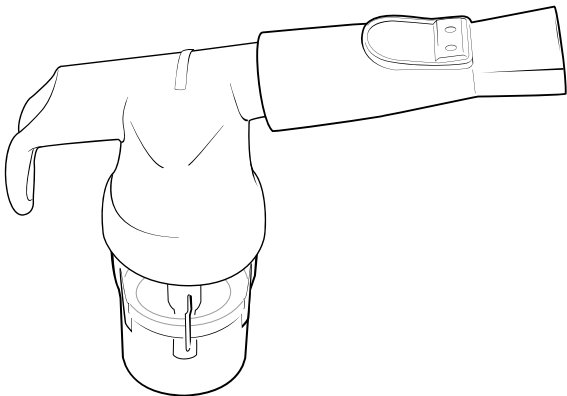

LATEX


Rx ONLY

NDC 08373-0870-00


Reorder Number: **HS870**

SINGLE PATIENT USE

Manufactured in China by Respirationics New Jersey, Inc.,
Parsippany, NJ 07054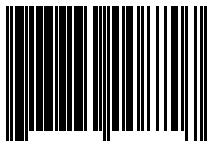

3 83730-00870 7

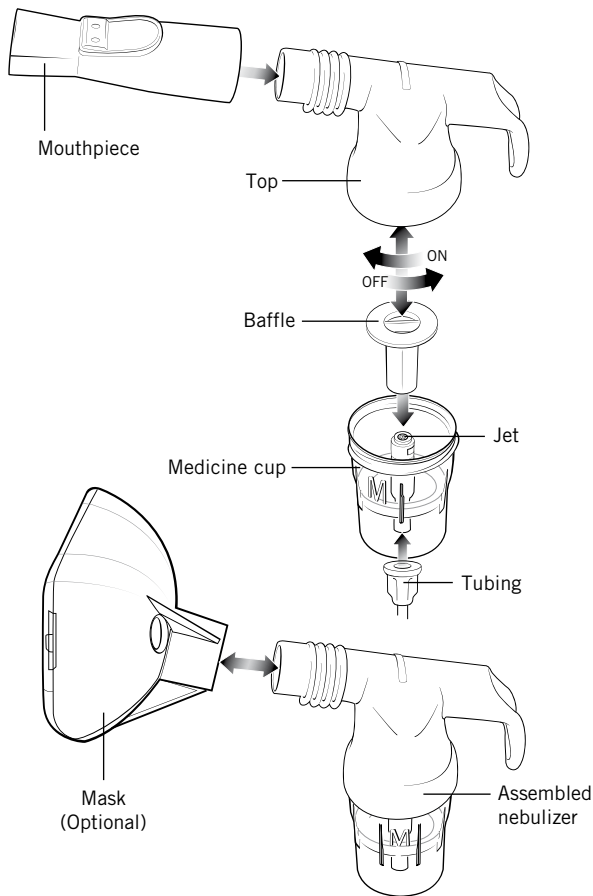SideStream® Plus

Reusable High Efficiency Nebulizer

The SideStream Plus is a handheld nebulizer designed to aerosolize medication approved for nebulization.

INSTRUCTIONS (Please keep these instructions)

GETTING READY

Please read these instructions carefully before using this product. This product should only be used under the supervision of a physician or licensed healthcare professional.


- Clean nebulizer prior to first use (see 'After Each Use' page 4, steps 3–5).
- Wash your hands before using the nebulizer.

INSTRUCTIONS FOR USE

- 1 Connect one end of the 'Tubing' to the compressor air outlet.
- 2 Push the other end of the 'Tubing' onto the bottom of the 'Medicine cup'.
- 3 Unscrew the 'Top' on the SideStream Plus and pour the medication into the 'Medicine cup'. If you prefer, you may also remove the 'Baffle'.

Top of 'M' is max. fill line – 8 ml.




- 4 Replace the 'Baffle', if you have removed it.
- 5 Screw the 'Top' back onto the 'Medicine cup'.
- 6 Connect the 'Mouthpiece' or 'Mask'.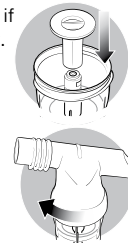

Your reusable SideStream Plus nebulizer is now ready for use.

TAKING A TREATMENT.

- To take the treatment, sit in an upright position and relax.
- Switch on the compressor.
- Make sure mist is visible, coming from the 'Mouthpiece/Mask'.
- Take the 'Mouthpiece' between your teeth, surround it with your lips and breathe in and out slowly through your mouth or place the 'Mask' in position over nose and mouth and breathe slowly in and out through your mouth.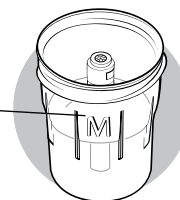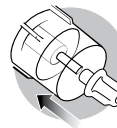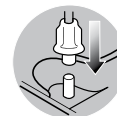

When the SideStream Plus begins to work intermittently and 'sputters', then the treatment is finished. There is no need to nebulize every drop of your medication, there will always be a small amount left in the 'Medicine cup'.

CLEANING

AFTER EACH USE

- 1 Disconnect the SideStream Plus from the 'Tubing'. Unscrew the 'Top' from the 'Medicine cup', rinse out the nebulizer and refill with clean water. Reassemble 'Top' and 'Medicine cup', reconnect the 'Tubing' and run SideStream Plus again for a few seconds to flush out any drug trapped in the nebulizer jets.
- 2 Then take apart the nebulizer.
- 3 Disconnect 'Tubing' from the 'Medicine cup'.
- 4 Wash all items (except 'Tubing') in hot soapy water.
- 5 Rinse all parts in clean water.
- 6 Shake off excess water and allow to air dry. Do not towel dry.
- 7 Reassemble and store the SideStream Plus in a clean place when not in use.
- 8 The SideStream Plus may be washed once a week contained within a basket on the top rack of your dishwasher.

ONCE A WEEK

- 1 Disassemble the nebulizer.
- 2 Boil the SideStream Plus parts (except 'Tubing') in water with 2–3 drops of dish washing liquid for 10 minutes.
- 3 Rinse all parts in clean water for at least 2 minutes.

- 4 Shake out excess water and allow to air dry.
- 5 Reassemble and store SideStream Plus in a clean place when not in use.

⚠ WARNING!

CLOGGED JETS OR IMPROPER ASSEMBLY MAY PREVENT PROPER DELIVERY OF MEDICATION.

CAUTION

To reduce the risk of infection, clean and disinfect the nebulizer between treatments.

Never use a cleaning brush or put sharp objects into the jet holes as this will damage the nebulizer.


Do not try to remove the 'Jet' as this is permanently attached.

IMPORTANT

Aerosol output rate (ml/min) is dependent on patient inhalation flow. SideStream Plus has been designed to deliver general purpose inhaled aerosol medications solutions and suspensions. SideStream Plus is designed to be used with a standard flow compressor (3.5–10l/min).

Not suitable for use in ventilator circuits.

It is recommended that you replace your SideStream Plus every six months.

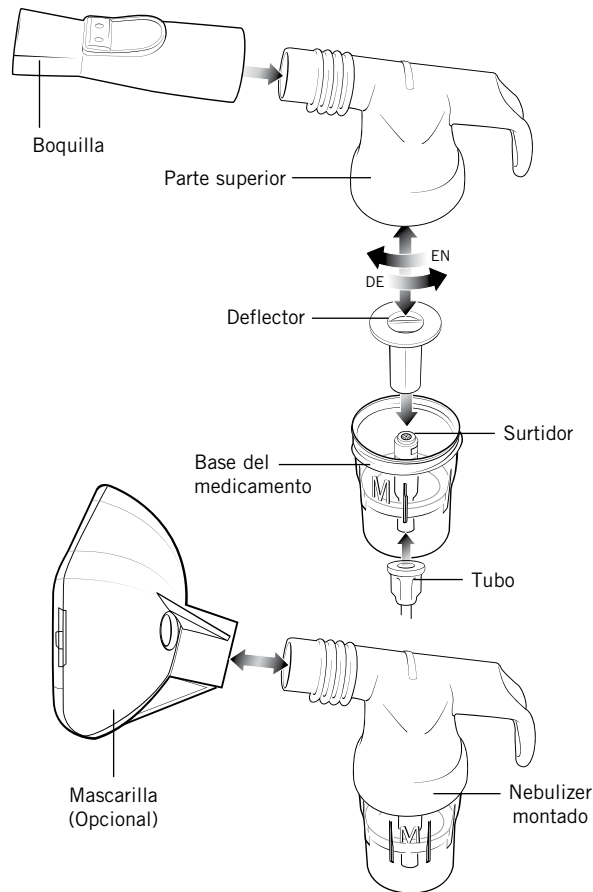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affiliates. All rights reserved.
SideStream is a registered trademark
Respironics New Jersey, Inc.
© 2008, Respironics, Inc. and its
of Respironics, Inc. and its affiliates.