

3M™ Tegaderm™ Ag Mesh Dressing with Silver

Tegaderm™ Ag Mesh Dressing



Delivering the Power of Silver

Easy. Versatile. Affordable.



3M™ Tegaderm™ Ag Mesh Dressing with Silver provides a fast-acting, long-lasting antimicrobial barrier in an affordable dressing

- Significant antimicrobial activity within the first few minutes of use
- Sustained microbial load reduction proven out to 14 days*
- Familiar gauze material allows for clinician flexibility

[Click to See Prices and Product Options](#)


**In-vitro* data on file.



The proven effectiveness of antimicrobial cationic silver in an affordable gauze dressing

Dressing Construction

Easy. Versatile. Affordable.

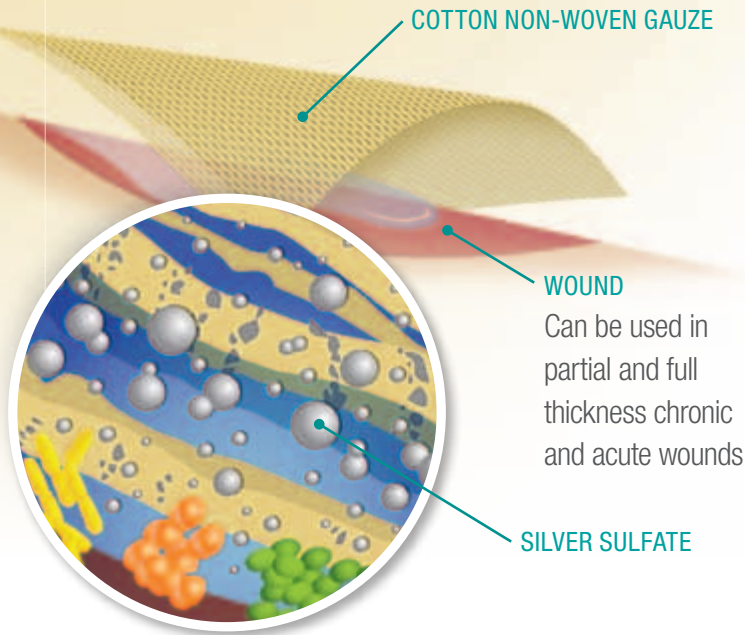

COTTON NON-WOVEN GAUZE

WOUND

Can be used in partial and full thickness chronic and acute wounds

SILVER SULFATE

SILVER SULFATE

is one of the most soluble silver salts

- Dissolves in the presence of moisture releasing antimicrobial cationic silver

COTTON NON-WOVEN GAUZE


- Material that clinicians know and trust
- Can be cut to desired shape and size
- Can be fluffed or packed into tunnels, sinus tracts and areas of undermining
- Soft and conformable

MOISTURE OPTIONS

Can be moistened with wound exudate, sterile normal saline, sterile water or hydrogel

- Sterile normal saline does not cause a significant change in pH, unlike some competitive products


Clinically proven

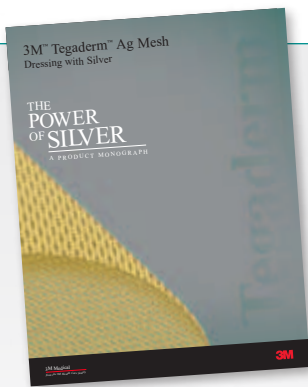


SUPPORTED by both *in vitro* and *in vivo* studies.

To download your copy, visit

<http://swcpromo.3M.com/powerofsilver>

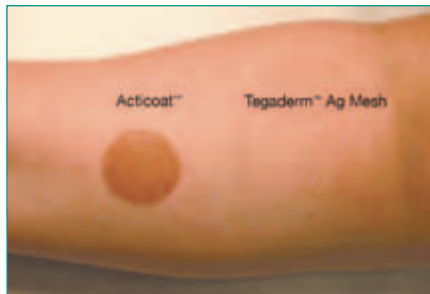


NO TO MINIMAL SKIN STAINING

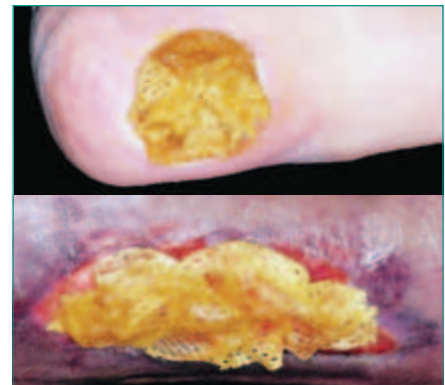


VERSATILITY

Dressing Application

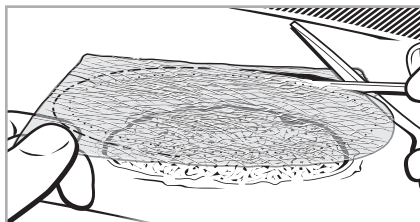


Step 1

Evaluate the wound and select the dressing size that will fit into the wound margins without overlap onto the surrounding skin. If necessary, trim or fold the dressing to fit the wound site.

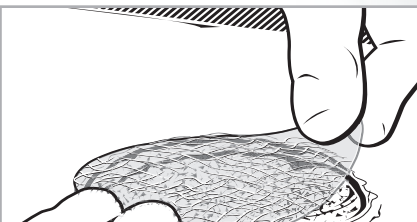

Step 2

Apply dressing to the wound bed. For dry to minimally draining wounds, dressing may be moistened with sterile normal saline or sterile water to provide a moist wound environment. For additional absorbency 3M™ Alginate Dressings may be used as wound fillers over the Tegaderm™ Ag Mesh dressing.

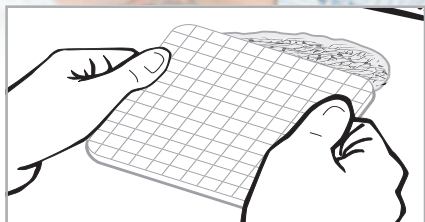

Step 3

Apply a suitable cover dressing that maintains a moist wound healing environment to minimize dressing adhering to the wound bed. Apply 3M™ Medipore™ +Pad Soft Cloth Adhesive Wound Dressing or 3M™ Tegaderm™ +Pad Film Dressing with Non-Adherent Pad as a cover dressing. For greater absorbency, apply 3M™ Tegaderm™ Foam Dressing (nonadhesive) or 3M™ Tegaderm™ High Performance Foam Adhesive Dressing.

Removing the Dressing





- Change the dressing as needed. Frequency of changing will depend on factors such as the type of wound and volume of drainage.
- At the time of dressing change, if the dressing is adhered to the wound surface, saturate with sterile normal saline or sterile water, allow the dressing to soften, and gently remove.
- Avoid forceful removal of the dressing to minimize disruption of the wound.

Indications for Use

- Donor sites
- Ulcers (Pressure, venous, arterial, and neuropathic)
- Open surgical wounds
- Trauma wounds
- Superficial partial thickness burns (second degree)
- Abrasions

3M™ Tegaderm™ Ag Mesh with Silver Dressing

Ordering Information:

	Catalog Number	NDC/NHRIC Number	Dressing Size	Dressings/ Box	Boxes/ Case	HCPCS Code
	90500	8333-9050-00	2 in. x 2 in. 5 cm x 5 cm	5	4	A6402
	90501	8333-9050-10	4 in. x 5 in. 10 cm x 12.7 cm	5	4	A6403
	90502	8333-9050-20	4 in. x 8 in. 10 cm x 20 cm	5	4	A6403
	90503	8333-9050-30	8 in. x 8 in. 20 cm x 20 cm	3	4	A6404

Disclaimer:

HCPCS codes have been provided to assist you in the preparation of insurance claims. Please note, however, that the reimbursement information provided by 3M Health Care and its representatives is intended to provide general information relevant to coverage and coding for 3M products. Insurers' reimbursement policies can vary and the use of the codes discussed here does not guarantee that an insurer will cover or pay at any particular level. Health care providers should exercise independent clinical judgment in choosing the codes which most accurately describe the products provided.

3M™ Tegaderm™ Dressings **SIMPLE. DEPENDABLE. TRUSTED.**

To learn more about 3M™ Tegaderm™ Ag Mesh Dressing with Silver, visit us at www.3M.com/TegadermAgMesh. For more information about the Skin & Wound Care family of products, visit us at www.3M.com/SkinWoundCare, contact your 3M Skin & Wound Care representative or call the 3M Health Care Customer Helpline at 1-800-228-3957. Outside of the United States, contact the local 3M subsidiary.



Skin & Wound Care Division
3M Health Care
3M Center, Building 275-4W-02
St. Paul, MN 55144-1000
USA
1-800-228-3957
www.3M.com/healthcare

3M Canada
P.O. Box 5757
London, Ontario N6A 4T1
Canada
1-800-364-3577
www.3M.com/ca/healthcare

3M, Medipore and Tegaderm are trademarks of 3M. Acticoat is a trademark of Smith & Nephew.

Please recycle. Printed in U.S.A.
Used under license in Canada.
© 3M 2006, 2011.
All rights reserved.
70-2009-7138-3