

Sacral Application Guide

- 1 Cleanse the wound/skin in accordance with local procedures. The peri-wound skin should be clean and dry to assist adhesion.
- 2 Select the appropriate size. For wounds, the dressing pad should overlap the wound by 3/4 inch. **Do not cut dressing to size.**
- 3 Remove center lining and fold the dressing in half.
- 4 Apply dressing to skin/wound. Gently press down from the center out to ensure good adhesion.
- 5 Peel off the remaining lining and smooth out the silicone border.
- 6 Check the dressing at regular intervals and change according to the clinical condition of the wound, when saturated or per hospital protocol.

STAYS IN PLACE: Unique design with large border ensures adhesion – even in the natal cleft



EASILY REPOSITIONED AND REPLACED: Unique silicone makes it easy to lift and look at intact skin and replace the dressing if it falls off the patient



FLEXIBLE TO PATIENT NEEDS:

Versatility in the design allows for 180° application to protect a wide area of at-risk skin



LARGE SACRAL



SMALL SACRAL

