

Instructions for precise measuring of Juzo® compression stump shrinkers

Please note:

To ensure a correct pressure distribution, please measure the first circumference 5 cm proximal to the end of the stump. Record this in the relevant place on the measurement sheet. As per the schematic, you need to provide **all** measurements above this according to the style you have selected.

Please ensure that you complete all the necessary measurements when a hip attachment or body part is included.

The length measurements are to be taken from the end of the stump.

Measure the lengths in a straight line, do not put the measuring tape along the stump.


If necessary, use a measuring board to obtain accurate length measurements.

In the case of **compression stump shrinker pantyhose**, the following options are available:

- One stocking leg for stump treatment: Please send both measuring sheets “stump shrinker” and “stockings/pantyhose flat knitted” and include a suitable mark.
- Both stocking legs for stump treatment. Determine all the measurements up to “T”.
- One stocking leg for stump treatment, the second with a leg extension. In this case please specify also the circumference and length measurements for “F” and “G”.

Compression stump shrinker with body part (KT Juzo® Expert body part – height 23 cm)

Available sizes	1	2	3	4	5
To suit waist	90	100	110	120	135

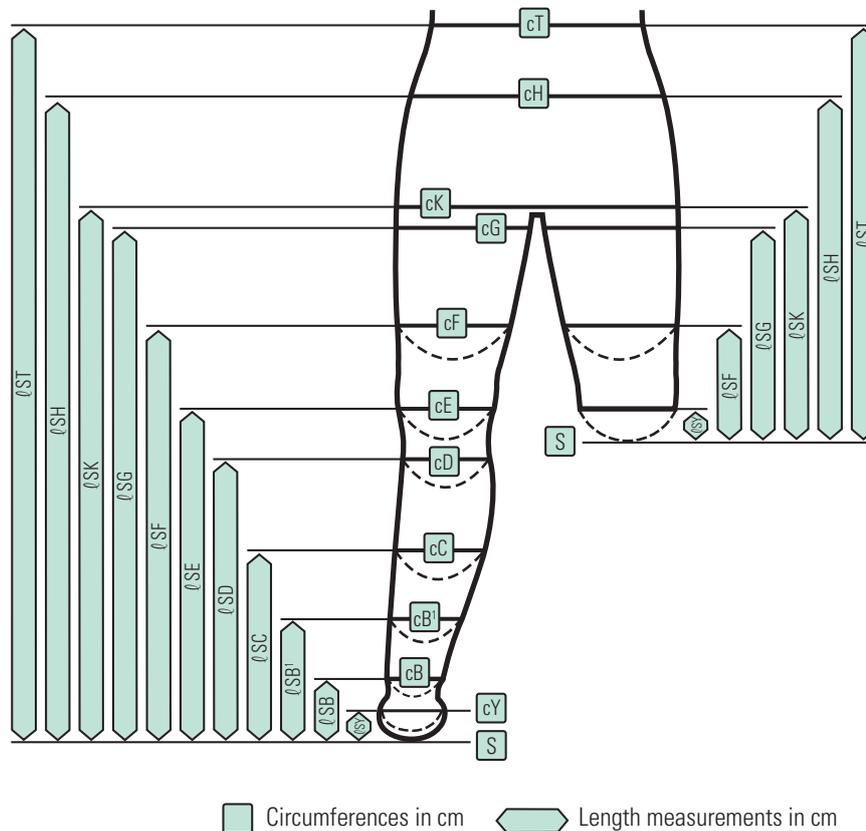

For the selection of the appropriate garment, we recommend the brochure Juzo® Expert Special-purpose medical garments.