

The Unistik® 2 Range

The Unistik® 2 range of single-use capillary blood sampling devices has 4 models providing different levels of blood flow for various applications.

CAPILLARY BLOOD SAMPLING

Unistik®2 Comfort

Low Flow
28G Depth: 1.8mm



Box of 50 AT 0740
Box of 100 AT 0742
Box of 200 AT 0747

Unistik®2 Normal

Medium Flow
21G Depth: 2.4mm



Box of 50 AT 0700
Box of 100 AT 0702
Box of 200 AT 0704

Unistik®2 Extra

Medium-High Flow
Tri-Beveled
21G Depth: 3.0mm



Box of 50 AT 0710
Box of 100 AT 0712
Box of 200 AT 0714

Unistik®2 Super

High Flow
Bi-Beveled
21G Depth: 3.0mm



Box of 100 AT 0752



OWEN MUMFORD
Making a World of Difference

NORTH AMERICA

Owen Mumford Inc.
1755 West Oak Commons Court
Marietta, Georgia 30062, USA
T : 800-421-6936 F : 770-977-2866
E : info@owenmumfordinc.com

HEAD OFFICE

Owen Mumford Ltd
Brook Hill, Woodstock
Oxfordshire OX20 1TU, United Kingdom
T : 01993 812021 F : 01993 813466
E : info@owenmumford.co.uk

UK / IRELAND SALES & MARKETING

T : 01608 645555 F : 01608 646546
E : info@owenmumford.co.uk

FRANCE

T : 02 32 51 88 70 F : 02 32 21 96 95
E : contact@owenmumford-fr.com

GERMANY

T : 06026/97750 F : 06026/977529
E : info@owen-mumford.de

www.owenmumfordinc.com

OUR ACCREDITATIONS INCLUDE:

- ISO13485 Certification
- Medical Device Directive 93/42/EEC
- CMDCAS 13485 (Canadian Health Authority)
- Environmental Management Systems ISO14001



AT/0700/40/0012/01



Unistik® 2

Single use Safety Lancets



For more information or samples, call 1-800-421-6936
or email info@owenmumfordinc.com

Unistik® 2 Single use Safety Lancets

engineered for safety and comfort

Maximum comfort

Owen Mumford's unique, patented design features a side release button and preset lancet speed to ensure ease of use for healthcare professionals and maximum comfort for patients.



Safe

Unistik® 2 is a single use device designed to ensure safety for healthcare professionals and patients.

The Unistik® 2 needle point is hidden before and after use and automatically retracts to avoid needlestick injuries and cross infection.

Easy to use

The Unistik® 2 range is color coded for easy recognition. The device is preset - so simply push in the lancet cap, twist off and Unistik® 2 is ready to sample.

The Unistik® 2 Range

Unistik® 2 Comfort

Low Flow
28G Depth: 1.8mm



Suggested Tests

Low flow blood glucose monitoring, HbA1c.

Box of 50	AT 0740
Box of 100	AT 0742
Box of 200	AT 0747

Unistik® 2 Normal

Medium Flow
21G Depth: 2.4mm



Suggested Tests

Medium flow blood glucose monitoring, Cholesterol, HbA1c.

Box of 50	AT 0700
Box of 100	AT 0702
Box of 200	AT 0704

Unistik® 2 Extra

Medium-High Flow
Tri-Beveled
21G Depth: 3.0mm



Suggested Tests

Cholesterol, Haemoglobin (Hgb), Diagnosis kit tests such as cardiac enzymes.

Box of 50	AT 0710
Box of 100	AT 0712
Box of 200	AT 0714

Unistik® 2 Super

High Flow
Bi-Beveled
21G Depth: 3.0mm




Suggested Tests

Blood gases & electrolytes, Chemistry test profiles, Diagnosis kit tests such as cardiac enzymes, H. pylori.

Box of 100	AT 0752
------------	---------


How to use Unistik® 2

1. Push

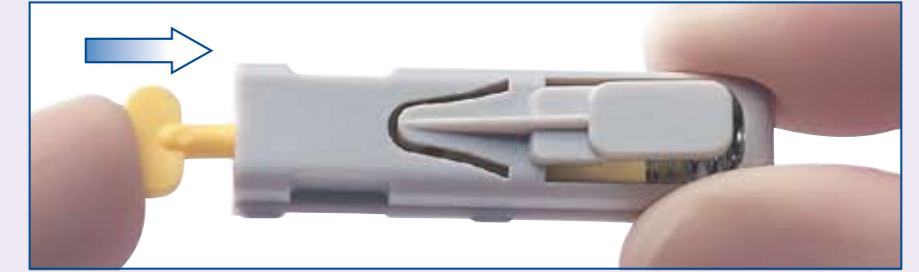

Push the lancet cap into the body of the device until it clicks.

2. Twist

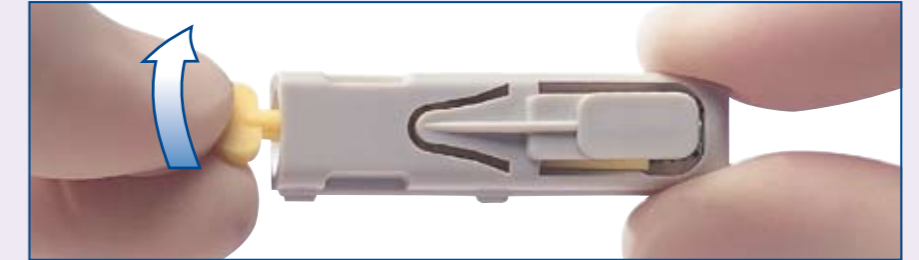

Twist off the lancet cap until you feel it separate from the device. Don't pull, just twist. Dispose of the cap in a suitable container.

3. Sample



Choose a site on the finger as indicated by the shaded areas on picture A. Press the device platform firmly against the chosen site and press the release button.

If required, massage the sample site, taking care not to squeeze too hard at the site.

The single-use needle retracts immediately after sampling, leaving the device safe for immediate disposal. Use a suitable sharps container.

What healthcare professionals thought of Unistik® 2

"Unistik® 2 makes fingersticks easy and comfortable for my patients."

Point of Care Coordinator
Madison, Wisconsin

