



Urocure Products, Inc. • 2735 Melbourne Avenue • Pomona, California 91767-1931 • U. S. A.  
T: (909) 621-6013 • F: (909) 621-4436 • E: [sales@urocare.com](mailto:sales@urocare.com) • I: [www.urocare.com](http://www.urocare.com)

## Uro-Bond® Thinner

Thinner for Brush-on Silicone Adhesive



### DESCRIPTION

Uro-Bond® Thinner (#501003) thinner for brush-on silicone adhesive has been specifically formulated for thinning medical bonding silicone adhesives. It is suitable for reducing the consistency of Uro-Bond® (#500003, #5000015, #500403 and #5004015) Brush-on Silicone Adhesive. Uro-Bond® Thinner's features include, but are not limited to:

- Clean-up by soap and water.
- Completely stable product.
- Does not promote hazardous polymerization.

### APPLICATIONS

The use of Uro-Bond® Thinner may be deemed necessary where Uro-Bond® Brush-on Silicone Adhesive has thickened. Urocure, however, does not recommend the use of Uro-Bond® Thinner as an adhesive remover. Suggested applications are:

- Use as a dilutant for Uro-Bond® II Brush-on Silicone Adhesive.
- May also be used as a degreasing agent or a solvent.

**NOTE:** Since the actual use of Uro-Bond® Thinner or products otherwise incorporating Uro-Bond® Thinner by others is beyond the control of Urocure Products, Inc., it is the user's responsibility to independently determine the safety, toxicity, suitability and acceptability of the product's performance in a specific application for his/her own use.

### STORAGE AND SHELF LIFE

When stored in its original unopened container at or below 72° F (22° C), Uro-Bond® Thinner for brush-on silicone adhesive has a shelf life of one (1) year from the date of shipment.

*PLEASE NOTE: Due to the highly evaporative chemical composition of Uro-Bond® Thinner. Do not leave jar open for prolonged periods of time. Close cap securely immediately after application.*

**KEEP THIS AND ALL CHEMICALS  
OUT OF REACH OF CHILDREN!**

### INSTRUCTIONS FOR USE

Uro-Bond® Thinner for brush-on silicone skin adhesive is normally used as supplied; it is added to Uro-Bond® brush-on silicone adhesive with the use of a teaspoon or an eye dropper. To obtain

the maximum results of Uro-Bond® Thinner, gradually add small quantities (1 or 2cc) of Uro-Bond® Thinner to thickened Uro-Bond® brush-on silicone adhesive until the desired consistency is obtained.

### REMOVAL & DISPOSAL PROCEDURES

To remove Uro-Bond® Thinner for brush-on silicone skin adhesive from the body surface, only the use of soap and water are necessary. Urocure recommends that you wash before eating, drinking, smoking or using lavatory facilities. To remove Uro-Bond Thinner for brush-on silicone adhesive from surfaces other than skin, use soap and water for clean-up or reusing. For clothing launder contaminated clothing before reuse. **DO NOT REUSE CONTAMINATED CLOTHING BEFORE LAUNDERING.** Spill: Should a spill occur, avoid breathing vapors/fumes and ventilate. Clean area by mop ping or with the use of absorbent material and place in closed container for disposal. Avoid contamination of ground and surface waters. **DO NOT FLUSH INTO SEWER!** The information and data contained herein is based on information we believe to be reliable and accurate; however, Urocure encourages you to thoroughly test any application and independently conclude the effectiveness and suitability of Uro-Bond® Thinner before use in a specific application.

### PACKAGING

Uro-Bond® Thinner for brush-on silicone adhesive is supplied in a 3 fl. oz. (88.7 ml.) glass jar and is secured with a tamper proof shrink-band seal. Uro-Bond® Thinner for brush on silicone adhesive is sold 1 bottle per box, 12 boxes per case.

### ORDERING INFORMATION

Please visit our website at: [www.urocare.com](http://www.urocare.com). Here you will find current product information, as well as the location of your nearest Urocure distributor. If you need assistance, please contact us via our website or call our toll-free number at: 1-800-423-4441. Our hours of operation are: Monday through Friday from 7:30AM to 4:00PM (PST).

The information contained herein is based on information we believe to be accurate and reliable; however, Urocure encourages you to thoroughly test any application and independently conclude the effectiveness and suitability of Uro-Bond® Thinner before use. Suggestion for use should not be taken as inducements to infringe upon any patent.



PLEASE RECYCLE  
Printed on  
Recycled Paper