
Airflo Plus

user instructions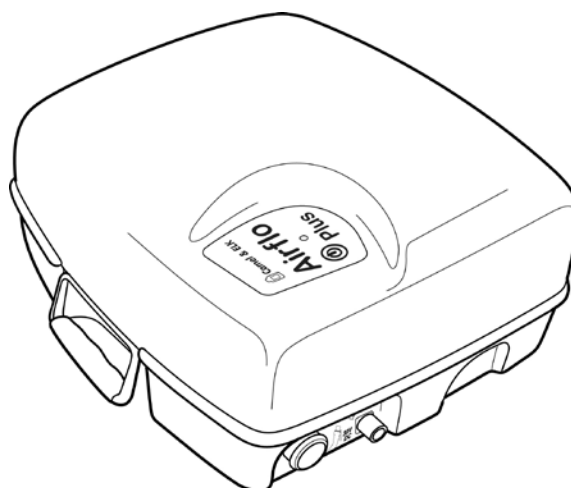

English

Deutsch

Français

Nederlands

Italiano

Español

Polski

Simple solutions for everyday independence

English

Intended Use

The Airflo Plus by Mangar International, is a low pressure air compressor, which has been designed to inflate the Mangar International 'Camel' and 'ELK' pneumatic lifting equipment through the Mangar 4-Way pneumatic hand control. The 4-Way hand control is supplied with the Camel and ELK.

Applied Parts (Accessories) for the Airflo Plus:

- Mangar ELK (Emergency lifting cushion)
- Mangar Camel (Emergency lifting cushion)

English	Pg 2
Deutsch	S10
Français	P 18
Nederlands	P 26
Italiano	Pag 34
Español	Pg 42

Contents


Standard Package Contents	Pg 3
Battery Status Indicator	Pg 3
Battery Charging	Pg 4
Operation	Pg 5
Cleaning & Storage	Pg 6
Maintenance	Pg 6
Trouble shooting guide	Pg 7
Specifications	Pg 7
Symbols used on Airflo Plus and Charger	Pg 8
Symbols used on packaging	Pg 9
Warranty	Pg 9

WARNINGS

- Be sure that you have read and understood these instructions and those provided with any other mangar international product, before operating the equipment.
- To avoid the risk of electric shock, NEVER recharge the battery in the bathroom, outdoors or in a damp environment.
- Always disconnect from the battery charger before cleaning.
- Use only the supplied battery charger.
- Do not operate the Airflo compressor in the presence of flammable anaesthetic mixtures with air, oxygen or nitrous oxide, as these mixtures can create a combustible environment leading to the potential for fire or even an explosion in the presence of a spark.
- Ensure that when in use, the Airflo Plus and its connected hoses are laid out in such a way as to avoid any risk of tripping or strangulation.
- Ensure that children and pets do not interfere with the safe operation of the Airflo and inflatable product.
- Keep the Airflo Plus and inflatable product away from sources of heat and any sharp surfaces.
- Disconnect the Airflo compressor from the inflatable product before transporting so as to avoid accidental inflation.
- The Airflo Plus should only be used for it's stated purpose and should not be modified in any way.

This product and associated packaging does not contain latex.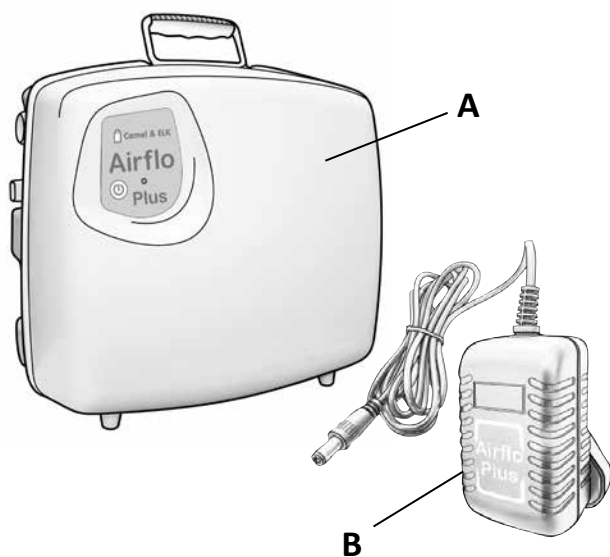

Standard Package Contents - fig. ①

- 1 x Airflo Plus compressor fitted with two 12v- 2.3Ah batteries (A)
- 1 x Battery charger (B)
- 1 x User instructions

1

Battery Status Indicator - fig. ②

Before use, momentarily press the 'Power' button (A) to check the battery charge state. If the battery is fully charged, the battery status indicator light (B) will show a steady Green light.

NOTE: If the battery pack has been removed and reinserted, the indicator light will show Steady Green for approximately 16 secs before showing the true battery state as below.

Steady Green = Fully charged.

Flashing Green = Recharge soon.


Steady Red = Recharge immediately.

Flashing Red = Over temperature cut-out.

Flashing Red/Green = Current overload cut-out.
(Leave for 3 seconds to self reset).

The Airflo Plus needs to be recharged when the indicator light is flashing Green.

Fully charge overnight before using the Airflo Plus for the first time and when new batteries have been fitted.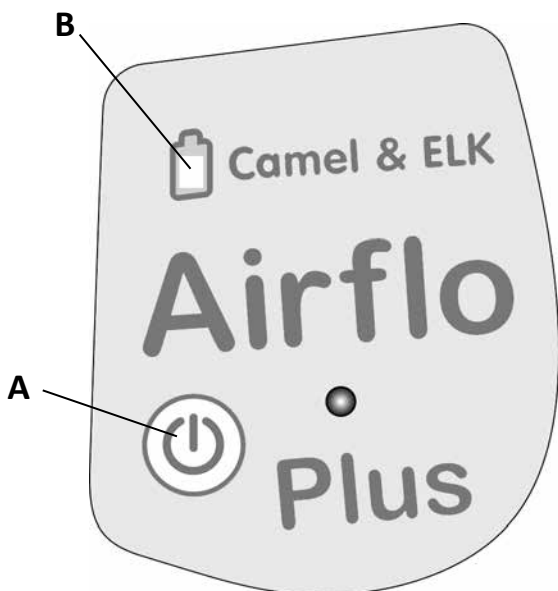

2

Battery Charging - fig. ③

The Airflo Plus is fitted with a quick release battery pack. The batteries can either be re-charged whilst in the Airflo Plus or easily removed and re-charged in the optional charging cradle.


- Check the battery charger casing and lead for signs of damage and if damaged DO NOT USE.
- Fit the appropriate mains plug adaptor to the battery charger (A).
- Plug the battery charger into a mains outlet socket and connect the charger output lead (B) to the Airflo Plus charging socket (C).
- The battery status indicator light (D) will flash green while the battery is charging and then turn to a steady (non flashing) light when the Airflo Plus is ready for use.

NOTE: The charger is not capable of providing sufficient power to charge the batteries while the compressor is running.


To maintain battery condition, It is recommended that the Airflo Plus is left continuously on charge when not in use. The batteries cannot be overcharged using this charger.

WARNING: DO NOT recharge the batteries in a bathroom or in a damp environment.

To disconnect the charger from the mains power supply, unplug the charger from the mains power socket.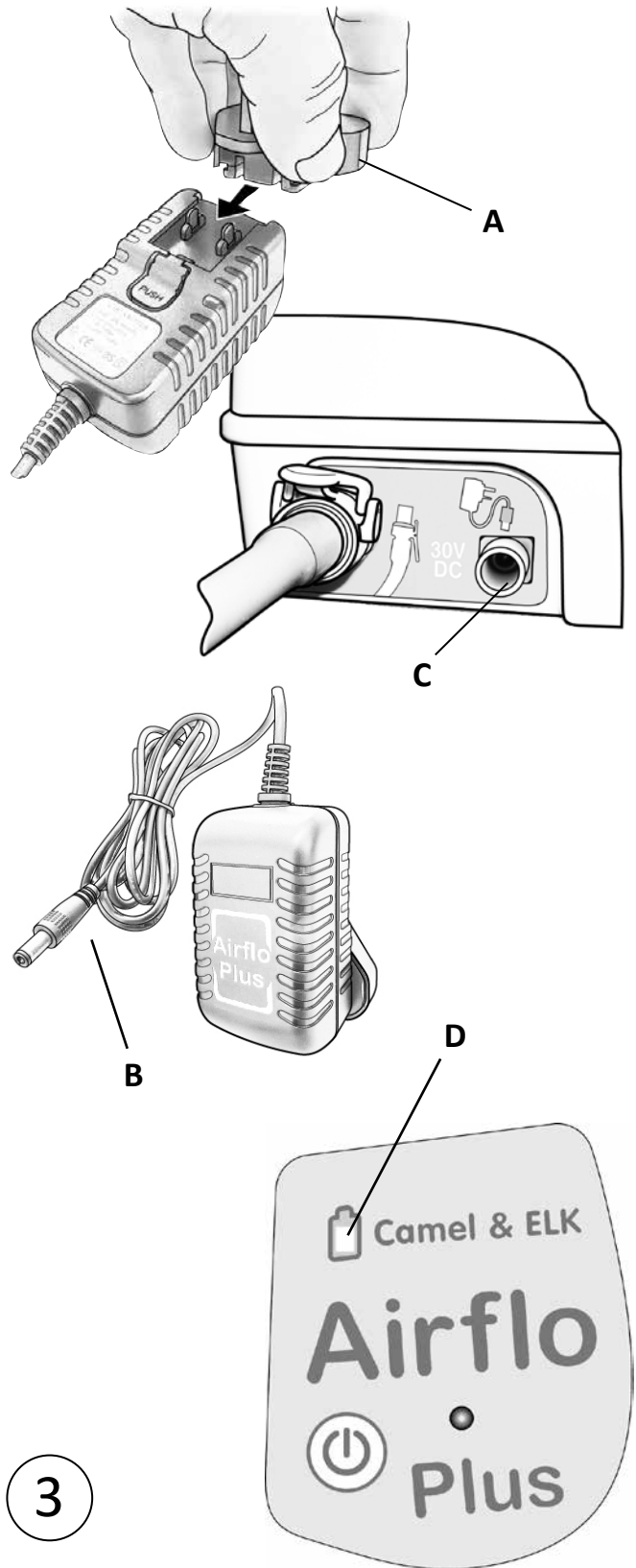


③

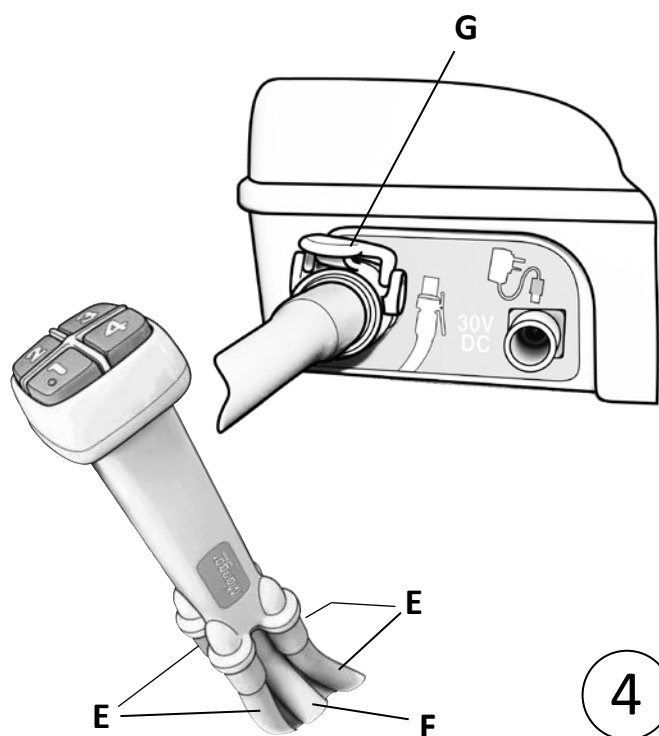Operation - figs. ④ ⑤

Disconnect the Airflo Plus from the battery charger and place it in a safe and convenient position.

- Connect the four hoses (**E**) from the 4-Way Hand Control to the corresponding numbered and coloured connectors on the Mangar International product.
- Connect the hose (**F**) from the Hand Control to the Airflo Plus air outlet socket (**G**). The connector will 'click' into place when firmly pushed onto the outlet.
- To switch on, momentarily press the 'Power' button (**H** fig.5). The light (**J**) will come on and will stay on while 'Power' is selected.
- Pressing any of the numbered Hand Control buttons will deliver air to the corresponding numbered hose (follow the user instructions for the Mangar International product that is to be inflated).
- To switch off, momentarily press the 'Power' button (**H**). The battery status indicator light (**D**) will go out after approximately 5 minutes.
NOTE: If the Airflo Plus has not been used for 1 hour, it will switch itself off.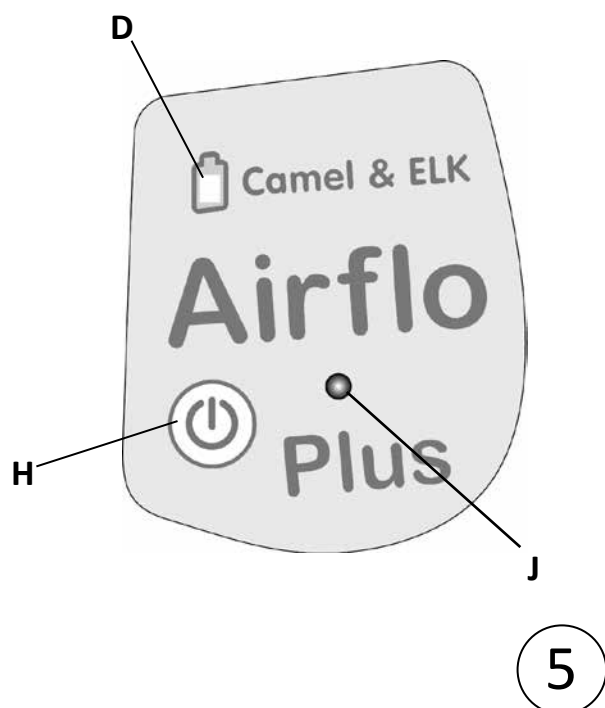

When the Airflo Plus is switched on, the internal microprocessor carries out a battery voltage check each time air is required and if the voltage becomes too low, the Airflo Plus will automatically switch itself off.

Cleaning

Unless contamination is suspected, periodic cleaning is all that is recommended. The Airflo Plus should be cleaned/disinfected as follows:

- Unplug the Airflo Plus from the battery charger. Clean the external surfaces using a non abrasive damp cloth with warm water. Wipe dry thoroughly with a dry towel before reconnecting to the battery charger. Do not allow moisture to enter any part of the compressor.
- Do not use solvent-based products.

Disinfecting

- Pre-clean using a damp cloth and then disinfect using a hard surface alcohol impregnated wipe.

Storage

- When not in use, store in a dry place together with these instructions. To maintain the batteries, when being stored for long periods, put the Airflo Plus on charge overnight every 3 months. NOTE: Ensure that the batteries are fully charged before storing the Airflo Plus for long periods.


Maintenance - fig. ⑥

At approximately 12 month intervals, inspect the air filter located in the base of the Airflo Plus (**A**), and if necessary, fit a new filter (a replacement filter is available through your supplier- part number: CA0074).

Battery replacement: (part number: CA0311)

- Turn the Airflo Plus upside down and remove the battery pack by pulling the handle (**B**).
- Fit the replacement battery pack, ensuring that it 'clicks' into place and is held by the four catches (**C**).

When it becomes apparent that the Airflo Plus is producing fewer than expected lifting cycles, the battery is probably reaching the end of it's working life and should be replaced.

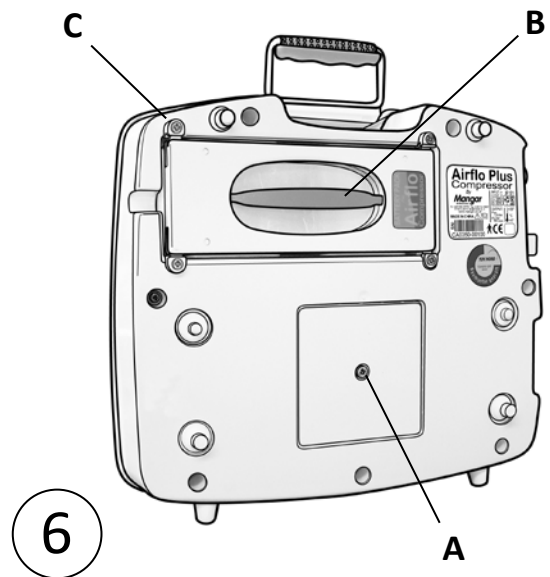The Airflo Plus contains no other user serviceable parts. If a fault should develop or if you require assistance in setting up using or maintaining the equipment, please contact our Customer Service Department on **Freephone 0800 2800 485**. A service manual (MI0338) is available for service personnel.

Disposal

When the Airflo Plus has reached the end of it's working life, remove the batteries for safe disposal in accordance with local regulations, and dispose of the Airflo Plus at your local household waste recycling centre (civic amenity site).

Trouble shooting guide

<p> Compressor does not run when the 4 way hand control buttons are pressed.</p> <p> Ensure that the Airflo Plus has been switched on.</p> <p> Check that there are no kinks in the air hose.</p>	<p> Battery State Indicator light flashing 'Red'.</p> <p> This happens when the compressor's internal temperature cut-out has tripped. Leave the Airflo Plus for 60-90 minutes to cool.</p>
<p> Compressor keeps stopping when trying to inflate the lifting aid.</p> <p> Check that there are no kinks in the air hose.</p>	<p> Battery State Indicator light flashing 'Red/Green'.</p> <p> This happens when the current overload cut-out has tripped. Leave for 3 seconds to self reset.</p> <p> Compressor causes electrical interference with another piece of electrical equipment.</p> <p> Move either the compressor or other item further away from each other until the interference stops.</p>

Specifications

Airflo Plus

Weight:	5.7Kg (12½ lbs)
Output pressure:	0.35 bar (5 psi)
Batteries:	2 x 12v- 2.3Ah (Maintenance free Valve Regulated Sealed Lead Acid)
Battery Life:	Approx 1000 charging cycles (2- 5yrs Depending on usage)
Lifting performance (<i>from full battery</i>):	
	ELK- 3 lifts
	Camel- 1 lift (minimum)
Environmental protection:	IP51 (Protection against large dust deposits & vertical falling drops of water)
Type	BF equipment
This equipment has been tested and meets the requirements of current EMC legislation	

Charger: Model number - CA0431

Input:	AC 100v- 240v, 50-60Hz, 0.3A
Output:	DC 30v, 0.45A
Double insulated Class II equipment	

Symbols used on Airflo Plus and Charger labels

Warning


Attention consult instructions

Type BF equipment (protection against electric shock)


Alternating current (AC)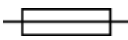

Fuse (with rating)

Direct current (DC)

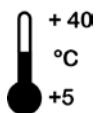Max/Min operating environment temperature


For indoor use only. Do Not connect to mains power in a bathroom or outdoors.

Class II Double Insulated equipment

Dangerous voltage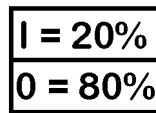

Duty cycle

I = Operating time

O = Non operating time

(Max continuous operating time = 5 minutes)

Recycle

Do not dispose of in household waste

Date of manufacture

Symbols used on packaging

	Keep packaging dry		Recycle packaging
	Handle with care		Do not dispose of in household waste
	Max/Min storage temperature	93% RH 15% RH	Max/Min humidity when transporting or storage
	Net & gross weights	700 mBar 1060 mBar	Max/Min pressure when transporting or storage
	Stack no more than five high	+70° C -25° C	Max/Min temperature when transporting or storage

Warranty

The guarantee is given against defects in material and workmanship and comprises piece, parts and labour. The guarantee is not valid in the case of fair wear and tear, wilful damage, common neglect, misuse or unauthorised alteration or repair by unqualified persons.

The following statements do not affect the consumer’s statutory rights. This product is sold on the understanding that in the event of any defect of manufacture or material appearing within the warranty period, the defect will be rectified free of charge providing that:

- (a) Reasonable evidence is provided (e.g. purchase invoice, delivery note) that the claim is being made within the warranty period.
- (b) The defect is not attributable to accidental damage (either in transit or otherwise), misuse or unauthorised repair.

Airflo Plus serial number:

Date of purchase:

Invoice/Delivery note number:

We strive to provide our customers with reliable and durable products of the highest quality. In addition to our standard warranty, and for added peace of mind, our Customer Service Department offers an **Annual Service and Clean package**.

To find out more about this service please call our Customer Service Department on **Freephone 0800 2800 485**.

Mangar International Ltd trading as Mangar Health, reserves the right to alter product specifications and/or any of the information contained within this document without notice.

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