

Reorder#	Description	Unit
909367	ChairPro Alarm Unit	Set
909366	ChairPro Alarm Unit	10/pk

### Features:

- Low battery alert - alarm sounds when battery is low
- Sounds - two tone buzzer, high (90db), low (80db)
- Battery Test
- Auto-Reset
- Two Interfaces (connections) - Modular (telephone type) for flat sensors and mini-plug for alternative sensors
- Cable strain relief
- Shock resistance bumpers

### Alarm Conditions

#### Condition

Alarm sounds when resident stands

Alarm sounds when modular connector or mini-phone plug are removed, while resident's sitting on sensor



Alarm sounds when sensor pad wire (grey) is cut, while resident is sitting on sensor. Alarm does not sound when auxiliary wire (black) is cut

Alarm sounds when batteries are low

Note: The alarm will not sound when the sensor pad wire (grey) is cut or accidentally pulled out unless it is activated. Also, when the sensor is activated by a resident sitting on it and the plug is accidentally pulled out it will ring but when re-inserted, the alarm will stop ringing. All of the conditions above apply either to the telephone type plug or mini-plug. The alarm sounds only when the mini-plug (black) is removed while sensor is activated. It will not ring when this wire is cut.

### OPERATION

1. Remove battery compartment cover on back of alarm unit and install two AA batteries, LED will light, steady.
2. Press "Battery Test" button. If alarm doesn't sound check battery connection and retest. If alarm fails to sound replace battery.
  - a. This alarm has a low battery alert and an alarm will sound during a "low battery condition".
3. Insert modular connector or mini-phone plug into the side of the alarm (Illustration A). Thread the wire through the "Cable Stain Relief" cavity as shown in (Illustration B) LED will flash. Make sure that there is some slack so that if the cord is tugged it doesn't put pressure on the connector. Alarm will sound. Press "Auto-Reset" button to silence alarm.
4. Before using, test unit by pressing the palm of your hand firmly against the sensor pad. While doing this, check alarm to make certain that LED is flashing. Remove your hand. Alarm should sound. Press "Auto-Reset" button to silence alarm. If alarm fails to sound, make certain the telephone type plug is secure.
5. Select High or Low alarm volume using the switch on right side of alarm, and select Tone 1 or 2
6. When alarm is activated by the resident leaving the wheelchair, it maybe silenced by pressing the "Auto-Reset" button. The alarm is now in stand-by mode. The system will automatically go into the monitoring mode when the resident is reseated. **There is no on/off switch.**

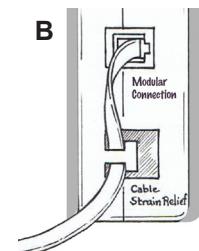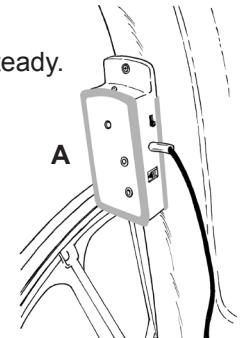

### Safety Reminders

1. Check battery before each use. If alarm is weak or fails to sound, replace the two AA batteries.
2. When unit is in use, check LED regularly. If it is not glowing, replace the two AA batteries.

### LED Conditions


Status	Condition
On Steady	No sensor installed, battery installed
Slow Flash	Sensor connected, battery installed
Rapid Flash	Sensor connected, alarm activated

### INSTALLATION

You may choose one of the three methods to attach the alarm unit to the wheelchair

#### Mounting Bracket

1. Remove the second-from-top screw from either the left or right side of the wheelchair backrest (other locations may be chosen by the caregiver).
2. Remove grommet from screw, place the screw through hole on alarm unit mounting bracket, and fasten screw (with the grommet) to wheelchair backrest (Illustration C).

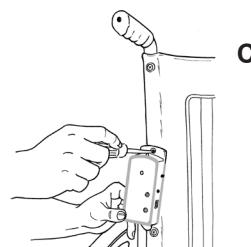


Continued on reverse side

## INSTALLATION (contd.)

### Cord Loop

1. Insert enclosed cord loop through bracket hole (Illustration D).
2. Hang from wheelchair push-handle (Illustration E).

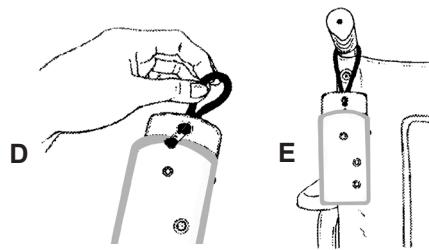


### Velcro® Tab

1. Remove the second-from-top screw from either the left or right side of the wheelchair backrest.
2. Remove grommet from screw. Place screw through eyelet on Velcro® tab. Fasten screw to wheelchair backrest. Make certain that "Hook" side faces away from the wheelchair (Illustration F).
3. Remove backing from Velcro® strip with "Loop" material. Secure adhesive to back of alarm unit, above battery compartment cover (Illustration G).
4. Attach alarm unit to Velcro® tab (Illustration H).



### CAUTION

1. The ChairPro Sensor Pad Alarm is not a substitute for proper supervision. Fall-risk residents/patients should be monitored regularly.
2. Check battery each time alarm is used. If weak, replace the two AA batteries.
3. LED should flash when patient/resident is seated on the sensor pad. If LED fails to flash, check battery. Replace if necessary. If LED does not glow after battery is replaced, check sensor pad. Replace if necessary.
4. If patient/resident is seated on a wheelchair cushion, place sensor pad on top of cushion. This system may not work if sensor is placed beneath cushion.

## BED INSTALLATION

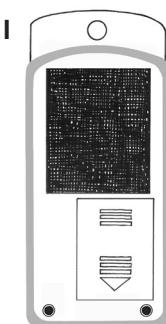

1. Remove backing from enclosed loop material Velcro® strip. Secure adhesive to back of alarm unit above battery compartment cover (Illustration I).
2. Remove backing from enclosed hook material Velcro® strip. Secure adhesive to the side of the headboard that faces the wall (Illustration J) Mount unit to headboard by pressing Velcro® loop material on unit against hook material on headboard (Illustration K).

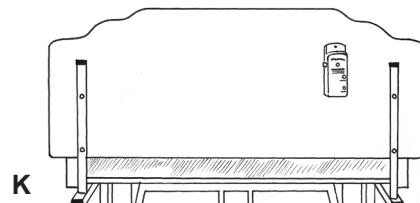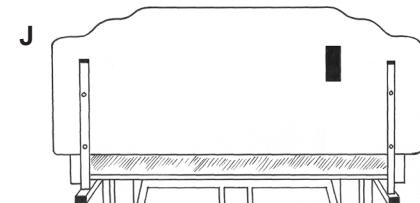

## BED SENSOR PADS