

Tubigrip®



Multi-purpose tubular bandage

[Click here for pricing and more product details](#)

Tubigrip® provides tissue support in the treatment of strains and sprains, soft tissue injuries, general edema, post-burn scarring and ribcage injuries.



How to use Tubigrip®



1. Cut Tubigrip® 10m to twice length required for limb, allowing an extra 2–3cm for overlap.



2. Pull Tubigrip® onto limb like a stocking.



3. Double Tubigrip® back over limb. Ensure upper edge is taken 2–3cm higher up the limb than the first.

How Tubigrip® works

Tubigrip® provides lasting, effective support with complete freedom of movement for the patient. Once the bandage has been applied, covered elastic threads within the fabric move to adjust to the contours of the body and distribute pressure evenly over the surface. Product application is quick and easy.

Tubigrip® stays securely in position without pins or tapes, unlike conventional bandages. Tubigrip® can easily be reapplied after washing, without loss of effect.

Tubigrip® measuring guides are available to help you select the appropriate size and level of support for the condition being treated.

Benefits of Tubigrip®

- Provides comfortable, effective tissue support
- Easy to apply and reapply
- Full range of sizes to suit most applications
- No pins or tapes needed
- Washable and reusable
- Choice of two colors (10m only)
- 10m dispenser style roll reduces waste

Material properties

Viscose, Elastane and Polyamide

Areas of use

Tubigrip® provides tissue support in the treatment of strains and sprains, soft tissue injuries, general edema, post-burn scarring and ribcage injuries.

Caution: This product contains natural rubber latex which may cause allergic reactions, including anaphylactic responses.

Tubigrip® Ordering Information

		Product Code		
		Natural	Beige	Beige
Shade	Size	3.28' (1m)	32.81' (10m)	32.81' (10m)
A	Infant feet and arms	-	1435	-
B	Small hands and arms	1520	1436	1449
C	Medium arms, small ankles	1521	1443	1450
D	Large arms, medium ankles, small knees	1522	1437	1451
E	Large ankles, medium knees, small thighs	1528	1434	1448
F	Large knees, medium thighs	1523	1438	1452
G	Large thighs	-	1439	1453
J	Small trunks	-	1440	-
K	Small/Medium trunks	-	1441	-
L	Large trunks	-	1442	-

Tubigrip® Sizing Guide

Size	Tissue Support Circumference		
	Light	Medium	High
A	-	10–12.5cm/3.9–4.9"	12.5–15cm/4.9–5.9"
B	10–12.5cm/3.9–4.9"	12.5–15cm/4.9–5.9"	15–24.5cm/5.9–9.6"
C	13.5–15cm/5.3–5.9"	15–24.5cm/5.9–9.6"	24.5–35.5cm/9.6–14"
D	15–24.5cm/5.9–9.6"	24.5–35.5cm/9.6–14"	35.5–45cm/14–17.7"
E	24.5–35.5cm/9.6–14"	35.5–45cm/14–17.7"	45–50.4cm/17.7–19.8"
F	35.5–45cm/14–17.7"	45–50.4cm/17.7–19.8"	50.4–60.7cm/19.8–23.9"
G	45–50.4cm/17.7–19.8"	50.4–60.7cm/19.8–23.9"	60.7–70.3cm/23.9–27.7"
J	60.7–70.3cm/23.9–27.7"	70.3–75.5cm/27.7–29.7"	-
K	70.3–75.5cm/27.7–29.7"	-	-

Light tissue support = 5–10 mmHg
 Medium tissue support = 10–20 mmHg
 High tissue support = 20–30 mmHg