


8" ALTERNATING PRESSURE MATTRESS

QUICK START GUIDE





Want to watch us do this instead? Just go to the following link to see how to set up and use your Alternating Pressure Mattress. Visit vhealth.link/6m9

1. Loosely secure the Mattress to the bed frame using the two (2) straps and rings. Do not tighten.
2. Connect the air hoses on the Mattress to the inflating nozzle on the Compressor Pump. You should hear a click to tell that it is connected.
3. Plug the pump into the wall outlet and press the Power  button to begin inflating the Mattress. Then enter the patient's weight with the **+** and **-** buttons. Inflation time is about forty (40) minutes.
4. Once inflated, Unlock the controls tighten the two (2) straps to secure the Mattress to the bed frame. You can lay a bed sheet (twin size) over the mattress if desired.



OTHER INFORMATION

- The cells interchangeably alternate every ten (10) minutes.
- In the event of a power failure, the Pump will set off an alarm to alert a caregiver, press the  button to reset.
- The pump automatically locks after two (2) minutes of inactivity. Press and hold  Button for three (3) seconds to unlock and lock.

vive®