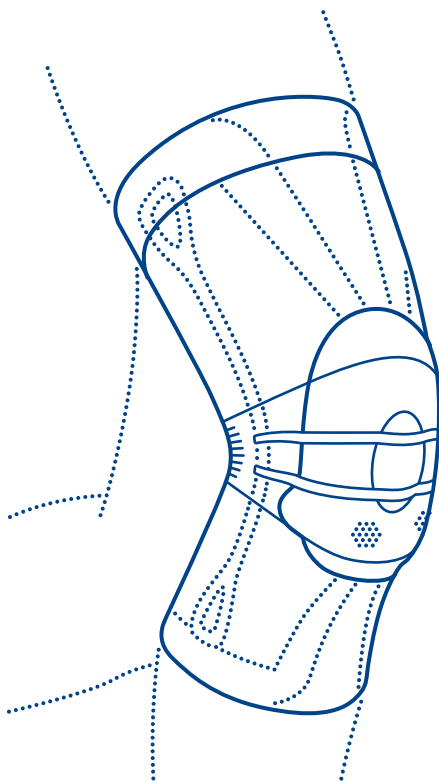
GenuTrain[®]

KNIEBANDAGE

KNEE SUPPORT

GENOUILÈRE

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Dear customer,
thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

The GenuTrain is a medical device. It is a support for the relief and stabilization of the knee joint.

Indications

- Tendinopathy, tendomyopathy, ligament insertion degeneration, meniscopathy
- Chronic, post-traumatic or post-operative irritation
- Recurring joint effusion
- Gonarthrosis (joint wear)
- Osteoarthritis (joint inflammation)
- (Relapse) prevention
- Feeling of instability

Risks of using this product

Please closely observe the specifications in these instructions for use and the instructions given by the specialist.

The benefits of this product are felt particularly during physical activity.

- Please remove the support during longer periods of rest.
- Once you have been prescribed the GenuTrain, only use it according to your indication and any additional instructions given by a medical specialist*. Please consult a specialist or your doctor first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Contraindications

No clinically significant adverse reactions have been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present; likewise, any raised scars with swelling, redness, and excessive heat build-up
- Varicose veins (varicosis)
- Impaired sensation in the leg due to neurological abnormalities
- Impaired blood flow in the leg (e.g. with diabetes mellitus)
- Impaired lymphatic drainage and soft tissue swelling of unknown origin

* A specialist is any person who is authorized, according to their state's regulations, to fit supports and orthoses and provide instructions on how to use them.

Application instructions

Putting on the GenuTrain

- Grip the donning aids at the top ends of the stays on both sides and pull the support up over the knee.
- Position the Omega+ Pad (ring-shaped pad) in such a way that it encloses the kneecap without exerting pressure. Image   

Removing the GenuTrain

- Grip the donning aids at the bottom ends of the stays on both sides and pull the support downwards. Image 

Make sure not to turn the support inside-out.

Cleaning instructions

Please use mild detergent and a lingerie bag. Never expose the product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the GenuTrain have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Styrene-ethylene-butylene-styrene (SEBS), Thermoplastic polyurethane (TPU), Spandex (EL), Cotton (CO), Polyurethane (PUR), Silicone (SI), Polyester (PES)

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Bauerfeind USA, Inc.
BAUERFEIND AG