# FOLDING WALKER

## **QUICK START GUIDE**

#### WHAT'S IN THE PACKAGE

- 1x Folding Walker
- 1x Removable Walker Bag
- 4x Adjustable Leg Sections (pre-installed)
- 2x Adjustable Wheel Legs

#### **OVERVIEW**

We are constantly answering questions and recording helpful videos to make using your Vive Mobility Folding Walker as easy as possible. Check out the included QR code and video link to help you through the process.



To see all FAQs in one place visit **vhealth.link/jpr** 

#### **OPERATING INSTRUCTIONS**

Product is fully assembled and ready to use. (This item will require no tools to assemble.)



For a video tutorial on folding the walker, check out **vhealth.link/z3e** 

To begin folding the walker, simply press down on each rubber tab located on the center support bar to unlock each side. Then pull the back of each side in towards the center of the walker to fold the walker.

To adjust the length of the legs, press down on the rounded metal button on the leg to unlock the leg section. Then push/pull the leg to the desired length, and align the depressed button with the adjustment hole corresponding to the desired length. The button will click back into place and lock the leg in the desired height. Repeat with all 4 legs and adjust to an even height that is stable and comfortable to use.

#### Walking with the Walker

- Align the middle of your foot with the back legs of the walker.
- Lift the walker up and place it a step's distance in front of you, making sure all 4 legs are stable on the ground.
- Carefully walk forward one step, and repeat.

#### **SPECIFICATIONS**

- Width 20"
- Depth 16'
- Adjustable height 32" 38"
- Product Weight 6.5 LBS
- Max User Weight 250 LBS

### **WARNINGS**

- Do not push one side of the walker ahead and then other side without lifting the walker first.
- Use Vive accessories only. Use of other manufacturers' accessories can result in improper fit and risk fall and/or injury.
- Do not use on stairs or escalators.



Distributed by

