English

Male Intermittent Urinary Catheters (PVC)

R_x
ONLY

INDICATIONS

Intermittent catheters are indicated for routine short term drainage of the bladder. The catheter is inserted through the urethra.

CAUTIONS, PRECAUTIONS & OBSERVATIONS

- Check the catheter's integrity prior to insertion.
- Select a replacement catheter if required.
- Do not use if package has been previously opened or damaged.
- Do not touch the catheter tip before insertion.
- If there is no urine flow or other complications occur (e.g. pain, difficult insertion or removal, bleeding, signs of urinary tract infection or discharge) please consult a healthcare professional.
- After incorrect insertion, catheters must be discarded and not re-used. A new catheter should be selected. Used catheters should always be discarded.

INSTRUCTIONS FOR MALE USE

NOTE: Healthcare professionals use aseptic technique

1. Wash hands thoroughly with soap and water. *(Figure 1)*
2. Withdraw the foreskin and wash the penis with soap and water. *(Figure 2)*
3. Peel open the package. *(Figure 3)*
4. Lubricate the catheter with water-soluble lubricant. *(Figure 4)*

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